



Fixed Wheel Attachment

Single with Glide Tips


Model 6270 - 3" Wheel
Model 6271 - 5" Wheel

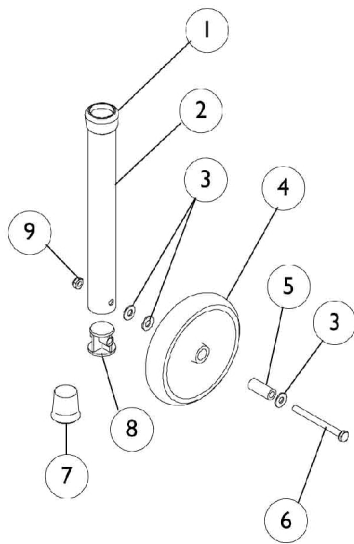

Model 6277 - Tall

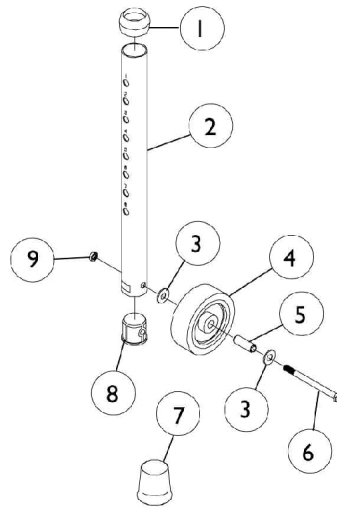

Model 6278 with
13" Rear Leg Extension

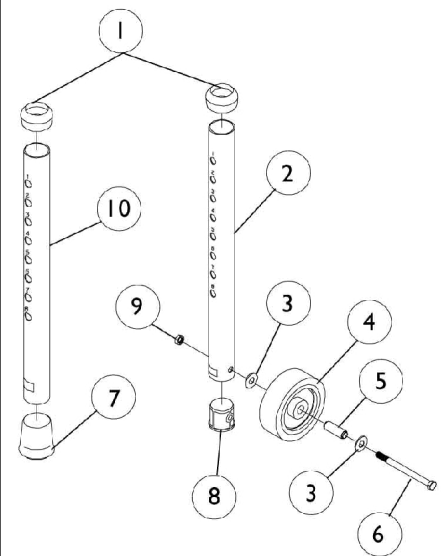

Attachments Sold in Pairs Single Side Shown

Fixed Wheel Attachment Single with Glide Tips

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A,B,C	1040655	Anti-Rattle, Grey (1-1/8")	1	4
2		1050684	Tube, Leg Extension (13")	1	2
3		1040292-NLA	NLA - Washer, Steel Flat (1/4 x 5/8 x 1/16")	1	6
4	C,D,E,J	1039466-NLA	NLA - Wheel (3" x 1")	1	2
4	C,D,E	1130782-NLA	NLA - Package, Wheel (3")	10	1
4	B,G	1123534	Wheel (5" x 1-1/4")	1	2
5	A,C,E	1039465	Bushing (1/4 x 3/8 x 1/13")	1	2
5	B	1106067	Bushing (1/4 x 1-3/8")	1	2
6		1014412	Screw, Hex Head (1/4-20 x 2-1/2")	1	2
6	B	1008496	Screw, Hex Head (1/4-20 x 2-3/4")	1	2
7	F,H	6280	Package, Tip Glide, Grey (1-1/8")	24	1
7	F,I	1041531-NLA	NLA - Tip, Glide, Grey (1-1/8")	1	2
7		1130781	Kit, Glide Tips	10	1
8		1040278	Bushing (1")	1	2
9		60275X000	Locknut (1/4-20)	1	2
10	E	1040671	Tube, Leg Extension (13")	1	2

NOTE: A - To be used with Model 6270.
 B - To be used with Model 6271.
 C - To be used with Model 6277.
 D - To be used with Model 9270.
 E - To be used with Model 6278.
 F - Used on Rear Legs (1-1/8").
 G - Due to style change, "After 2/6/02" Wheels should be ordered in pairs.
 H - Includes Instruction Sheet.
 I - Individual item no longer available, refer to Kit Part #1130781.
 J - Individual item no longer available, refer to Kit Part #1130782.

NOTE: Due to 5" Wheel style change, please order in pairs after 2-6-02 for Model 6271.