


Arm/Horn Assembly - RPS350-2 (Before 7/2/10)

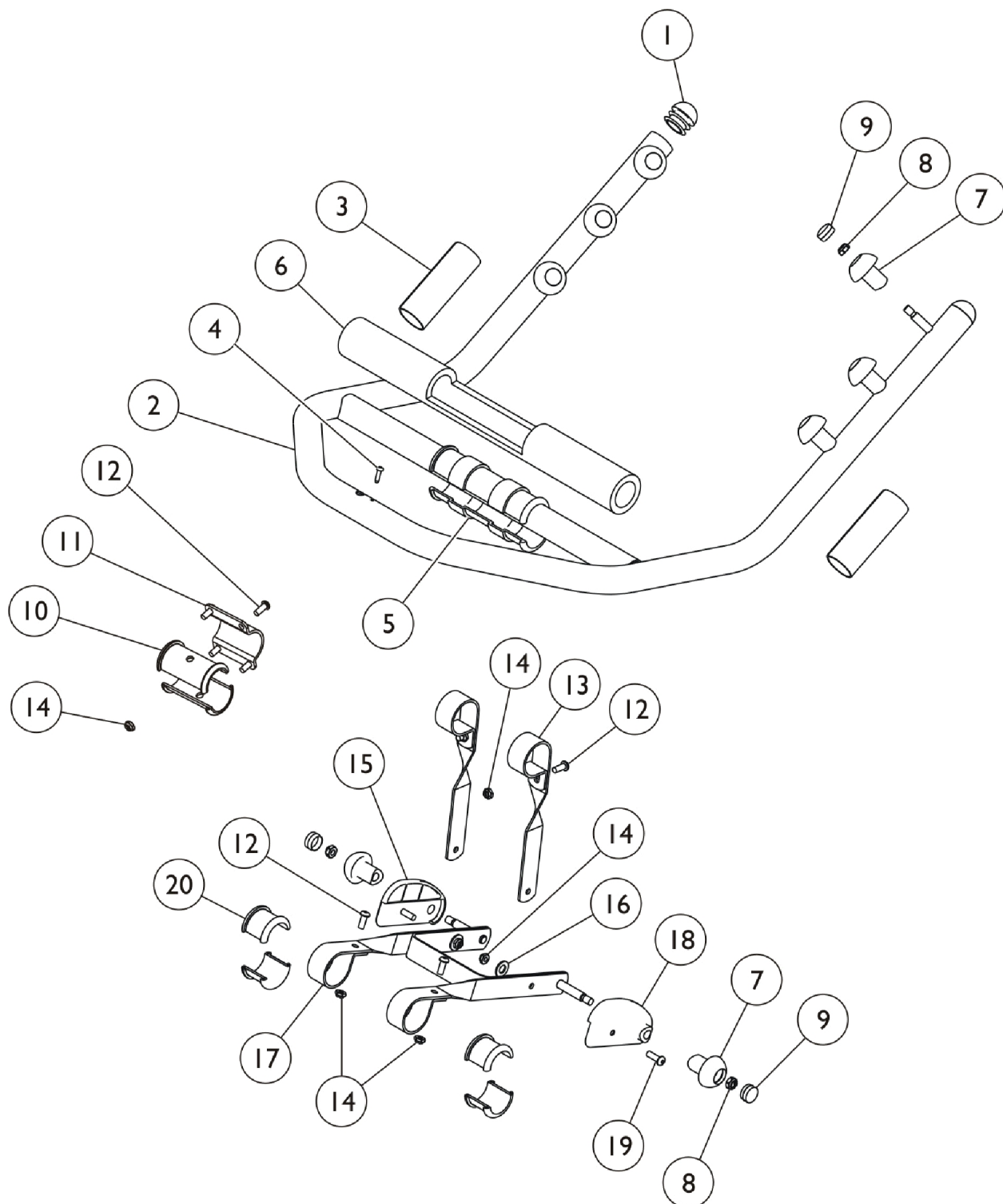

Arm/Horn Assembly - RPS350-2 (Before 7/2/10)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1071249	Plug Button, Dome, Black (1-1/4")	1	2
2		1071091-NLA	NLA - Arm/Horn Tube Assembly	1	1
3		1076388	Pad, Grip	1	2
4		12642X000	Screw (#8-18 x 5/8")	1	1
5		1070180-NLA	NLA - Bushing, Drive Lift	1	2
6		1070969	Bar, Pad	1	1
7		1070177	Knob, Hook, Black	1	8
8		1028769	Locknut (5/16-18)	1	8
9		1070902	Cap, Hook	1	8
10		1070175	Bushing, Horn	1	2
11		1071834	Bracket, Horn, Grey	1	1
12		1007269	Screw, Socket Button Head (1/4-20 x 5/8")	1	8
13		1071835-NLA	NLA - Bracket, Linkage, Grey	1	2
14		1025195	Locknut (1/4-20)	1	10
15		1071831-NLA	NLA - Spacer, Guard - Left	1	1
16		1049527	Washer, Nylon (3/8 x 3/4 x 1/16")	1	2
17		1071154-NLA	NLA - Linkage Bracket Assembly	1	1
18		1071830-NLA	NLA - Spacer, Guard - Right	1	1
19		1009838	Screw, Socket Button (1/4-20 x 3/4")	1	2
20		1070176	Bushing, Handle	1	4