


Seat Frame Assembly (Adult 16"-24" Wide)

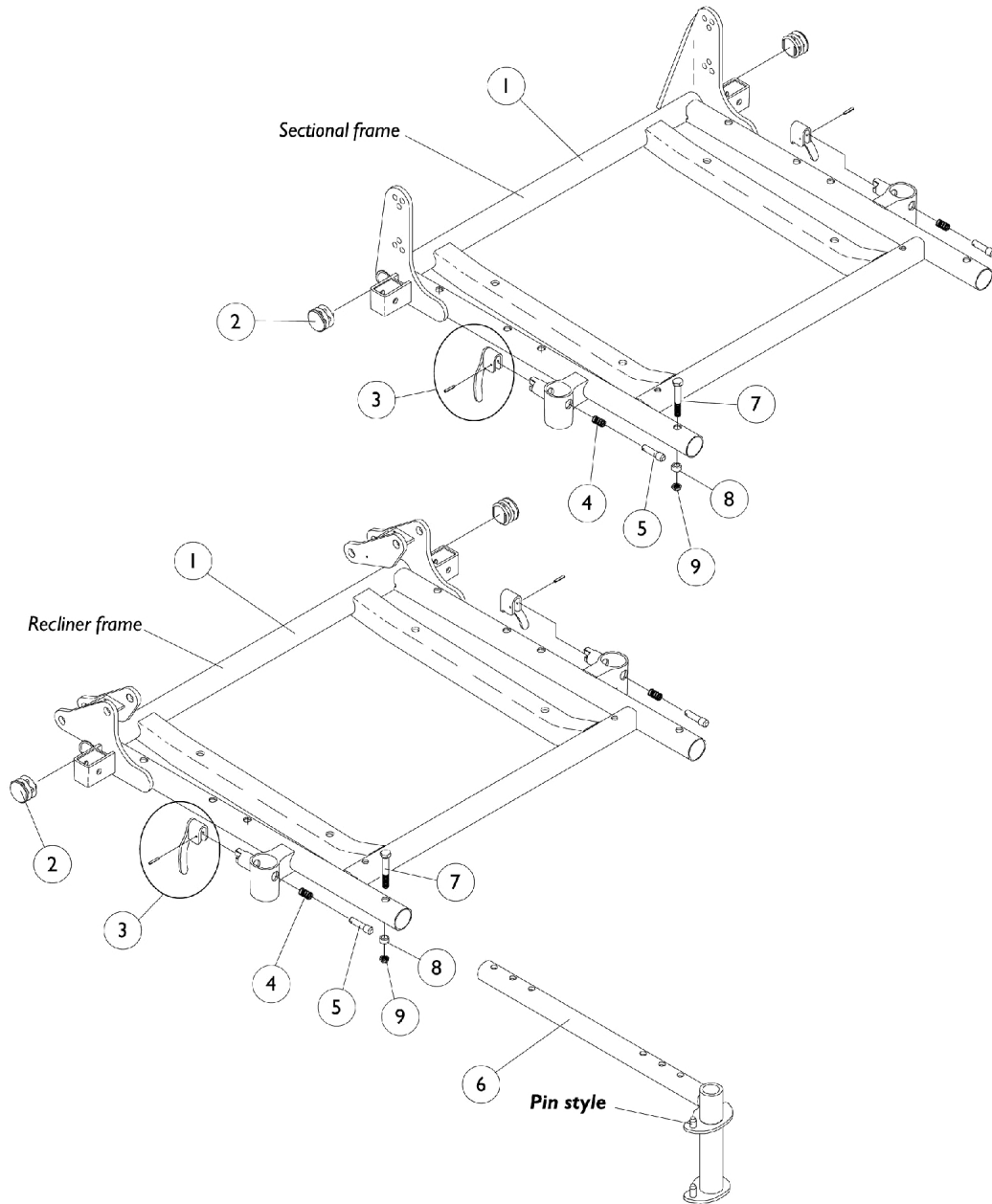

Seat Frame Assembly (Adult 16"-24" Wide)

#1. Seat Frame with Hardware

Sectional

Seat Width	16"-17" Depth	18"-19" Depth	20"-21" Depth	22" Depth
16" Width	1070672-NLA	1070681-NLA	1070690-NLA	1070699-NLA
17" Width	1070673-NLA	1070682-NLA	1070691-NLA	1070700-NLA
18" Width	1070674-NLA	1070683-NLA	1070692-NLA	1070701-NLA
19" Width	1070675-NLA	1070684-NLA	1070693-NLA	1070702-NLA
20" Width	1070676-NLA	1070685-NLA	1070694-NLA	1070703-NLA
21" Width	1070677-NLA	1070686-NLA	1070695-NLA	1070704-NLA
22" Width	1070678-NLA	1070687-NLA	1070696-NLA	1070705-NLA
23" Width	1070679-NLA	1070688-NLA	1070697-NLA	1070706-NLA
24" Width	1070680-NLA	1070689-NLA	1070698-NLA	1070707-NLA

#1. Seat Frame with Hardware

Recliner

Seat Width	16"-17" Depth	18"-19" Depth	20"-21" Depth	22" Depth
16" Width	1070708-NLA	1070717-NLA	1070726-NLA	1070735-NLA
17" Width	1070709-NLA	1070718-NLA	1070727-NLA	1070736-NLA
18" Width	1070710-NLA	1070719-NLA	1070728-NLA	1070737-NLA
19" Width	1070711-NLA	1070720-NLA	1070729-NLA	1070738-NLA
20" Width	1070712-NLA	1070721-NLA	1070730-NLA	1070739-NLA
21" Width	1070713-NLA	1070722-NLA	1070731-NLA	1070740-NLA
22" Width	1070714-NLA	1070723-NLA	1070732-NLA	1070741-NLA
23" Width	1070715-NLA	1070724-NLA	1070733-NLA	1070742-NLA
24" Width	1070716-NLA	1070725-NLA	1070734-NLA	1070743-NLA

Seat Frame Assembly (Adult 16"-24" Wide)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		See Chart	Item #1 Seat Frame with Hardware - Recliner	1	1
1		See Chart	Item #1 Seat Frame with Hardware - Sectional	1	1
2		1029501	Plug Button, Black (1")	1	2
3		1090768	Kit, Lock Handle w/Rivets (Pair)	2	1
4		47471X074	Spring, Arm Lock	1	2
5		1028594	Pin, Arm Lock, Black	1	2
6	B	1070356-NLA	NLA - Hanger Assembly, Front Rigging, Pin Style, Black - Right (12 - 21" D)	1	2
6	B	1070357-NLA	NLA - Hanger Assembly, Front Rigging, Pin Style, Black - Left (12 - 21" D)	1	2
6	C	1070358-NLA	NLA - Hanger Assembly, Front Rigging, Pin Style, Black - Right (22" D)	1	2
6	C	1070359-NLA	NLA - Hanger Assembly, Front Rigging, Pin Style, Black - Left (22" D)	1	2
7	A	1030949	Screw, Hex Head (1/4-20 x 1-1/2")	1	2
8	A	1047449-NLA	NLA - Spacer (1/4 x 3/8 x 3/16")	1	2
9	A	7930333	Locknut (1/4-20)	25	2
NOTE: A - This item not included with seat frame hardware. B - 16 to 21" Deep C - 22" Deep					