



Front Riggings - Myon Junior Elevating Legrest Hangers and Calf Pad

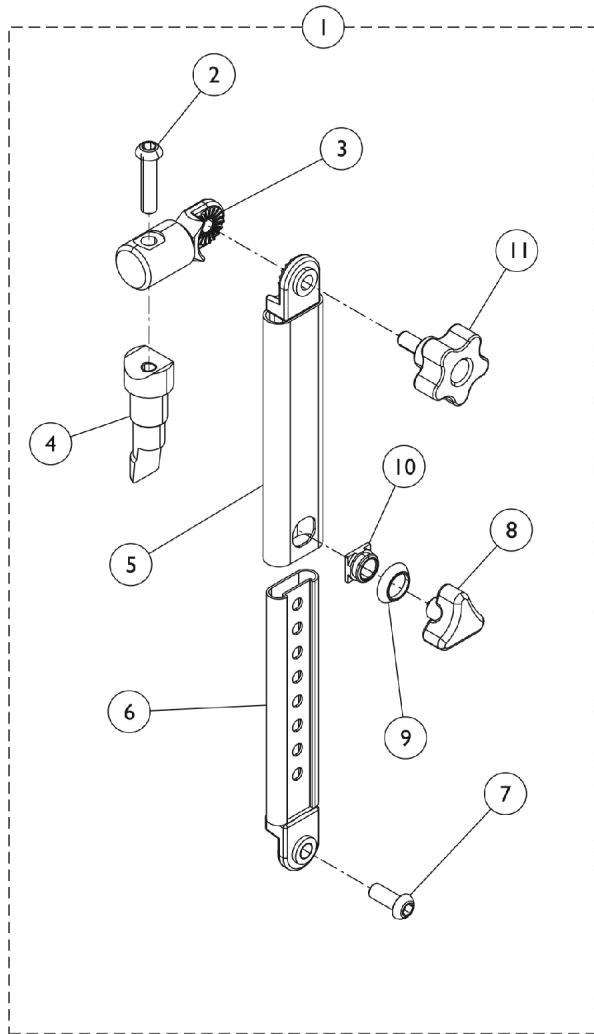

Elevating Legrest Hanger

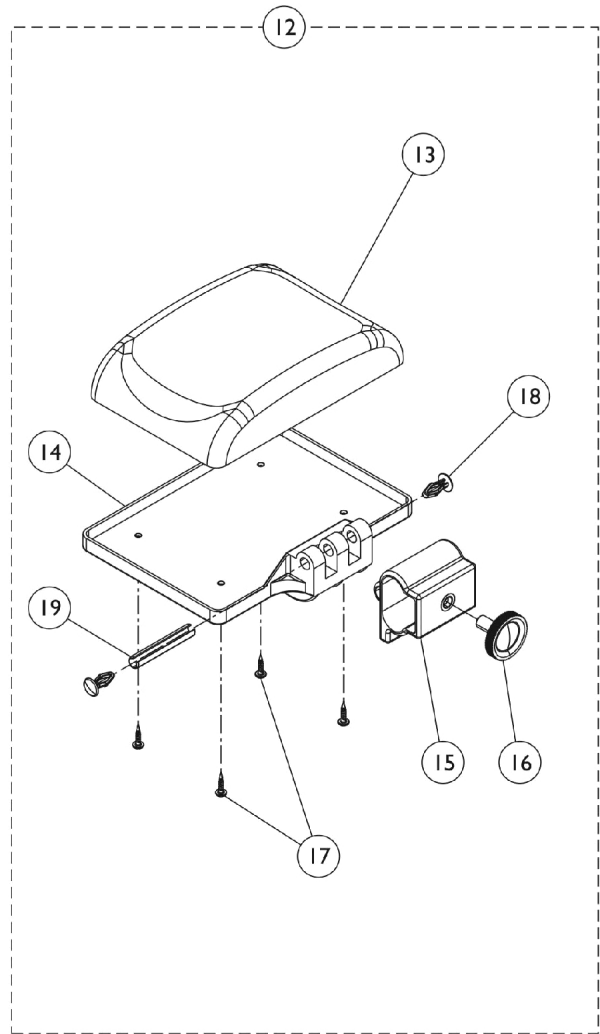

Elevating Legrest Calfpad

Front Riggings - Myon Junior Elevating Legrest Hangers and Calf Pad

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1450950	Hanger Assembly, Elevating Legrest (12" W)	1	2
2			Screw (M8 x 35mm)	1	1
3			Support, Hanger, Elevated Legrest	1	1
4			Indexer, Hanger, 90 Degree - Myon Junior	1	1
5			Tube, Upper, Elevating Legrest (12" W)	1	1
6			Tube, Lower, Elevating Legrest (12" W)	1	1
7			Screw, CBHC (M8 x 20mm)	1	1
8			Screw, Ball (7.5 mm)	1	1
9			Bushing, Armrest	1	1
10			Nut, Square, Armrest	1	1
1		1428808	Hanger Assembly, Elevating Leg Rest (13 - 15" W)	1	2
2			Screw (M8 x 35mm)	1	1
3			Support, Hanger, Elevated Legrest	1	1
4			Indexer, Hanger, 90 Degree - Myon Junior	1	1
5			Tube, Upper, Elevating Legrest (13 - 14" W)	1	1
6			Tube, Lower, Elevating Legrest (13 - 14" W)	1	1
7			Screw, CBHC (M8 x 20mm)	1	1
8			Screw, Ball (7.5 mm)	1	1
9			Bushing, Armrest	1	1
10			Nut, Square, Armrest	1	1
12		1403981	Calf Pad Assembly w/ Hardware (12 - 15" W)	1	1
13			Calf Pad (15 - 17" W)	1	1
14			Plate, Attaching, Calf Pad (15 - 17" W)	1	1
15			Mounting, Calf Pad Back	1	1
16			Button, Tightening	1	1
17			Screw, VBA (M3.5 x 15)	1	4
18			Clamp, Calf Pad Plate	1	2
19		Pin, Spring (M8 x 50mm)	1	1	