



# Captain Seat Frame and Actuator Hardware Before 11/01/01

NOTE: Low Back (L.B.) Captain Seat Description changed to Extended Back (E.B.) Captain Seat as of 10/02/00.

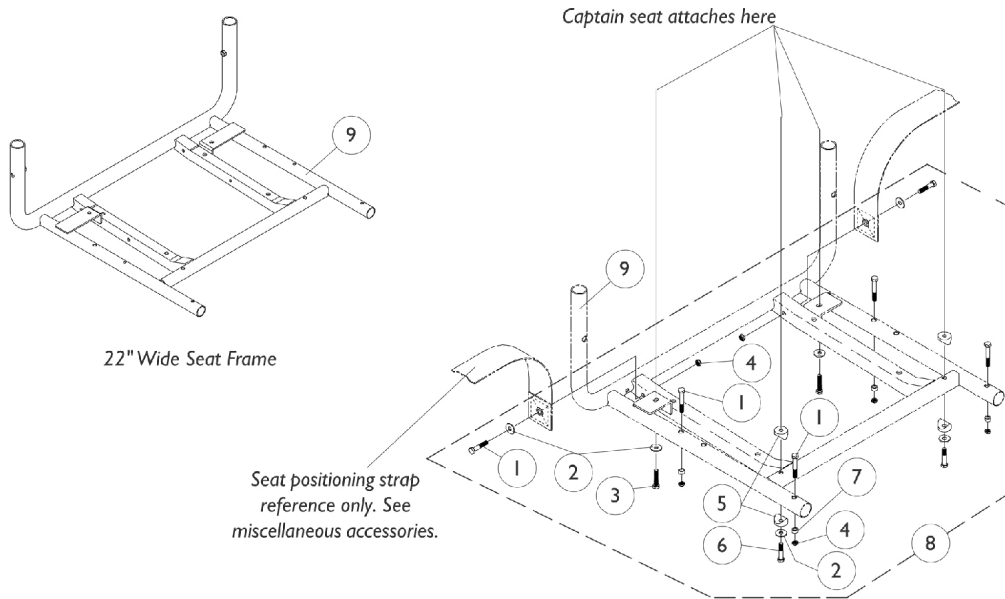

Upper/lower actuator brackets are non-serviceable

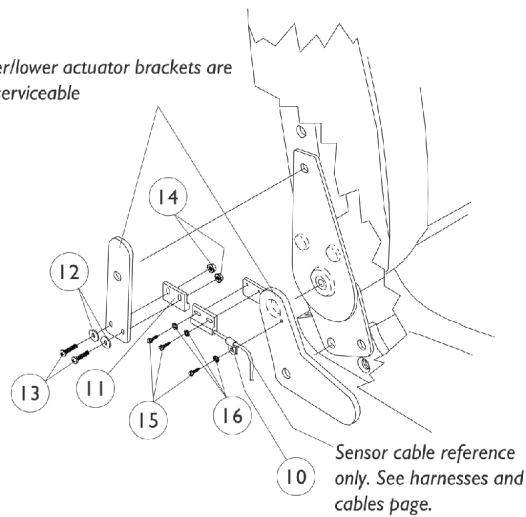

**Actuator Attaching Hardware  
(High Back 18" Wide Models Only)**

# Captain Seat Frame and Actuator Hardware

## Before 11/01/01

**NOTE: Low Back (L.B.) Captain Seat Description changed to Extended Back (E.B.) Captain Seat as of 10/02/00.**

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1030949	Screw, Hex Head (1/4-20 x 1-1/2")	1	6
2		1026900	Washer (1/4 x 23/32 x 1/16")	1	6
3	C	1071603-NLA	NLA - Screw, Hex Head w/ Patch (M6 x 1 x 20mm)	1	4
3	A	1045267	Screw, Hex Head w/ Patch (1/4-20 x 3/4")	1	4
4		7930333	Locknut (1/4-20)	25	6
5		1026099	Spacer, Coved (1/4 x 11/16 x 1/2")	1	4
6	C	1070434-NLA	nla - Screw, Hex Head w/ Patch (M6 x 1 x 50mm)	1	2
6	A	1088898-NLA	NLA - Screw, Hex Head w/ Patch (1/4-20 x 1-3/4")	1	2
7		1047449-NLA	NLA - Spacer (1/4 x 3/8 x 3/16")	1	4
8	B,C	1070450-NLA	NLA - Hardware, Seat Attaching	1	1
8	A,B	1079208-NLA	NLA - Hardware, Captain Seat Mounting	1	1
9	D	1070455-NLA	NLA - Frame, Seat, Black (18" W)	1	1
9	E	1101913-NLA	NLA - Frame, Seat, Black (22" W)	1	1
10		1098624-NLA	NLA - Clamp, Cable	1	1
11		1024001	Sensor, Actuator Switch	1	1
12		16348X007	Rivet Washer (1/8 x 3/8 x 1/16")	1	2
13		1074128	Screw, Phillips Pan Head (#4-40 x 7/16")	1	2
14		P0870X000	Nut, Hex Keps (#4-40)	1	2
15		1025116-NLA	NLA - Screw, Slotted Hex Washer Head Tap (#4-3/8")	1	3
16		1008525-NLA	NLA - Washer, Internal Tooth Lock (#4-40)	1	3

NOTE: A - Extended back, forward folding 18" Wide and High Backs 22" Wide  
 B - Consists of items 1-7  
 C - All high backs 18" wide  
 D - Extended Back, Forward Folding and High Backs 18" Wide  
 E - High Back 22" Wide