
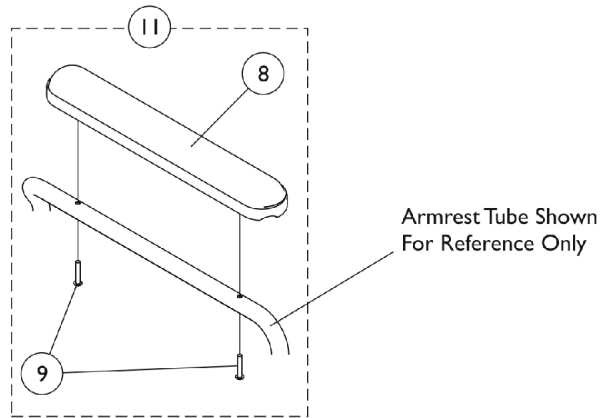
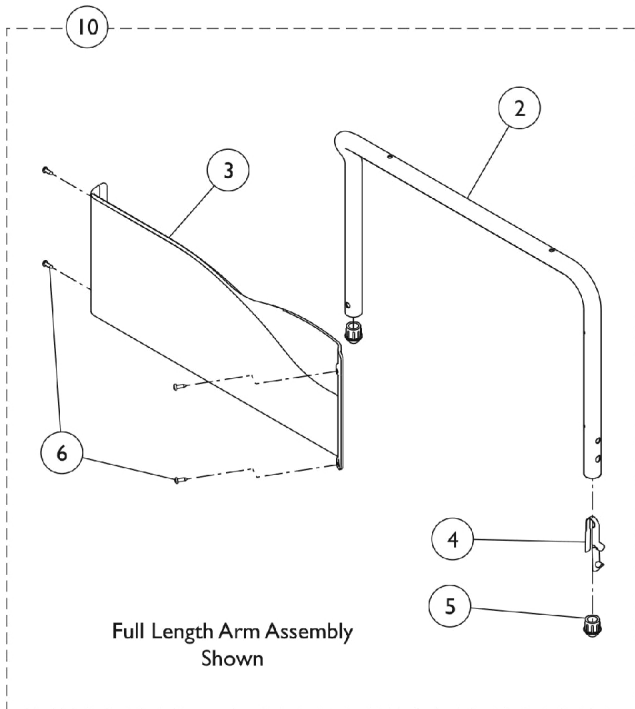
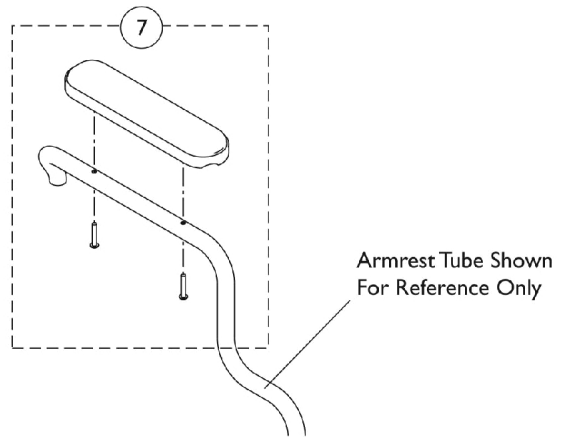


Space Saver Fixed Height Arms

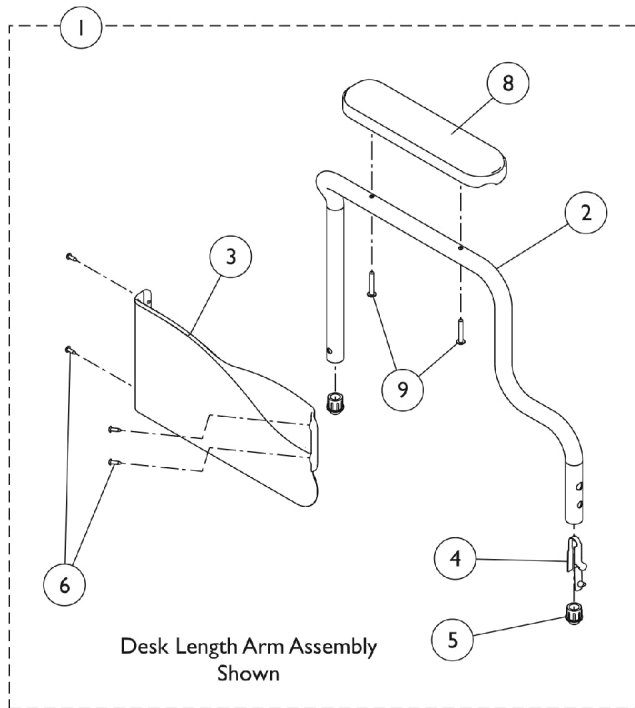

Space Saver Fixed Height Arms

Item Number	Note	Part Number	Description	Quantity Per			
				Package	Assembly		
1	C,E	8881130858	Kit, Armrest, Desk Length, Black - Right (Style 88)	1	1		
2			Tube, Armrest, Space Saver, Black - Right	1	1		
3			Guard, Clothing, Desk Length - Right	1	1		
4			Spring, Torsion w/ Button Lock	1	1		
5			Plug, Armrest, Black (3/4") (18GA)	1	2		
6			Screw, Self Tapping (#6-20 x 1/2")	1	4		
8			Arm Pad, Desk Length w/ Law Label	1	1		
9			Package, Screw Phillips Pan Head (#10-32 x 1-1/8")	1	2		
1			C,E	8881130859	Kit, Armrest, Desk Length, Black - Left (Style 88)	1	1
2	Tube, Armrest, Space Saver, Black - Left	1			1		
3	Guard, Clothing, Desk Length - Left	1			1		
4	Spring, Torsion w/ Button Lock	1			1		
5	Plug, Armrest, Black (3/4") (18GA)	1			2		
6	Screw, Self Tapping (#6-20 x 1/2")	1			4		
8	Arm Pad, Desk Length w/ Law Label	1			1		
9	Package, Screw Phillips Pan Head (#10-32 x 1-1/8")	1			2		
1	C,F	1057705			Arm Assembly w/o Arm Pad, Full Length, Black - Right	1	1
2			Armrest, Space-Saver, Full, Black - Right	1	1		
3			Guard, Clothing, Full Length - Right	1	1		
4			Spring, Torsion w/ Button Lock	1	1		
5			Plug, Armrest, Black (3/4") (18GA)	1	2		
6			Screw, Self Tapping (#6-20 x 1/2")	1	4		
11	C,F	1057706	Arm Assembly w/o Arm Pad, Full Length, Black - Left	1	1		
2			Armrest, Space-Saver, Full, Black - Left	1	1		
3			Guard, Clothing, Full Length - Left	1	1		
4			Spring, Torsion w/ Button Lock	1	1		
5			Plug, Armrest, Black (3/4") (18GA)	1	2		
6			Screw, Self Tapping (#6-20 x 1/2")	1	4		
7	A,G	8881053570	Arm Pad, Desk Length w/ Law Label	1	2		
			Package, Screw Phillips Pan Head (#10-32 x 1-1/8")	2	1		
11			A,H	8881053571	Arm Pad, Desk Length w/ Law Label	1	2
8	Package, Screw Phillips Pan Head (#10-32 x 1-1/8")	2			1		
9	D	1101538-NLA			NLA - Decal, Armrest	1	1
			D,B	1123307	Kit, Clothing Guard, Space Saver, Desk Length w/ Screws and Decals (Pair)	2	1
					D,B	1123308	Kit, Clothing Guard, Space Saver, Full Length w/ Screws & Decals (Pair)

Space Saver Fixed Height Arms

NOTE: A - Specify U67 upholstery color when ordering.
B - Clothing Guards not sold individually. Kit includes clothing guards, screws, and decals.
C - Due to Clothing Guard change, it is recommend (for chairs built Before 11/25/03) when only one Arm is requested the appropriate Clothing Guard Kit (item 7) be ordered also so that the arms will match.
D - Not Shown
E - Includes: Items 2 - 6, 8, and 9.
F - Includes: items 2 - 6.
G - Includes Arm Pad and Screws.
H - Includes items 8 and 9.