


Lower Tilt Assembly

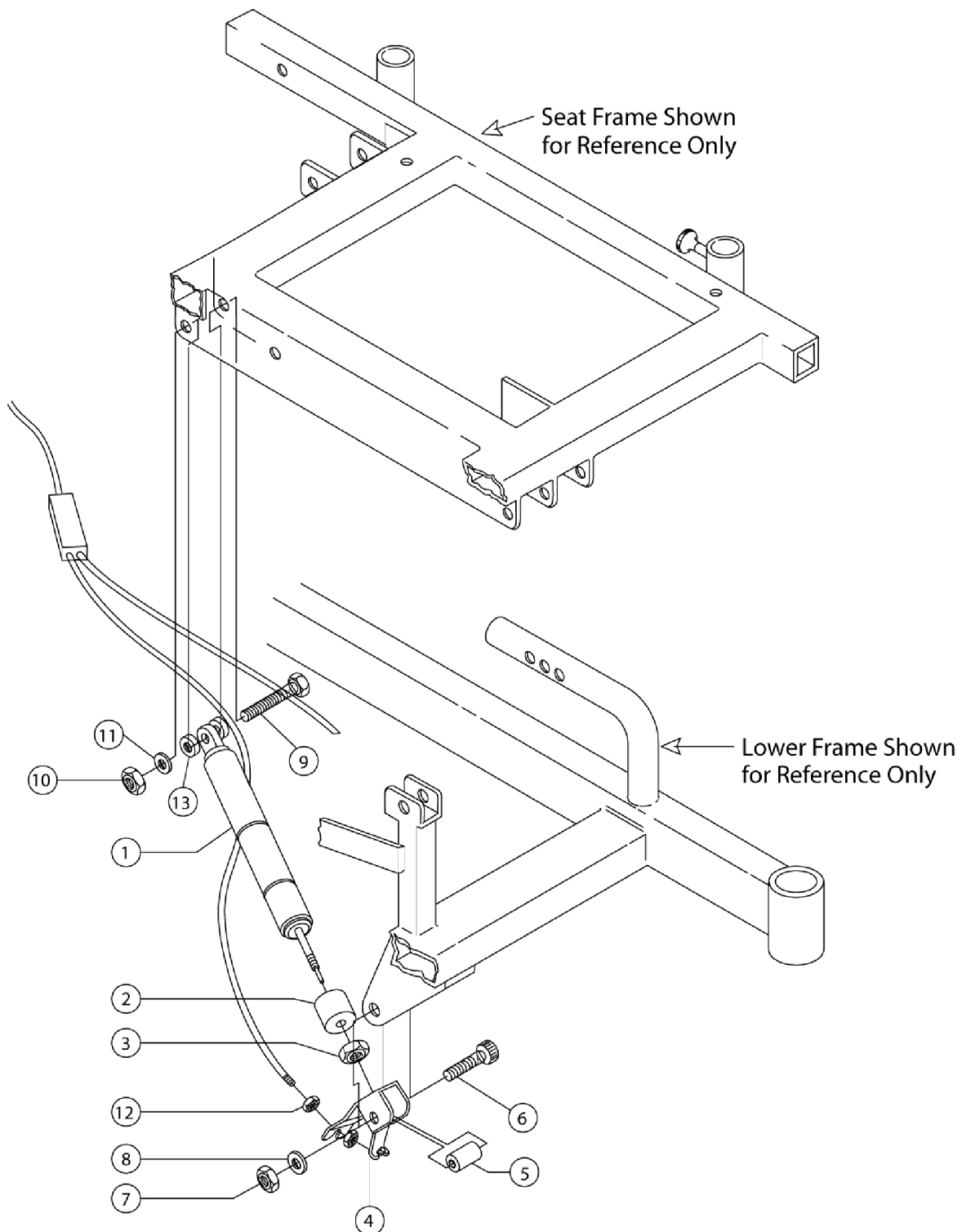

Lower Tilt Assembly

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1-13		440140	Kit, Gas Strut and Hardware	--	1
1		1082422	Strut, Gas, Black	1	--
2		440145	Spring, Elastomer	1	--
3		361572	Locknut, Steel Zinc (M10 x 1)	1	--
4		TR432	Activator, Strut	1	--
5		1132396	Spacer, Clear (13/32 x 1/2 x 27/32")	1	--
6		360114	Screw, Shoulder Hex Socket (5/16 x 1-1/4")	1	--
7		1025195	Locknut (1/4-20)	1	--
8		319476	Washer	1	--
9		360135	Screw, Shoulder Hex Socket (3/8 x 1-1/2")	1	--
10		1026158	Locknut (5/16-18)	1	--
11		1025704	Washer, Flat (5/16 x 5/8 x 1/16")	1	--
12		R2759	Locknut, Steel Zinc (M6)	2	--
13		440152	Spacer, Gas Strut (7/16 x 1/2 x 1-15/16")	2	--