


Wheel Lock Assembly

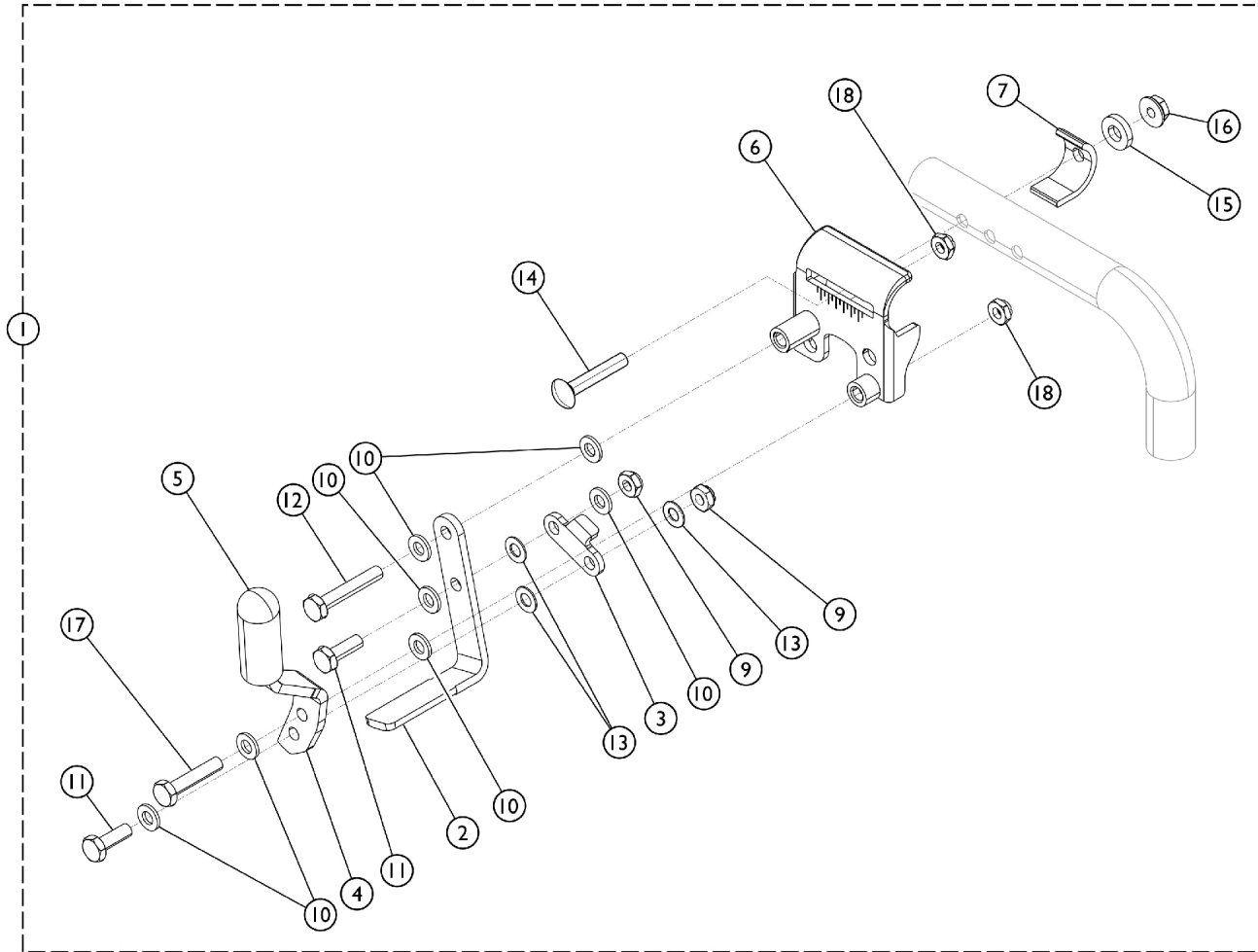

Wheel Lock Assembly

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1101452	Wheel Lock Assembly, Push to Lock, Black - Right	--	1
1	A	1101453	Wheel Lock Assembly, Push to Lock, Black - Left	--	1
1	A	1031796	Wheel Lock Assembly, Pull to Lock, Hemi - Right	--	1
1	A	1031797	Wheel Lock Assembly, Pull to Lock, Hemi - Left	--	1
2		1021405-NLA	NLA - Shoe, Wheel Lock	2	--
3		1021404-NLA	NLA - Link, Black	2	--
4		1021408-NLA	NLA - Handle, Wheel Lock, Black - Right	1	--
		1021409-NLA	NLA - Handle, Wheel Lock, Black - Left	1	--
5		47376x024	Wheel Lock Tip - Black	2	--
6		1031787-NLA	NLA - Bracket, Mounting, Black - Right	1	--
		1031788-NLA	NLA - Bracket, Mounting, Black - Left	1	--
7		1031777	Bracket, Back Up, Black	2	--
8		1026105-NLA	NLA - Spacer, Nylon (1/4 x 7/16 x 1/8")	2	--
9		1025765	Locknut (1/4-28)	2	--
10		1026033	Washer, Nylon (1/4 x 1/2 x 1/16")	6	--
11		1028687	Screw, Hex Head (1/4-28 x 3/4")	4	--
12		1021406	Screw, Hex Head (1/4-20 x 1-1/4")	2	--
13		1026103	Washer, Nylon (1/4 x 1/2 x 1/32")	6	--
14		1021402	Bolt, Carriage (1/4-20 x 1-1/4")	2	--
15		319476	Washer	2	--
16		1021403	Nut, Hex Flang (1/4-20)	2	--
17		1030949	Screw, Hex Head (1/4-20 x 1-1/2")	2	--
18		1025195	Locknut (1/4-20)	4	--
	B	1007319	Wheel Lock Extension Handle Assembly, Black (6")	1	--
	B	1038599	Wheel Lock Extension Handle Assembly, Black (6") (Pair)	2	--

NOTE: A - Includes items 1 - 18.
 B - Not shown in illustration.