


Legrest Hangers

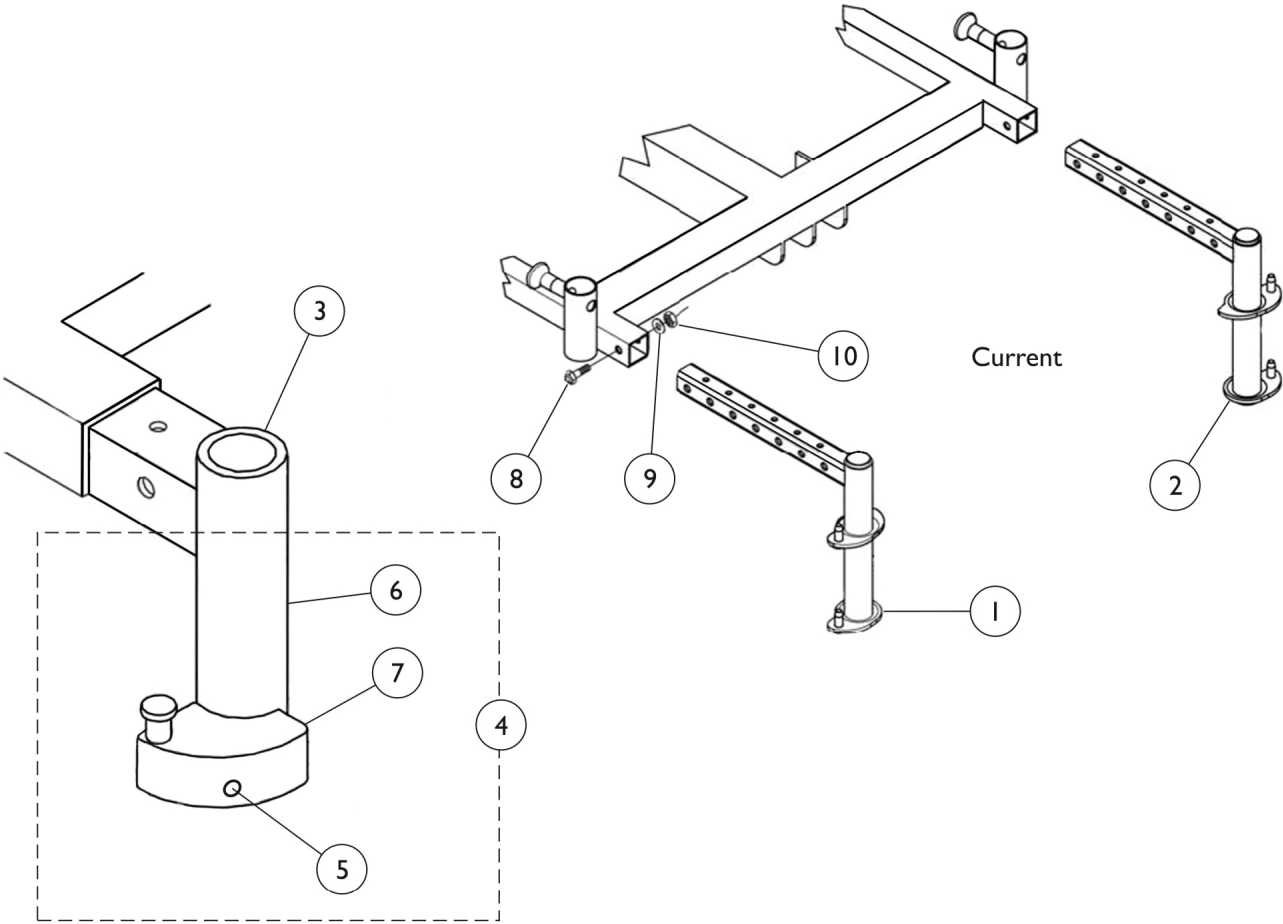

Sector Block Legrest Hanger - Action
(Right Side Shown)

Legrest Hangers

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1095751	Legrest Hanger, Pin Style w/Cap - Right	--	1
2		1095752	Legrest Hanger, Pin Style w/Cap - Left	--	1
3	A	0116802	Legrest Hanger, Sector Style - Right	--	1
N/S	A	0116902	Legrest Hanger, Sector Style - Left	--	1
4	B	1054571	Kit, Sector Block - Right	--	1
N/S	B	1054572	Kit, Sector Block - Left	--	1
N/S	C	1054866	Kit, Sector Block Hardware (Pair)	1	1
5			Screw, Socket Head w/ Patch (1/4-20 x 2")	1	2
6			NLA - Washer, Steel (1/4 x 1/2 x 3/64")	1	2
7			NLA - Block, Sector - Right	1	1
N/S			NLA - Block, Sector - Left	1	1
8		1030949	Screw, Hex Head (1/4-20 x 1-1/2")	--	2
9		319476	Washer	--	2
10		1025736	Locknut (1/4-20)	--	2
NOTE: A - Sector Block Kit is not included with the Legrest Hanger. B - Kit includes items 5-7. C - Kit includes items 5-7 to accommodate the right and left side of the wheelchair. The Sector Block is No Longer Available to be sold separately.					
NOTE: Pin Style Hangers should not be available with chair depths of 16" or 17" when choosing 14.5, 15.25 height.					