


Front Riggings - Legrest Support Assembly - AL4

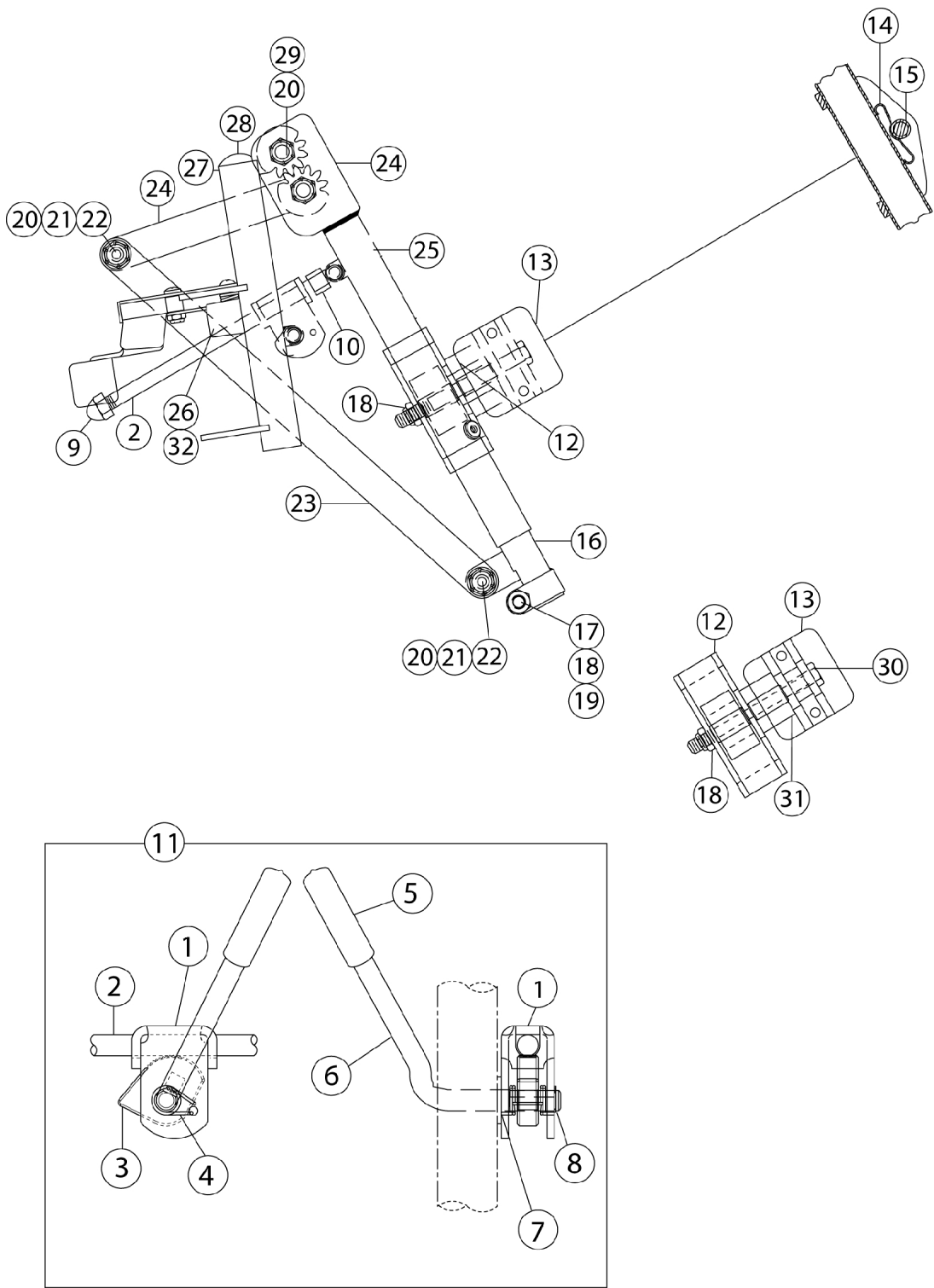

Front Riggings - Legrest Support Assembly - AL4

1041797 Right or 1041798 Left

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		04528X002	Bracket, Release, Black	1	1
2		42326X004	Bar, Ratchet - Right/Left	1	1
3		1037125	Cam, Legrest	1	1
4		1037485	Spring, Release	1	1
5		1040060	Tip, Plastic, Black	1	1
6		1037126	Handle, Release	1	1
7		1013519	Washer, Nylon (5/16 x 11/16 x 1/16")	1	1
8		1040831	Ring, Snap	1	1
9		10261X000	Nut, Acorn (5/16-24)	1	1
10		47330X003	Spacer (5/16 x 1/2 x 3/8")	1	2
11	A	1046329	Kit, Elevating Legrest Ratchet Hardware (Pair)	1	1
12		1025104	Bracket, Legrest, Chrome	1	1
13		04526X019	Plate, Legrest Pad	1	1
14		48823X008	Spring, Leaf	1	1
15		47328X003	Bushing, Pivot (5/16 x 7/16 x 7/8")	1	1
16		1022848	Tube, Slide, Articulating, Chrome - Right	1	1
17		1027377	Washer (#20)	1	1
18		60264X001	Locknut (5/16-24)	1	3
19		11232X009	Bolt, Lug (5/16-24 x 1-3/8")	1	1
20		40363X000	Hex Nut (3/8"-16)	1	3
21		1022866-NLA	NLA - Screw, Socket Button Head (3/8-16 x 3/4")	1	2
22		1049527	Washer, Nylon (3/8 x 3/4 x 1/16")	1	2
23		1022846-NLA	NLA - Bar, Link, Black	1	1
24		1022877-NLA	NLA - Tube, Support and Drive Link, Chrome - Right	1	1
24		1022878-NLA	NLA - Tube, Support and Drive Link, Chrome - Left	1	1
25		1024911-NLA	NLA - Decal, Smartleg	1	1
26		1022874	Bumper	1	1
27		1022871-NLA	NLA - Legrest Support Tube Assembly, Adult, Chrome - Right	1	1
27		1022872-NLA	NLA - Legrest Support Tube Assembly, Adult, Chrome - Left	1	1
28		1027097	Plug Button, Black (3/4")	1	1
29		1022854-NLA	NLA - Screw, Shoulder Slotted Head (1/2 x 1-1/8", 3/8-16)	1	1
30		1014807	Screw, Hex Head (5/16-24 x 3-1/4")	1	1
31		1025133	Spacer, Legrest Pad, Black	1	1
32		1030979	Screw, Phillips Pan Head Tap (#10-32 x 3/4")	1	1

NOTE: A - Includes items 1-10

NOTE: See pages 33-35, 40-44, 47, and 50 for Legrest Footplate Assemblies.

Front Riggings - Legrest Support Assembly - AL4

AL4 Calf Pad

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
		04529X003	Pad, Calf with Cover, Black - Right (14 - 15" W)	1	2
		04529X004	Pad, Calf with Cover, Black - Left (14 - 15" W)	1	2
		7500003	Pad, Calf with Cover	1	2
		1091764	Cover, Calf Pad, Black Vinyl - Right/ Left	1	1
		1091765	Calf Pad Assembly, Single with LBL	1	1
		1066591	Hardware, Legrest Pad	1	2
			Screw, Phillips Pan Head (#10-32 x 11/16")	1	2
			Screw, Phillips Pan Head (#10-32 x 1")	1	2
			Instruction Sheet, Legrest Pad Mtg.	1	1