
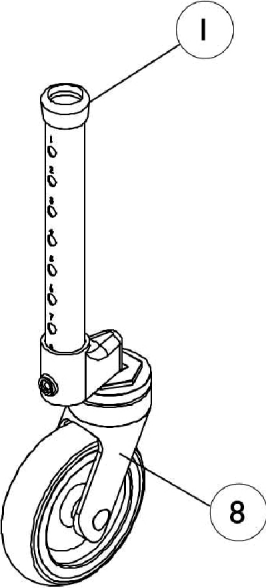


Swivel Wheel Attachments With 13" Sure Glide Brakes

Model 6272 - 3" Wheel
Model 6268 - 5" Wheel

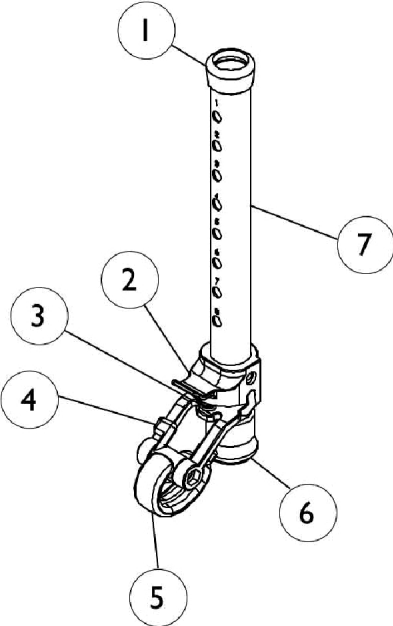

*Attachments Sold in Pairs
Single Side Shown*

Swivel Wheel Attachments With 13" Sure Glide Brakes

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1040655	Anti-Rattle, Grey (1-1/8")	1	4
2		1044942	NLA - Housing, Spring	1	2
3	A	1021734-NLA	NLA - Spring	1	2
3	B	1044944	Spring, Compression (31/64 x 1 x 3/64")	1	2
4		1044943	Fork	1	2
5		1053179	Glide	1	2
6	C	1040656	Tip, Rubber, Dark Grey (1-1/8")	1	2
7	B	1044929-NLA	NLA - Tube, Leg Extension (13")	1	2
8	E,G	1039469-NLA	NLA - Caster Assembly, Swivel (3")	1	2
8	H	1123541-NLA	NLA - Caster Assembly, Swivel (3")	1	2
8		1044509-NLA	NLA - Caster, Swivel (5")	1	2
8	I	1123542	Caster Assembly, Swivel (5")	1	2
	A,C,D	1040848-NLA	NLA - Sure Stop Sub-Assembly (13")	1	2
	A,C,D	1042876-NLA	NLA - Tube, Inside Glide	1	2
	A,C,D	1041532-NLA	NLA - Tip, Glide, Grey (3/4")	1	2
	A,C,D	1040840-NLA	NLA - Plug Button, Grey (7/8")	1	2
	C,D,F	1039493-NLA	NLA - Adapter, Bushing - Right	1	1
	C,D,F	1039494-NLA	NLA - Adapter, Bushing - Left	1	1
	C,D,F	21243X000-NLA	NLA - Screw, Phillips Pan Head (#6-20 x 3/8")	1	2
	C,D,F	1040984-NLA	NLA - Label - Right	1	1
	C,D,F	1040985-NLA	NLA - Label - Left	1	1

NOTE: A - Before 7/95.
B - After 6/95.
C - For 1-1/8" Tubing.
D - Not Shown.
E - Due to style change, when replacing 3" Wheels "Before 11/5/00" order in pairs.
F - Before 8/99.
G - Item no longer available, refer to Part #1123541.
H - Replaces Part #1039469.
I - Replaces Part #1044509.