
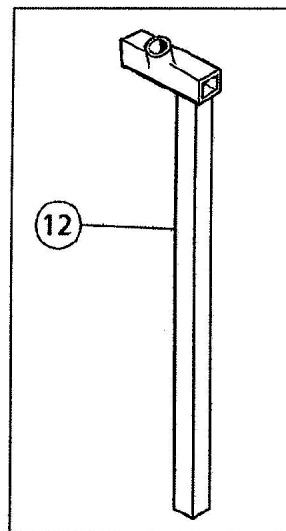
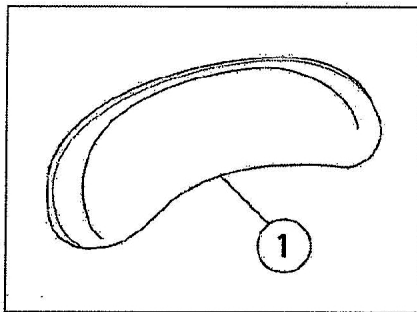


Headrest Assembly

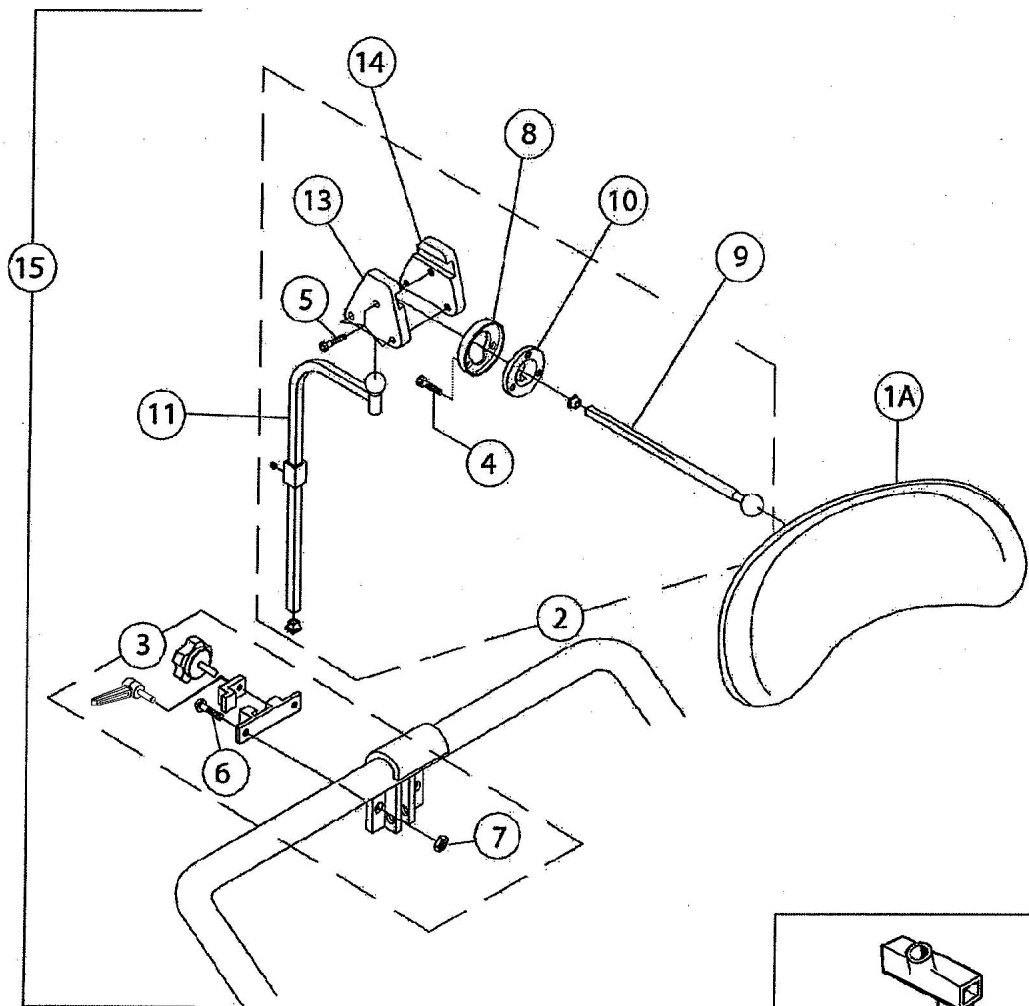

Headrest Assembly

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		5500060	Standard Headrest, Pad Only	1	1
1A		5500070	Comfort Headrest, Pad Only	1	1
2		0197024	Fixture Assembly, Offset Support	1	1
3	D,E	1160332	Kit, Headrest Clamp, Knob Style	1	1
3	D,F	0273100	Kit, Headrest Clamp, Lever Style	1	1
4		1130086	Screw, Socket Head Cap (#10-32 x 1")	1	1
5		367-059	Screw, Hex Socket Head (M6 x 25mm)	1	1
6		1055305-NLA	NLA - Screw, Socket Button Head (1/4-28 x 3/4")	1	2
7		1025765	Locknut (1/4-28)	1	2
8		1130080	Spacer, Headrest	1	1
9		1130084	Tube, Square Slide, Black - Short	1	1
10		1130085	Washer, Slotted	1	1
11		1130087	Square Wldmt w/ Bend, Black	1	1
12		450108	Bracket, Headrest Post "L"	1	1
13		1055023-NLA	NLA - Clamp, Horizontal Pivot	1	1
14		1055024-NLA	NLA - Clamp, Threaded, Horizontal Pivot	1	1
15	A	5500071	Comfort Headrest Assembly w/ Multi Axis Hardware (CHC45)	1	1
16	B	5500061	Kit, Headrest, Standard (SPH)	1	1
17	C	1130083	Headrest, Comfort w/ Standard Mounting Hardware (CHSTD)	1	1
NOTE: A - Includes items 1A, 2, 3, 8, and 9. For Model CHC45. B - Includes items 1, 3, and 12. Not Shown. For Model SPH. C - Includes items 1A, 3, 4, 8, 9, 10, and 12. For CHSTD. D - Lever in Kit 0273100 is interchangeable with the Knob in Kit 1160332. E - Headrest Clamp Kit with Knob used Before 10/01/09 F - Headrest Clamp Kit with Lever used After 09/30/09					