


Front Stabilizer Assembly

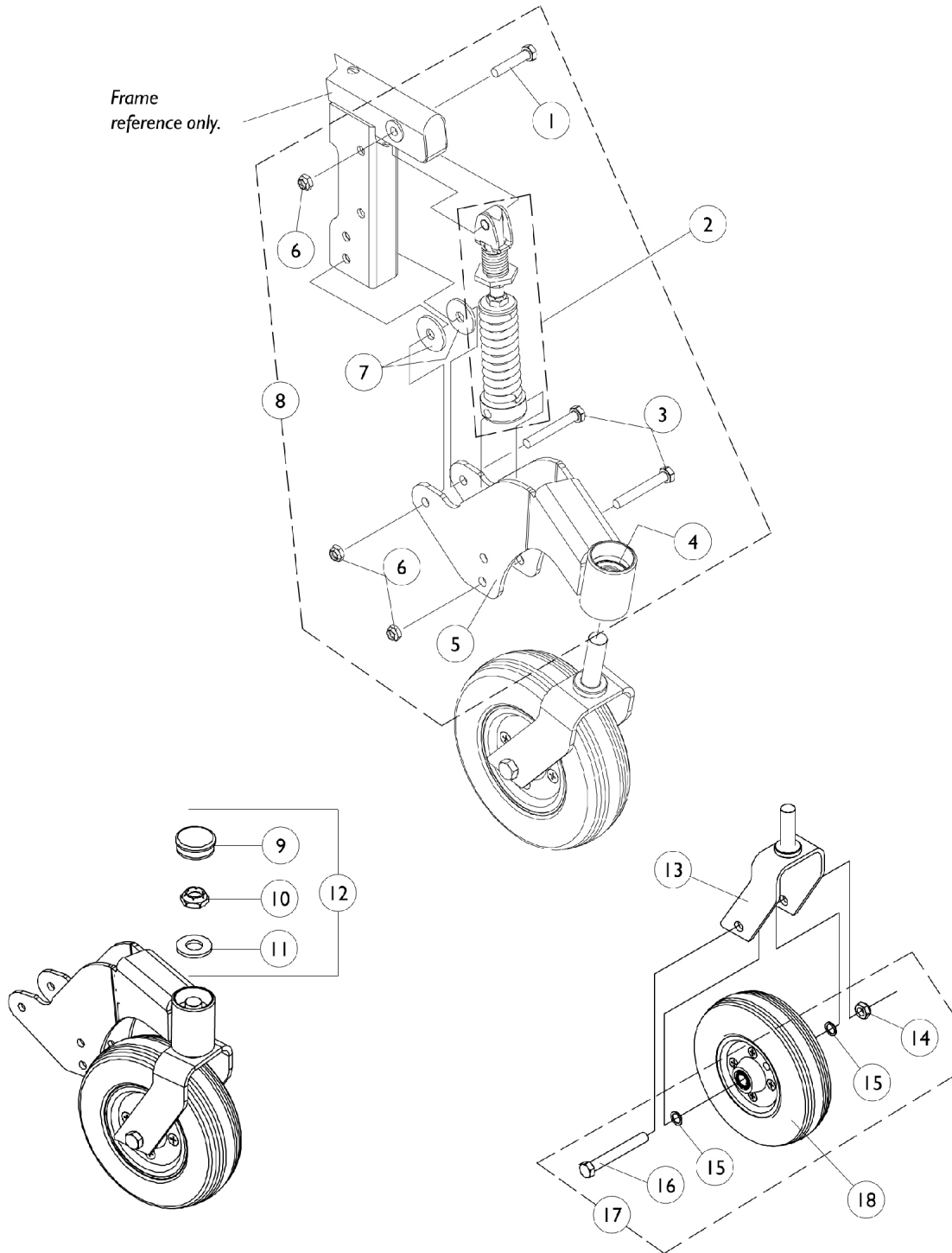

Front Stabilizer Assembly

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1028514	Screw, Hex Head (5/16-24 x 1-1/2")	1	1
2		1094630-NLA	NLA - Spring, Dual Compression Column Assembly	1	1
3		1025764	Screw, Hex Head (5/16-24 x 2-1/4")	1	2
4		40117X006	Bearing (5/8")	1	2
5		1094616-NLA	NLA - Swing Arm w/ Bearings Assembly, Black - Right	1	1
5		1094617-NLA	NLA - Swing Arm w/ Bearings Assembly, Black - Left	1	1
6		7930330-NLA	NLA - Locknut (5/16-24)	25	3
7		1071487-NLA	NLA - Washer, Nylon (23/64 x 1-15/64 x 7/64")	1	2
8	A,B	1094621-NLA	NLA - Stabilizer Assembly - Right	1	1
8	A,B	1094622-NLA	NLA - Stabilizer Assembly - Left	1	1
8	A,C	1094624-NLA	NLA - Stabilizer Assembly, One Piece Footboard - Right	1	1
8	A,C	1094625-NLA	NLA - Stabilizer Assembly, One Piece Footboard - Left	1	1
9		1056739	Plug Button, Head Tube, Black	1	1
10		1028550	Locknut (5/8-18)	1	1
11		1041444	Washer, Nylon (5/8 x 1-5/16 x 1/8")	1	1
12	D	1057949	Hardware, Fork Stem	1	1
13		1094607-NLA	NLA - Fork Assembly (7")	1	1
14		7930331-NLA	NLA - Locknut (7/16-20")	1	1
15		7930329	Washer, Steel (15/32 x 5/8 x 3/64")	25	1
16		1025726	Screw, Hex Head (7/16-20 x 3-1/4")	25	2
17	E	1042489	Hardware, Caster Axle	1	2
18	F	1094620-NLA	NLA - Wheel Assembly (7" x 2")	1	1
18		1094618	Tire, Urethane, Grey (7" x 2")	1	1
18		1062845-NLA	NLA - Kit, Rim w/ Hardware (Pair)	1	1
18		1020592	Spacer, Bearing (7/16 x 5/8 x 1-3/4")	1	1
18		1006049	Bearing (7/16")	1	2

NOTE: A - Item number 2 not included
 B - Includes items 1 and 3-7
 C - Includes items 3-7
 D - Includes items 9-11
 E - Includes items 14-16 to accommodate 1 side only.
 F - Part numbers 1094618, 1062845, 1020592, and 1006049 are available for individual sale.