



Wheels, Drive and Hub Mount

NOTE: The new chrome painted 14" x 3" spoke wheels are retrofitable to chairs made before 10/17/01. Wheel hub assembly part number (1103010) must be ordered as well.

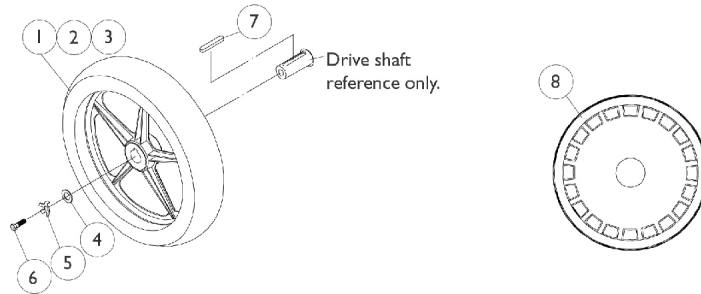


Drive Wheel Assembly (12-1/2" x 2-1/4")

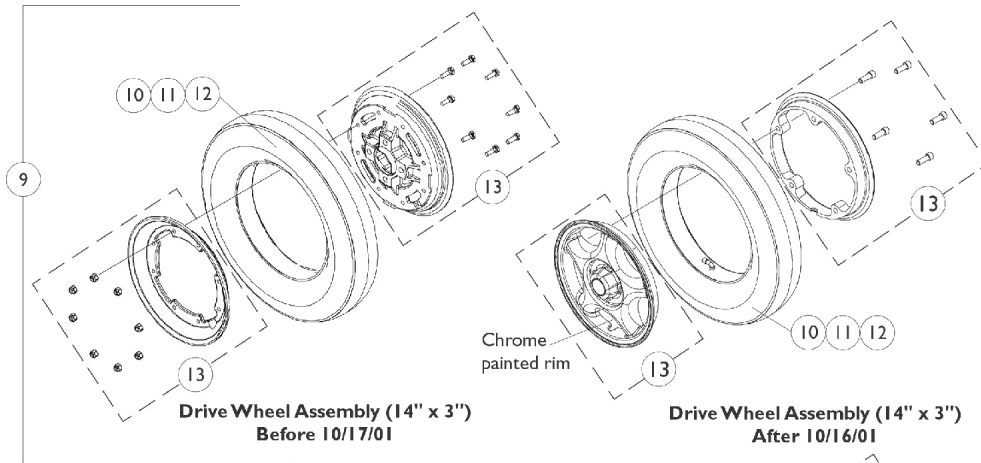

**Drive Wheel Assembly (14" x 3")
Before 10/17/01**

**Drive Wheel Assembly (14" x 3")
After 10/16/01**

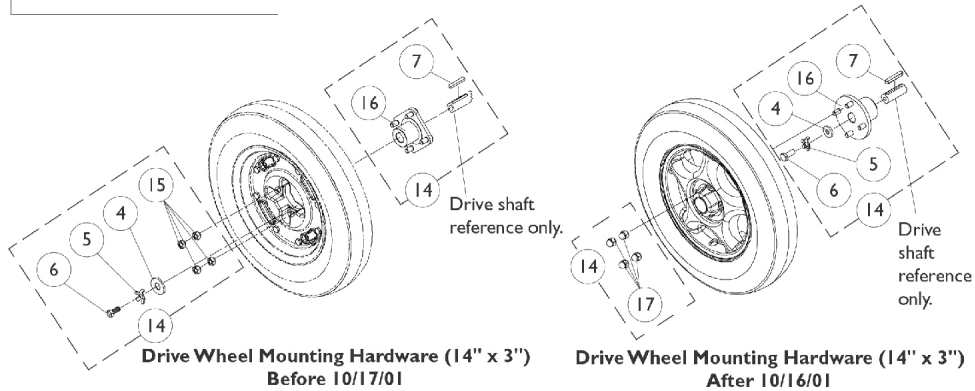

**Drive Wheel Mounting Hardware (14" x 3")
Before 10/17/01**

**Drive Wheel Mounting Hardware (14" x 3")
After 10/16/01**

Wheels, Drive and Hub Mount

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1064177	Drive Wheel Assembly, 5 Spoke w/ Foam Fill Insert (12-1/2 x 2-1/4") Includes: Locking Tab Washer with Instruction Sheet	1	1
1		1064178-NLA	NLA - Drive Wheel Assembly, 5 Spoke, Pneumatic (12-1/2 x 2-1/4") Includes: Locking Tab Washer with Instruction Sheet	1	1
2		1011524	Tire, Pneumatic, Grey (12-1/2" x 2-1/4")	1	1
3		1011525	Tube, Inner, Black (12-1/2 x 2-1/4")	1	1
4		1079247	Washer, Flat (21/64 x 53/64 x 7/64")	1	1
5		1064179	Kit, Locking Tab Washer w/ Instruction Sheet (Pair)	2	1
6		7930322-NLA	NLA - Package, Screw, Hex Head (M8 x 1.25 x 20mm)	25	1
7		1023264	Key (6mm x 6mm x 40mm)	1	1
8	I	1074974-NLA	NLA - Cover, Shroud (Black), Drive Wheel	1	1
9	A	1090649-NLA	NLA - Pneumatic Wheel Drive Assembly (14" x 3")	1	1
9	B	1104438	Drive Wheel Assembly, Pneumatic, Grey Tire/Silver Rim (14 x 3")	1	1
9	A	1090650-NLA	NLA - Drive Wheel Assembly w/ Flat Free Insert, Grey Tire/Black Rim (14 x 3")	1	1
9	B	1104437	Drive Wheel Assembly, Flat Free, Grey Tire/Silver Rim (14 x 3")	1	1
10		1101754	Tire w/ Foam Fill Insert, Grey (14" x 3")	1	1
11		1039652	Tire, Pneumatic, Grey (14" x 3")	1	1
12		1039651	Tube, Inner, Black (14 x 3")	1	1
13	A,G	1090651-NLA	NLA - Kit, 8" Rim w/ Hardware (14" x 3" Wheel)	1	1
13		7930326-NLA	NLA - Package, Screw, Hex Head (1/4-20 x 3/4")	25	8
13		7930333	Locknut (1/4-20)	25	8
13	B,H	1104882-NLA	NLA - Kit, 8" Rim w/ Hardware (14 x 3" Wheel)	1	1
13		1101094	Screw, Socket Head w/ Patch (5/16-18 x 7/8")	1	5
14	A,C	1090648-NLA	NLA - Hardware, Drive Wheel Mounting w/ Instruction Sheet (14 x 3")	1	1
14	B,D	1103011-NLA	NLA - Hardware, Drive Wheel Mounting w/ Instruction Sheet (14" x 3")	1	1
15		1024444	Locknut (5/16-24)	1	4
16	A	1086681-NLA	NLA - Hub, Wheel (14" x 3")	1	1
16	B	1100576-NLA	NLA - Hub, Wheel (14" x 3")	1	1
17		1100580	Nut, Beveled Hex (5/16-24)	1	4
	B,E,F	1103010-NLA	NLA - Wheel Hub Assembly	1	1

NOTE: A - Before 10/17/01
 B - After 10/16/01
 C - Includes items 4-7, 15 and 16
 D - Includes items 4-7, 16 and 17
 E - Includes items 16 and 17
 F - Not Shown
 G - Part numbers 7930326 and 7930333 are available for individual sale.
 H - Part number 1101094 is available for individual sale.
 I - Shroud cover not available on drive wheels offered after 2/13/03