


# Steering

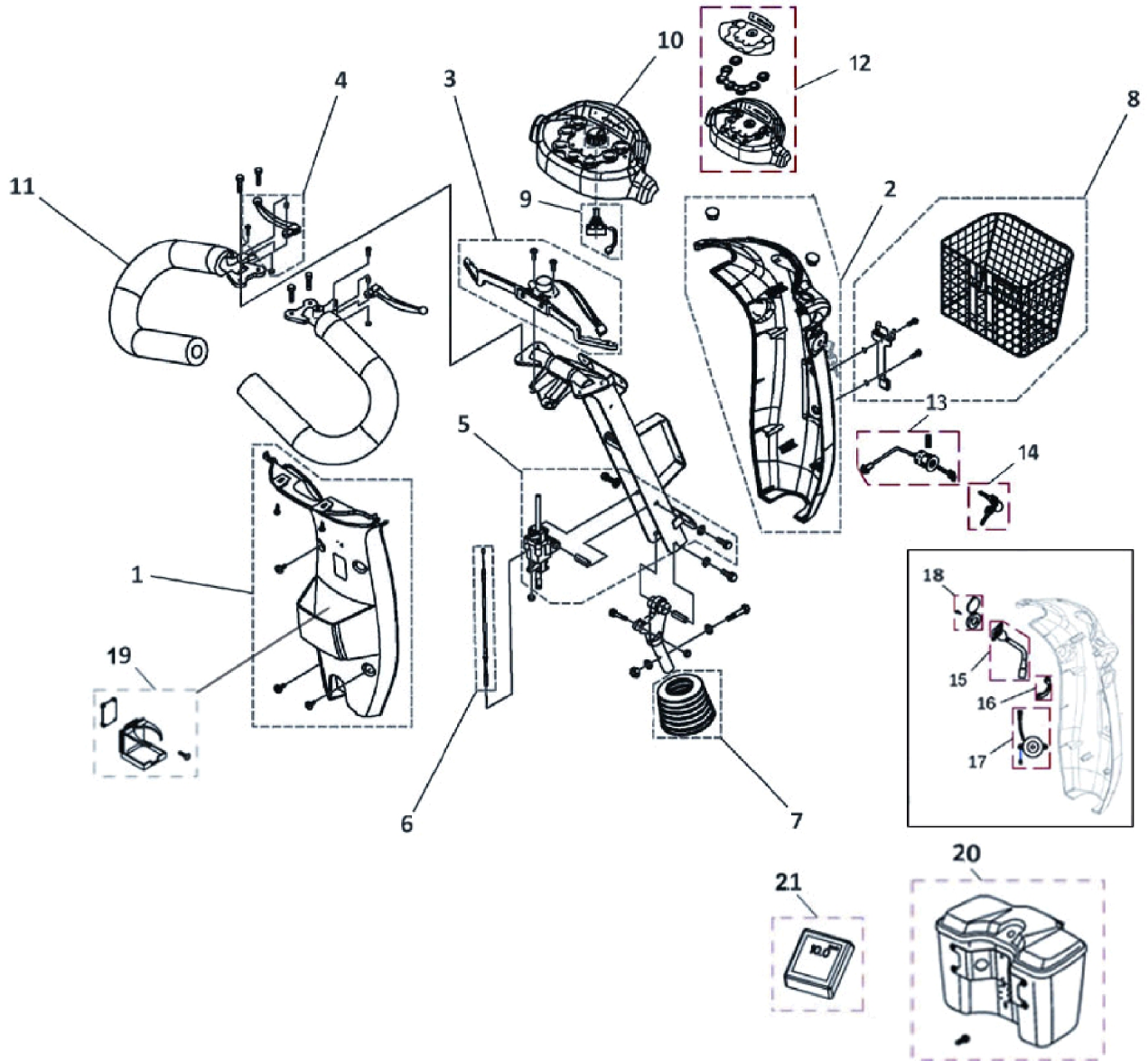

# Steering

| Item Number | Note | Part Number | Description                                    | Quantity Per |          |
|-------------|------|-------------|--|--------------|----------|
|             |      |             |  | Package      | Assembly |
| 1           |      | 1521725     | Cover, Rear Steering                           | --           | 1        |
| 1           |      | 1537796     | Cover, Front Steering, Slate Grey, Comet HD    | --           | 2        |
| 1           |      | 1530006     | Cover, Steering, Front, Complete, Red          | --           | 1        |
| 2           |      | 1521726     | Cover, Front Steering, Silver                  | --           | 1        |
| 2           |      | 1521727     | Cover, Front Steering, Blue                    | --           | 1        |
| 3           |      | 1521575     | Starter  | --           | 1        |
| 4           | A    | 1521578     | Regulator Rod                                  | --           | 2        |
| 5           |      | 1505279     | Adjustable Steering Assembly                   | --           | 1        |
| 6           |      | 1521583     | Cable, Starter to Adjustable Steering Assembly | --           | 1        |
| 7           |      | 1521586     | Cover, Steering Joint                          | --           | 1        |
| 8           |      | 1521584     | Basket, Front                                  | --           | 1        |
| 9           |      | 1505460     | Knob, Speed Control                            | --           | 1        |
| 10          |      | 1505462     | Variable Resistor, Complete (30.0KO)           | --           | 1        |
| 11          |      | 1527790     | Foam Grip                                      | --           | 2        |
| 12          |      | 1532908     | Cover, Upper Dash                              | --           | 1        |
| 13          |      | 1505451     | Ignition                                       | --           | 1        |
| 14          |      | 1164848     | Package, Ignition Key                          | 10           | 1        |
| 15          |      | 1523543     | Charger Port                                   | --           | 1        |
| 16          |      | 1523544     | Retaining Plate, Charger Port                  | --           | 1        |
| 17          |      | 1506153     | Buzzer   | --           | 1        |
| 18          |      | 1523534     | Cover, Charger Port                            | --           | 1        |
| 19          | B    | 1521594     | Assembly, Drink Holder                         | --           | 1        |
| 20          | B    | 1537710     | Front Lock Box, Comet HD, Slate Grey           | --           | 1        |
| 20          | B    | ACC270      | Front Lock Box, Comet, Blue                    | --           | 1        |
| 20          | B    | ACC260      | Front Lock Box, Comet, Silver                  | --           | 1        |
| 20          | B    | 1551638     | Front Lock Box, Comet, Red                     | --           | 1        |

NOTE: A - One each for left or right side.  
 B - Additional Accessory, not included with the Scooter.