



Crossbraces and Hardware

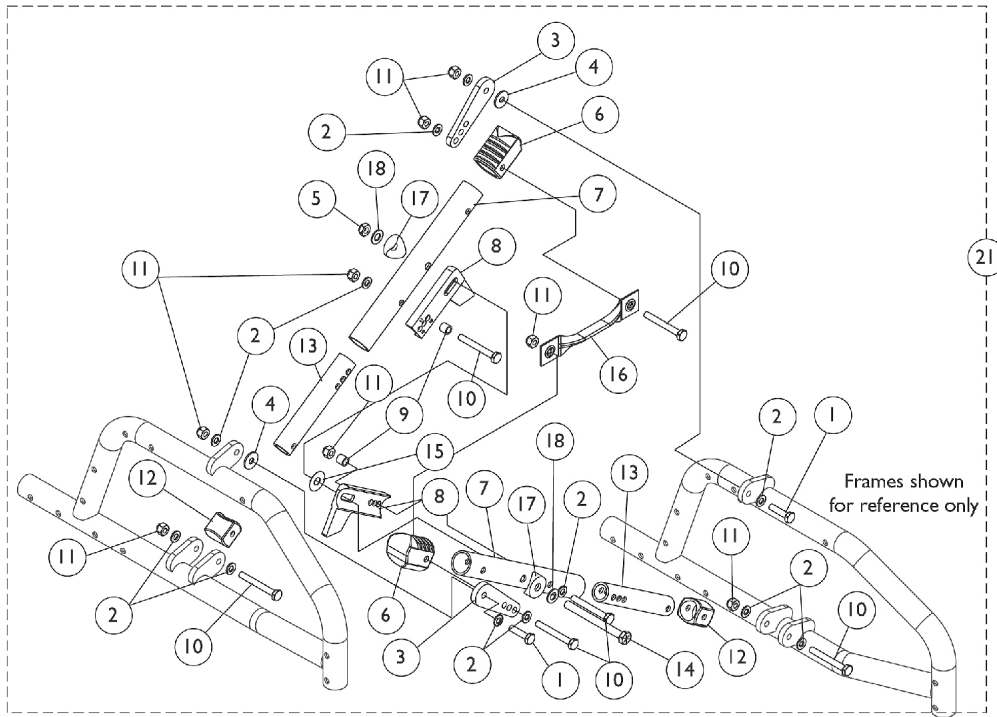

10-14" Wide Chairs

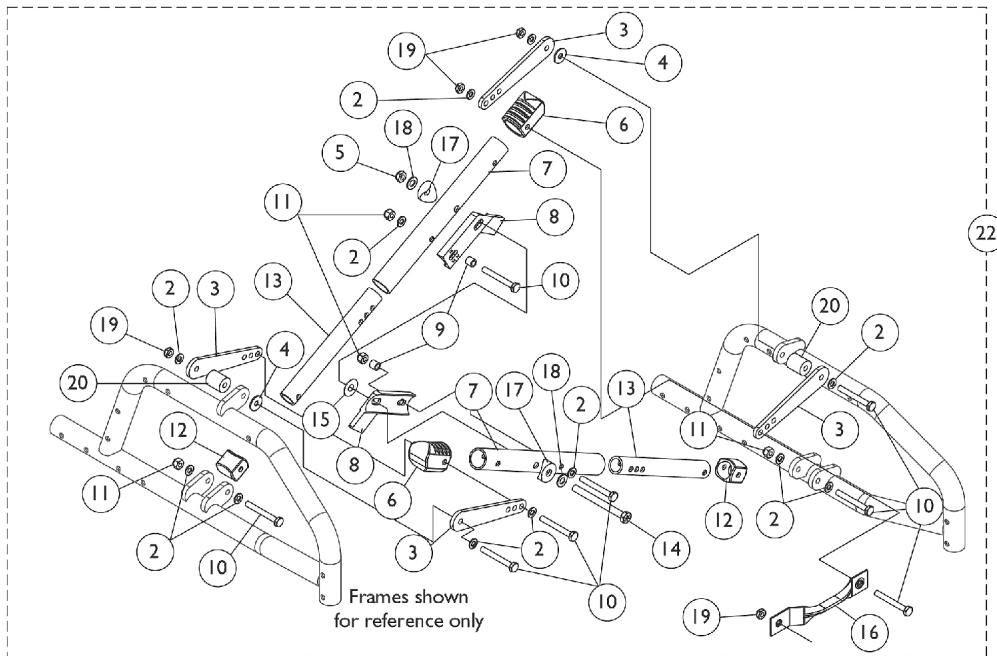

14-18" Wide Chairs

Crossbraces and Hardware

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1055296-NLA	NLA - Screw, Hex Head (1/4-28 x 1")	1	2
2		1026907	Washer (1/4 x 1/2 x 1/16")	1	12
3		1118521	Link, Pivot, Black (10 - 12" W)	1	2
3		1118522	Link, Pivot, Black (12 - 14" W)	1	2
3		1118523	Link, Pivot, Black (14 - 16" W)	1	2
3		1118524	Link, Pivot, Black (16 - 18" W)	1	2
4		1108477-NLA	NLA - Washer, Nylon (1/4 x 3/4 x 5/8")	1	2
5		1025730	Locknut (5/16-24)	1	1
6		1118501-NLA	NLA - Plug Button, Crossbrace - Upper	1	2
7		1118506	Tube, Crossbrace -Upper (10-12" W)	1	2
7		1118507	Tube, Crossbrace -Upper (12-14" W)	1	2
7		1118508	Tube, Crossbrace -Upper (14-16" W)	1	2
7		1118509	Tube, Crossbrace -Upper (16-18" W)	1	2
8		1118526	Bracket, Stop, Black (10-12" W)	1	2
8		1118527	Bracket, Stop, Black (12-14" W)	1	2
8		1118528	Bracket, Stop, Black (14-16" W)	1	2
8		1118529	Bracket, Stop, Black (16-18" W)	1	2
9		1157260	Spacer, Crossbrace (1/4 x 7/16 x 5/16)	1	2
10	A	1055349	Screw, Hex Head (1/4-28 x 2")	1	6
11	B	1065362	Locknut (1/4-28)	1	8
12	G	1118502-NLA	NLA - Cap, End, Lower, Black	1	2
13		1118511	Tube, Crossbrace -Lower (10-12" W)	1	2
13		1118512	Tube, Crossbrace -Lower (12-14" W)	1	2
13		1118513	Tube, Crossbrace -Lower (14-16" W)	1	2
13		1118514	Tube, Crossbrace -Lower (16-18" W)	1	2
14		1025705-NLA	NLA - Screw, Hex Head (5/16-24 x 3-1/4")	1	1
15		1046497-NLA	NLA - Washer, Nylon (5/16 x 7/8 x 1/64")	1	1
16	C,G	1107937-NLA	NLA - Strap, Pull	1	1
16	D	1118532	Strap, Pull	1	1
17		1016263	Spacer, Coved (3/4 x 1 x 1/2")	1	2
18		1025704	Washer, Flat (5/16 x 5/8 x 1/16")	1	2
19		1025765	Locknut (1/4-28)	1	4
20		1118531	Bushing, Nylon (3/4 x 1/4 x 7/8")	1	2
21	E	1118535	Crossbrace Assembly, Black (10 - 12" W) (Pair)	2	1
21	E	1118536	Crossbrace Assembly, Black (12 - 14" W) (Pair)	2	1
22	F	1118537	Crossbrace Assembly, Black (14 - 16" W) (Pair)	2	1
22	F	1118538	Crossbrace Assembly, Black (16 - 18" W) (Pair)	2	1

NOTE: A - Chairs 14" wide and greater require 8 ea. of item 10 Screw
 B - Chairs 14" wide and greater require 4 ea. of item 11 Locknut
 C - 10-13" Wide Chairs
 D - 14-18" Wide Chairs
 E - Includes items 1-18
 F - Includes items 2-20
 G - No longer available as an individual service part, use Item 21 (1118535 or 1118536) or Item 22 (1118537 or 1118538) as a replacement.