



Footplate and Pivot Tube Assemblies (1651) Composite

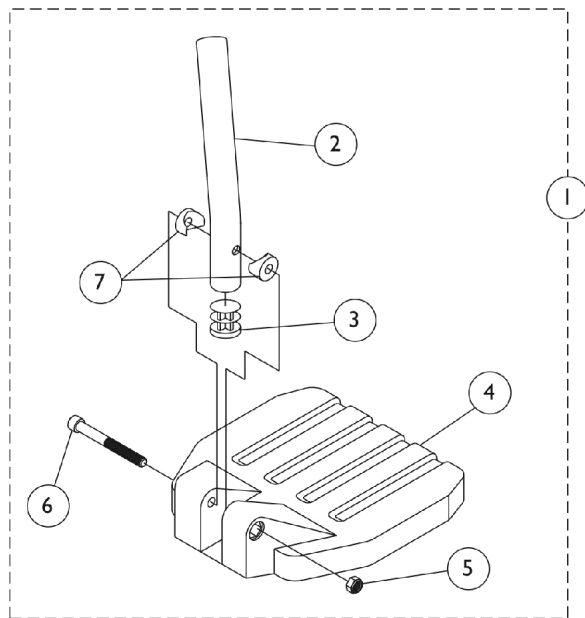

**Used on all front riggings except
Wedge Style Elevating Legrests AT5044 and AT5044LIFT**

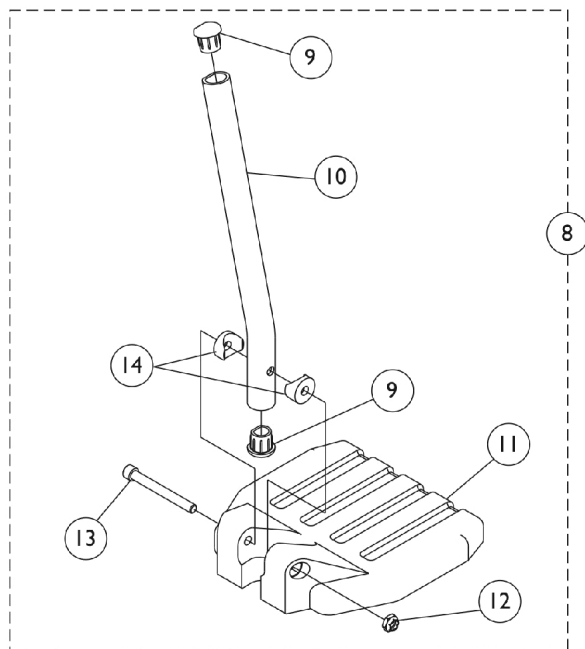

Used on Wedge Style Elevating Legrests AT5044 and AT5044LIFT

Footplate and Pivot Tube Assemblies (1651) Composite

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	F	See Chart	#1 60 Degree Footrest (13" - 19")	1	1
1	F	See Chart	#1 60 Degree MFX Footrest (9-3/4" - 14")	1	1
1	F	See Chart	#1 70 Degree Non Tapered Footrest (14" - 19")	1	1
1	F	See Chart	#1 70 Degree Tapered Footrest (14-1/2" - 19-1/2")	1	1
2	A	1120382	Tube, Pivot/Slide, Curved End, Black - Long (9-7/8")	1	1
2	B	1120389	Tube, Pivot/Slide, Straight, Black - Long (9-7/8")	1	1
2	C	1120390	Tube, Pivot/Slide, Straight, 2" Longer ALPT, Black (11-7/8")	1	1
2	D	1120391	Tube, Pivot/Slide, Curved End, 2" Longer ALPT, Black (11-7/8")	1	1
3		1134426	Package, Plug Button, Black (3/4")	10	1
4		1026879	Footplate, Composite, Small, Black (6" W x 6" L)	1	1
4		1026880	Footplate, Composite, Medium, Black (7" W x 6" L)	1	1
4		1026881	Footplate, Composite, Large, Black (8" W x 6" L)	1	1
5		1025195	Locknut (1/4-20)	1	1
6		60285X006	Screw, Socket Head (1/4-20 x 2-1/4")	1	1
7		1025457	Spacer, Coved (1/4 x 3/4 x 11/32")	1	2
8	G	See Chart	#8 AT5044 Elevating Legrest - Wedge Style (16" - 20")	1	1
9		1134427	Package, Plug Button, Black, "D" Shaped (3/4")	10	1
10	E	1134428	Pivot/Slide Tube w/ D Insert, Black	1	1
10	E	1134429	Pivot/Slide Tube w/ D Insert, Black (2" Longer)	1	1
11		1026879	Footplate, Composite, Small, Black (6" W x 6" L)	1	1
11		1026880	Footplate, Composite, Medium, Black (7" W x 6" L)	1	1
11		1026881	Footplate, Composite, Large, Black (8" W x 6" L)	1	1
12		1025195	Locknut (1/4-20)	1	1
13		60285X006	Screw, Socket Head (1/4-20 x 2-1/4")	1	1
14		1025457	Spacer, Coved (1/4 x 3/4 x 11/32")	1	2
NOTE: A - 60 Degree Footrests B - 70 Degree Footrests C - 70 Degree Footrests D - 60 Degree Footrests E - AT5044 Wedge Style Legrests F - Includes items 2-7 G - Includes items 9-14					

#1 60 Degree Footrest 13" - 19" Curved End

Right/Left	Small Long 13" - 14" Wide	Small 2" Longer 13" - 14" Wide	Medium Long 15" - 16" Wide	Medium 2" Longer 15" - 16" Wide	Large Long 17" - 24" Wide	Large 2" Longer 17" - 24" Wide
Right	1028391	1120416	1028387	1120418	1028383	1120420
Left	1028392	1120417	1028388	1120419	1028384	1120421

Footplate and Pivot Tube Assemblies (1651) Composite

#1 60 Degree MFX Footrest 9-3/4" - 14" Curved End

Right/Left	Small Short 13" - 14" Wide	Medium Short 15" - 16" Wide	Large Short 17" - 24" Wide
Right	1028391	1028387	1028383
Left	1028392	1028388	1028384

#1 70 and 80 Degree Non Tapered (70LIFTNT/80NTAPER) Footrest 14" - 19" Straight

Right/Left	Small Short 13" - 14" Wide	Small 2" Longer 13" - 14" Wide	Medium Short 15" - 16" Wide	Medium 2" Longer 15" - 16" Wide	Large Short 17" - 24" Wide	Large 2" Longer 17" - 24" Wide
Right	1120404	1120410	1120406	1120412	1120408	1120414
Left	1120405	1120411	1120407	1120413	1120409	1120415

Footplate and Pivot Tube Assemblies (1651) Composite

#1 70 Degree Tapered (70STAPER) - 14-1/2" - 19-1/2" Straight

Right/Left	Small Short 16" Wide	Small 2" Longer 16" Wide	Medium Short 17" - 18" Wide	Medium 2" Longer 17" - 18" Wide	Large Short 19" - 24" Wide	Large 2" Longer 19" - 24" Wide
Right	1120404	1120410	1120406	1120412	1120408	1120414
Left	1120405	1120411	1120407	1120413	1120409	1120415

#8 AT5044 Elevating Legrest - Wedge Style 16" - 20" Curved End

Right/Left	Small Long 14" Wide	Small 2" Longer 14" Wide	Medium Long 15" - 16" Wide	Medium 2" Longer 15" - 16" Wide	Large Long 17" - 24" Wide	Large 2" Longer 17" - 24" Wide
Right	1132078	1132107	1132080	1132109	1132082	1132111
Left	1132079	1132108	1132081	1132110	1132083	1132112