



Footplate and Pivot Tube Assemblies (A1562) Aluminum

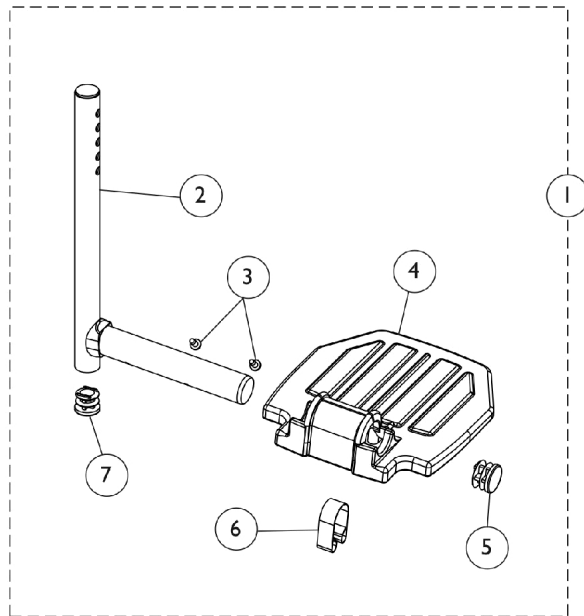

Used on all front riggings except
Wedge Style Elevating Legrests AT5044 and AT5044LIFT

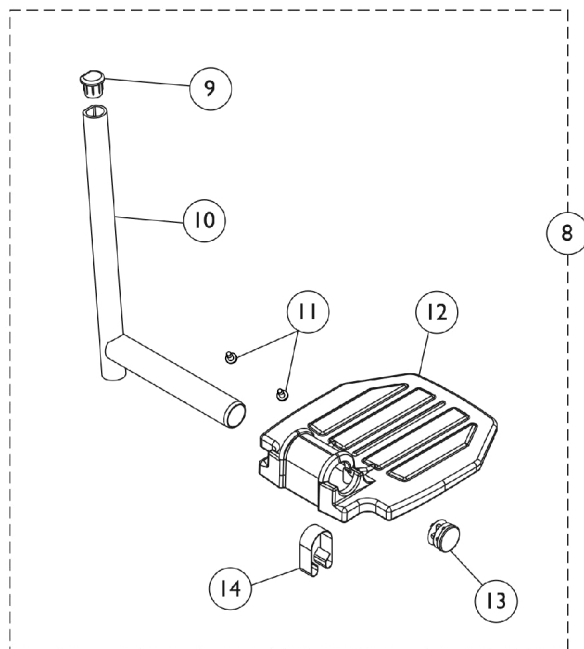

Used on Wedge Style Elevating Legrests AT5044 and AT5044LIFT

Footplate and Pivot Tube Assemblies (A1562) Aluminum

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	See Chart	#1 Aluminum Footplate w/ Pivot/Slide Tube Assembly	1	1
2		1120330	Pivot/Slide Tube w/ Plug, Black	1	1
3		1025729	Screw, Phillips Pan Head Tap (#8-32 x 3/8")	1	2
4		44200X032-NLA	NLA - Footplate, Aluminum, Extra Small, Black (5-3/4" W x 6" L)	1	1
4		44200X059	Footplate, Aluminum, Small, Black (6-3/4" W x 6" L)	1	1
4		44200X027	Footplate, Aluminum, Medium, Black (7-3/4" W x 6" L)	1	1
5		1130582	Package, Plug Button, Black (7/8")	10	1
6		1006981	Spring, Tension	1	1
7		1134426	Package, Plug Button, Black (3/4")	10	1
	B	1074568	Package, Push Plug, Footplate	2	1
8	C	See Chart	#8 Aluminum Footplate w/ Pivot/Slide Tube Assembly	1	1
9		1134427	Package, Plug Button, Black, "D" Shaped (3/4")	10	1
10		1134431	Pivot/Slide Tube w/ Plug Buttons, Black	1	1
11		1025729	Screw, Phillips Pan Head Tap (#8-32 x 3/8")	1	2
12		44200X059	Footplate, Aluminum, Small, Black (6-3/4" W x 6" L)	1	1
12		44200X027	Footplate, Aluminum, Medium, Black (7-3/4" W x 6" L)	1	1
13		1130582	Package, Plug Button, Black (7/8")	10	1
14		1006981	Spring, Tension	1	1

NOTE: A - Includes items 2-7.
B - Not Shown
C - Includes items 9-14.

#1 Aluminum Footplate w/ Pivot/Slide Tube Assembly

All Front Riggings except Wedge Style AT5044 Elevating Legrests

Description	Footplate Size	Chair Width	Right	Left
All Front Riggings Except Tapered Footrests	Extra Small	12" - 13"	1120333	1120334
All Front Riggings Except Tapered Footrests	Small	14" - 15"	1120335	1120336
All Front Riggings Except Tapered Footrests	Medium	16" - 22"	1120337	1120338
70 Degree Tapered Footrests	Small	16" - 17"	1120335	1120336
70 Degree Tapered Footrests	Medium	18" - 24"	1120337	1120338

Footplate and Pivot Tube Assemblies (A1562) Aluminum

#8 Aluminum Footplate w/ Pivot/Slide Tube Assembly

Wedge Style AT5044 Elevating Legrests

Footplate Size	Chair Width	Right	Left
Small	14" - 15"	1132113	1132114
Medium	16" - 24"	1132073	1132074