
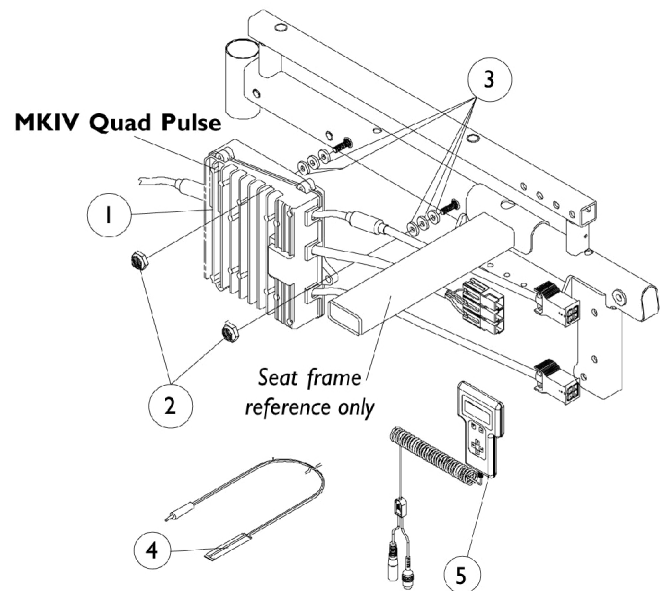


MKIV Controller, Programmer and Hardware

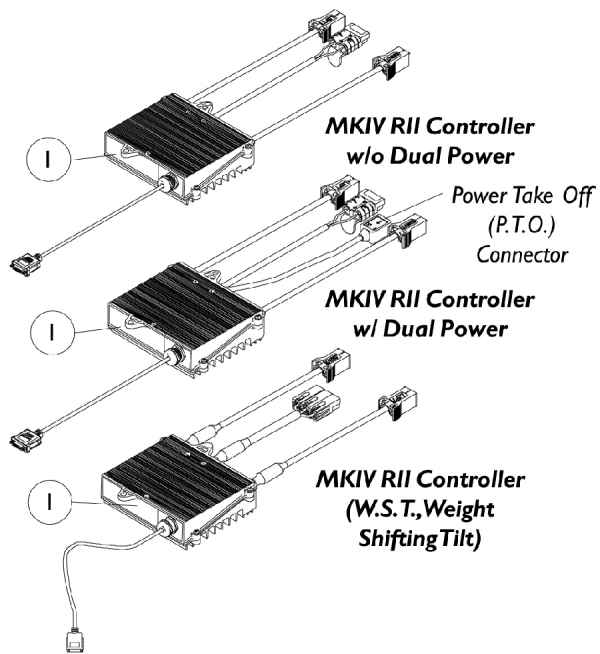

MKIV Controller, Programmer and Hardware

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	C,E	1105707-NLA	NLA - Controller, MKIV Quad Pulse	1	1
1		1092603-NLA	NLA - Controller, MKIV RII	1	1
1	B	1095262-NLA	NLA - Controller, MKIV RII, Dual Power	1	1
1	C,F	1095269-NLA	NLA - Controller, MKIV RII, Retro	1	1
2		7930323-NLA	NLA - Package, Locknut (1/4-20)	25	2
3	G	1018739	Washer (1/4 x 5/8 x 3/32")	1	2
3	H	1018739	Washer (1/4 x 5/8 x 3/32")	1	6
4		1020818	Switch, Red Flex Ribbon (Tape) Reset	1	6
4	L	1123276	Kit, Switch, Black Egg Reset w/ Hook/Loop Adhesive Back Fastener & Attaching Screws (EGSBLK)	1	1
5	A	1109091-NLA	NLA - Programmer, Remote	1	1
	D,I	1064404-NLA	NLA - Virtual Service Modem Kit	1	1
	D,J	1091631-NLA	NLA - Virtual Service Software Kit	1	1
	D,I	1103051-NLA	NLA - MKIV One 2 One Software Kit	1	1
	D,K	1046117-NLA	NLA - Printer and Interface	1	1
	D	1095204-NLA	NLA - Cable, Virtual Service, Charger/Modem	1	1
	D	1091980-NLA	NLA - Disk, 3.5 H.D. Floppy with V.S. Software	1	1
	D	1091637-NLA	NLA - Adaptor, Null Modem	1	1
	D	1091636-NLA	NLA - Service Manual - V.S. Software Kit	1	1
	D	1026572-NLA	NLA - Paper, Printer	1	1
	D	1027280-NLA	NLA - Harness, Printer Wiring	1	1

NOTE: A - Remote programmer part # (1109091) with the dual connectors (3-Pin and 5-Pin is the only programmer functionable with the new RII joystick part # (1092610)
 B - RII dual power controller is used with a manual recliner, high back captain seat and Power Tilt Only.
 C - Wiring harness part number 1095623 must be present when MKIV controllers part number 1105707 or 1095269 are used.
 D - Not Shown
 E - RX, A, A+, TCHJOY, PSF/PSR
 F - Weight Shifting Tilt
 G - Non-Weight Shifting Tilt Seating System
 H - Weight Shifting Tilt Seating System
 I - Part number 1095204 is available for individual sale.
 J - Part numbers 1091980, 1091637, 1091636, and 1095204 are available for individual sale.
 K - Part numbers 1026572 and 1027280 are available for individual sale.
 L - Not Shown. After 5/23/04 the Egg Reset Switch became available as a service part and can be used as a direct replacement for the tape reset switch part # 1020818.