



# Van Seat Assembly


If converting a chair from Office Style Semi-Recline Van Seat to Semi-Recline Van Seat w/ Cushion or Semi-Recline Van Seat w/ Solid Seat Base, see Van Seat Frame and Mounting Hardware category for additional parts required.

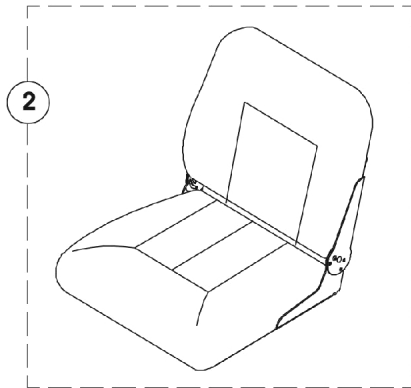

**Van Seat  
Fold-Down,  
Medium Back**

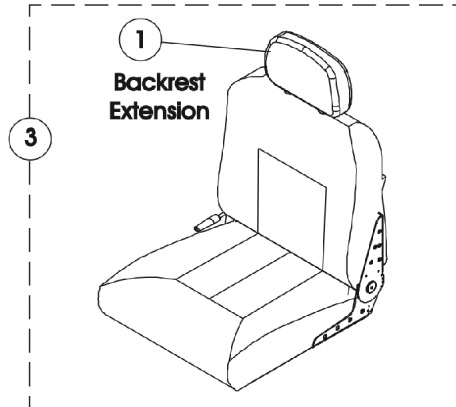



**Van Seat  
Semi-Recline**

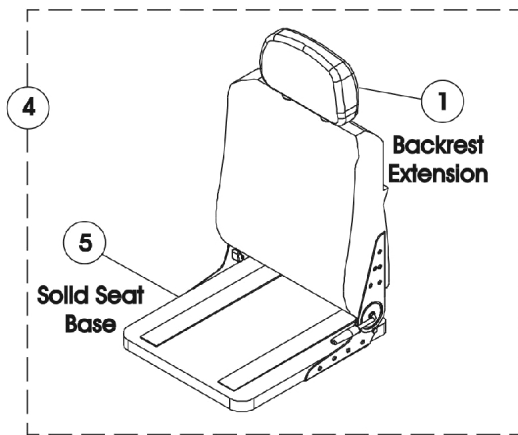

**Van Seat  
Semi-Recline  
w/ Solid Seat Base**

Office Style Van Seat includes arm receiver tube. The arm receiver tube, however, is NOT available as a service part. Also, this seat does not come with the front rigging hanger bracket (see Front Rigging Hanger

**Office Style Van Seat  
Semi-Recline (Charcoal grey vinyl)**

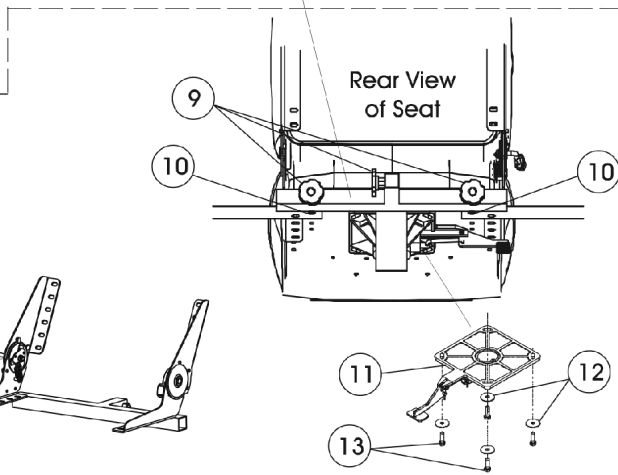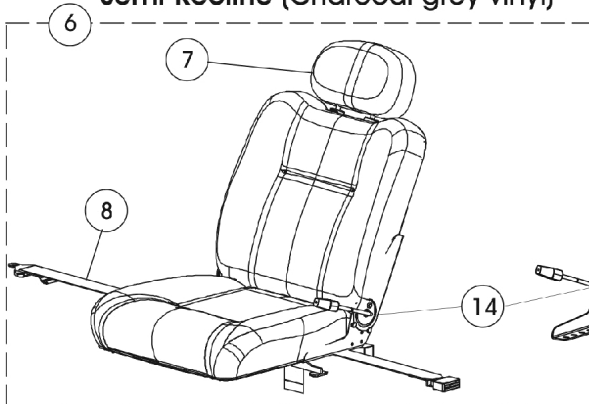

## Van Seat Assembly

**If converting a chair from Office Style Semi-Recline Van Seat to Semi-Recline Van Seat w/ Cushion or Semi-Recline Van Seat w/ Solid Seat Base, see Van Seat Frame and Mounting Hardware category for additional parts required.**

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1106969-NLA	NLA - Extension, Backrest, Van Seat, Grey Fabric	1	1
1		1108408	Extension, Backrest, Captain Seat, Grey Vinyl	1	1
1		1106968-NLA	NLA - Extension, Backrest, Van Seat, Tan Vinyl	1	1
2		See Chart	Fold Down Van Seat Assembly (18" Wide)	1	1
3		See Chart	Van Seat Assembly (Semi-Recline)	1	1
4		See Chart	Van Seat w/ Solid Seat Base Assembly (Semi-Recline)	1	1
5	A,D,E	1123384-NLA	NLA - Base, Solid, Captain Seat, Grey Vinyl (16" W)	1	1
5	A,D	1123385-NLA	NLA - Base, Solid, Van Seat, Grey Vinyl (18" W)	1	1
5	A,D	1159739	Base, Solid, Captain Seat, Grey Vinyl (18" W)	1	1
5	A,D	1123386-NLA	NLA - Base, Solid, Van Seat, Grey Vinyl (20" W)	1	1
5	A,D	1159740	Base, Solid, Captain Seat, Grey Vinyl (20" W)	1	1
6	B	See Chart	Office Style Van Seat Assembly	1	1
7		1127314	Extension, Backrest, Semi Recline Office Style Van Seat	1	1
8		1034625	Strap, Positioning, Seat/Chest w/ Auto Style Push Button Buckle (Eyelet to Eyelet Length 61")	1	1
9		1127312-NLA	NLA - Package, Adjustment Knob (5/16-18 x 3/4")	3	1
10	C	1127311-NLA	NLA - Package, Screw Socket Head Cap w/ Patch (1/4-20 x 3/4")	4	1
11		1116413-NLA	NLA - Pivot, Seat	1	1
12		1092409	Washer, Steel (5/16 x 1-1/4 x 1/16")	1	4
13		1118458-NLA	NLA - Screw, Hex Head w/Patch (1/4-20 x 1-3/8")	1	4
14		1144924	Bracket, Seat Frame, Black (18" W)	1	1
14		1148876	Bracket, Seat Frame, Black (20" W)	1	1

**NOTE:** A - Solid Seat Base is the only item available for service parts with regard to these particular Van Seats. Seat Hinges are not included.  
 B - Includes items 7-14. When replacing Office Style Seat, the front rigging hanger attachment kit may need to be ordered as well since it is not included (see Front Rigging Hanger category).  
 C - Package includes 4 screws, they are used to attach the Seat Positioning Strap (item 8)  
 D - Solid seat base is not available on Office Style Van Seats  
 E - 16" Solid Base is no longer available. A complete van seat should be ordered as replacement. See chart titled, "Van Seat (Semi-Recline) w/ Solid Seat Base Assembly."

### #2 Fold Down Van Seat Assembly (18" Wide)

**(Backrest Not Offered)**

All Grey Fabric Upholstery	All Grey Vinyl Upholstery	All Tan Vinyl Upholstery
N/A	1108837	N/A

Fold down, medium back van seat has four stop angle adjustment settings that include 95, 100, 105, & 110 degrees and adjustable seat depths of 16", 17", & 18".

### #3 Van Seat Assembly (Semi-Recline)

**When converting from a Van Seat w/ Cushion to a Van Seat w/ Solid Seat Base, a new Seat Mounting Bracket is required also. See Category titled "Frame, Van Seat and Mounting Hardware - M50/M51" to order the Seat Bracket.**

## Van Seat Assembly

If converting a chair from Office Style Semi-Recline Van Seat to Semi-Recline Van Seat w/ Cushion or Semi-Recline Van Seat w/ Solid Seat Base, see Van Seat Frame and Mounting Hardware category for additional parts required.

### #3 Van Seat Assembly (Semi-Recline)

When converting from a Van Seat w/ Cushion to a Van Seat w/ Solid Seat Base, a new Seat Mounting Bracket is required also. See Category titled "Frame, Van Seat and Mounting Hardware - M50/M51" to order the Seat Bracket.

Upholstery Type	Van Seat with Backrest 16" Wide	Van Seat with Backrest 18" Wide	Van Seat with Backrest 20" Wide	Hinge Bracket w/ Lever Semi-Recline Right	Hinge Bracket w/o Lever Semi-Recline Left	Screw, Phillips Truss Head (1/4-20 x 3/4")
All Grey Fabric	1150811-NLA	1109254-NLA	1159892-NLA See Note(s) A	1116634 See Note(s) B	1116635 See Note(s) B	5000136 See Note(s) C
All Grey Vinyl	1150810	1109242	1159891	1116634 See Note(s) B	1116635 See Note(s) B	5000136 See Note(s) C
All Tan Vinyl	1150812-NLA	1109248-NLA	1109249-NLA	1116634 See Note(s) B	1116635 See Note(s) B	5000136 See Note(s) C

NOTE: A - Was 1109255.

B - Not Shown. Hinge Brackets used for ALL Semi-Recline Seat models only. For attaching Hinge Bracket, see part number 5000136.

C - Not shown, Screw for Hinge Bracket(s) 1116634 and 1116635. Total of (7) required for each Bracket.

Semi-Recline Van Seat reclines approx. 90-114 degrees. Adjustable depths are (16", 17", & 18" deep for 16" & 18" wide seats) and (18", 19", & 20" deep for 20" wide seats). Recline release levers are standard on the right side. Left side release levers can be ordered as a special with a new van seat.

### #4 Van Seat w/ Solid Seat Base Assembly (Semi-Recline)

When converting from a Van Seat w/ Cushion to a Van Seat w/ Solid Seat Base, a new Seat Mounting Bracket is required also. See Category titled "Frame, Van Seat and Mounting Hardware - M50/M51" to order the Seat Bracket.

Upholstery Type	Van Seat with Backrest 16" Wide	Van Seat with Backrest 18" Wide	Van Seat with Backrest 20" Wide
All Grey Fabric	N/A	N/A	N/A
All Tan Vinyl	N/A	N/A	N/A
All Grey Vinyl	1150819	1150820	1159886

Semi-Recline Van Seat reclines approx. 90-114 degrees. Adjustable depths are (16", 17", & 18" deep for 16" & 18" wide seats) and (18", 19", & 20" deep for 20" wide seats). Recline release levers are standard on the right side. Left side release levers can be ordered as a special with a new van seat.

### #6 Office Style Van Seat Assembly

Charcoal Grey Vinyl Upholstery 18" Wide	Charcoal Grey Vinyl Upholstery 20" Wide
1127716-NLA	1148882-NLA

Semi-Recline Van Seat reclines approx. 90-114 degrees. Adjustable depths are 16", 17", & 18" deep. Recline release levers are standard on the right side.