


AT5043 Flip-Up Footplates

70 Deg. MFX and 90 Deg. Swingaway Supports

NOTE: Heel loops are not included. See footrest and legrest accessories.

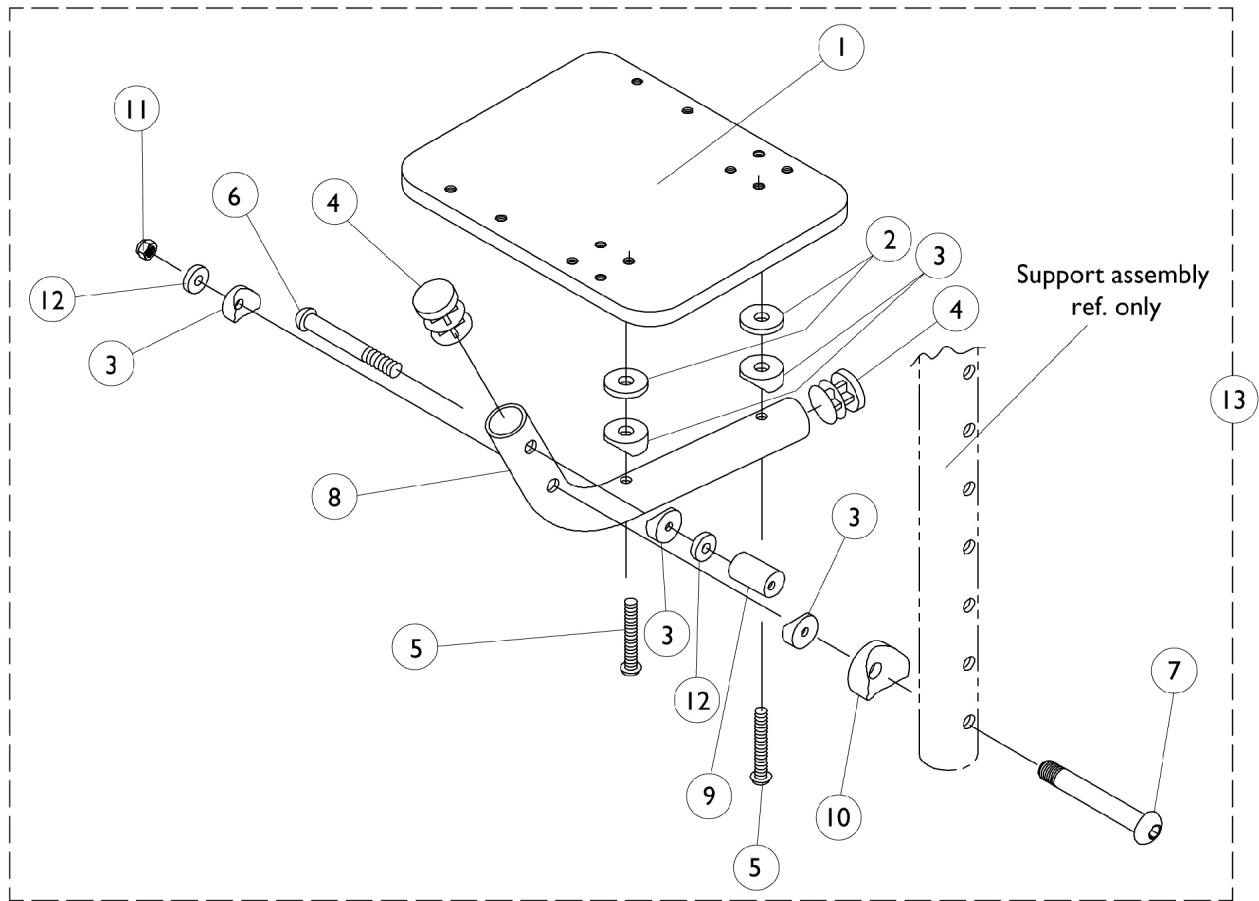

AT5043 Flip-Up Footplates

70 Deg. MFX and 90 Deg. Swingaway Supports

NOTE: Heel loops are not included. See footrest and legrest accessories.

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1044082-NLA	NLA - Footplate, Black (4-1/4" W x 5-1/4" L) (12" W)	1	1
2		1029454-NLA	NLA - Washer, Nylon (3/16 x 3/4 x 5/32")	1	2
3		1025457	Spacer, Coved (1/4 x 3/4 x 11/32")	1	5
4		1027097	Plug Button, Black (3/4")	1	2
5		1163311	Screw, Socket Button Head (10-32 x 1-1/4")	1	2
6		1008495	Screw, Socket Button Head (1/4-20 x 1-1/2")	1	1
7		1023967-NLA	NLA - Screw, Socket Button Head (1/4-20 x 2-5/8")	1	1
8		1029428-NLA	NLA - Tube, Footrest, Black	1	1
9		1028417-NLA	NLA - Cam, Adjustment	1	1
10		40157X020	Spacer, Coved (1/4 x 1 x 9/16")	1	1
11		7930333	Locknut (1/4-20)	25	1
12		1018739	Washer (1/4 x 5/8 x 3/32")	1	2
13		1029434-NLA	NLA - Flip-Up Footplate Assembly, Black - Right	1	1
13		1029435-NLA	NLA - Flip-Up Footplate Assembly, Black - Left	1	1