


# Tilt Hardware

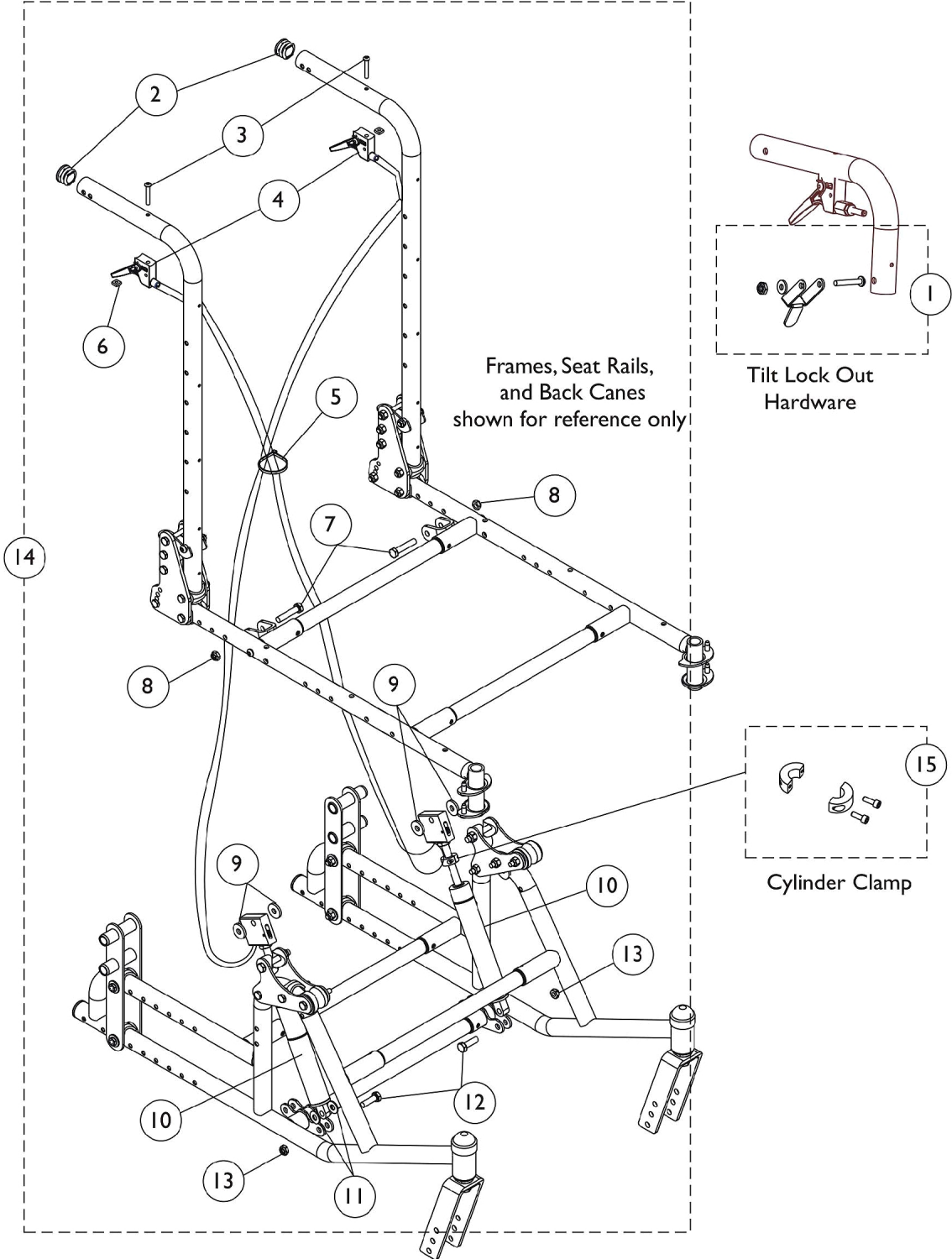

## Tilt Hardware

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1118578	Kit, Tilt Lock Out Lever with Hardware	1	1
			NLA - Nut, Hex Lock (#6-32)	1	1
			NLA - Screw, Phillips Pan Head (#6-32 x 7/8")	1	1
			NLA - Lever, Lock Out Tilt, Silver Vein	1	1
			NLA - Washer, Flat (5/32 x 3/8 x 1/32")	1	1
2		1029501	Plug Button, Black (1")	1	2
3		1163311	Screw, Socket Button Head (10-32 x 1-1/4")	1	2
4		1078451	Release Handle Assembly w/ Cable	1	2
5		1021260	Tie Wrap (5-5/8" L)	1	1
6		1086207	Nut, Square (#10-32)	1	2
7		1028514	Screw, Hex Head (5/16-24 x 1-1/2")	1	2
8		1025730	Locknut (5/16-24)	1	2
9		1016301	Washer, Nylon (5/16 x 7/8 x 1/8")	1	4
10		1075255	Cylinder, Gas w/ Release Cable - Right	1	1
10		1075256	Cylinder, Gas w/ Release Cable - Left	1	1
11		1074133	Washer, Steel (5/16 x 3/4 x 1/16")	1	2
12		1003548	Screw, Hex Head (5/16-18 x 1")	1	2
13		1028769	Locknut (5/16-18)	1	2
14	A	1121581	Kit, Tilt Hardware	1	1
15		1073707	Clamp, Cylinder with Screws	1	2

NOTE: A - Includes items 2-13