
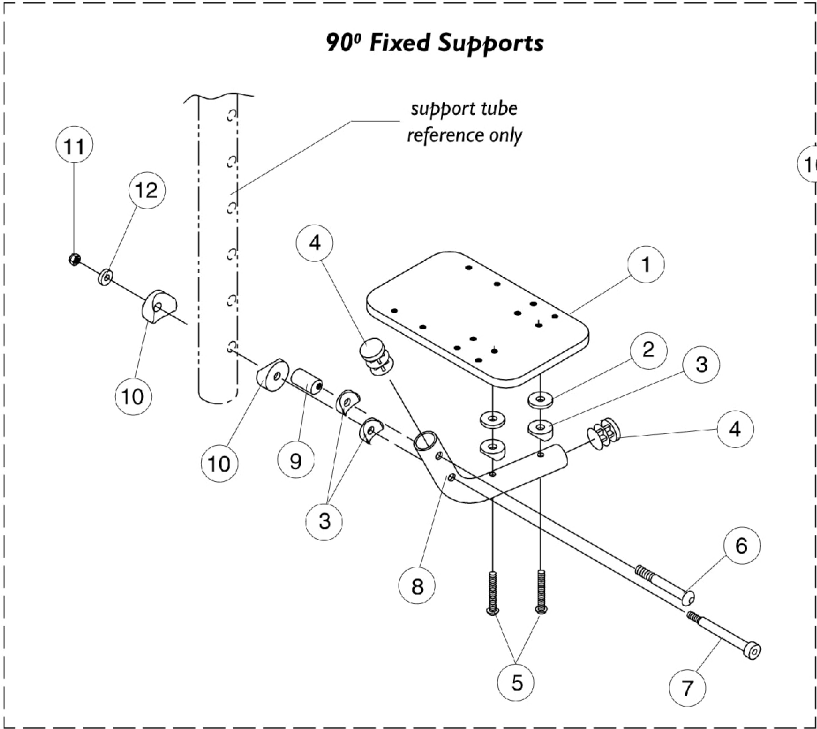


90 Degree Fixed Support Footrest Assembly - AT5043

After 1/92

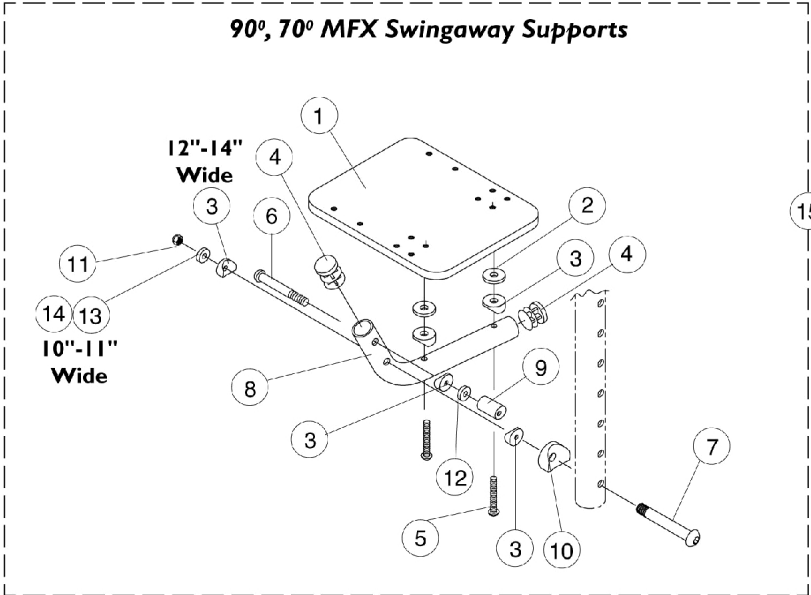

90 Degree Fixed Support Footrest Assembly - AT5043

After 1/92

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1044078-NLA	NLA - Footplate, Black (3-1/4" W x 5-1/4" L) (10" W)	1	1
1	B	1044082-NLA	NLA - Footplate, Black (4-1/4" W x 5-1/4" L) (12" W)	1	1
2		1029454-NLA	NLA - Washer, Nylon (3/16 x 3/4 x 5/32")	1	2
3		1025457	Spacer, Coved (1/4 x 3/4 x 11/32")	1	4
4		1027097	Plug Button, Black (3/4")	1	2
5		1163311	Screw, Socket Button Head (10-32 x 1-1/4")	1	2
6		1008495	Screw, Socket Button Head (1/4-20 x 1-1/2")	1	1
7	A	1030952	Screw, Shoulder Hex Socket Head (1/4" x 2", #10-24)	1	1
7	B	1023967-NLA	NLA - Screw, Socket Button Head (1/4-20 x 2-5/8")	1	1
8	A	1029429-NLA	NLA - Tube, Footrest, Black	1	1
8	B	1029428-NLA	NLA - Tube, Footrest, Black	1	1
9		1028417-NLA	NLA - Cam, Adjustment	1	1
10		40157X020	Spacer, Coved (1/4 x 1 x 9/16")	1	1
11		1025732-NLA	NLA - Locknut (#10-24)	1	1
12	A	1026034-NLA	NLA - Washer, Nylon (3/16 x 7/16 x 1/8")	1	1
13	A	1025715	Washer (1/4" x 7/16" x 1/16")	1	1
14	B	1018739	Washer (1/4 x 5/8 x 3/32")	1	1
15	A,C	1029432-NLA	NLA - Flip-Up Footplate Assembly - Right	1	1
15	A,C	1029433-NLA	NLA - Flip-Up Footplate Assembly - Left	1	1
15	B,C	1029434-NLA	NLA - Flip-Up Footplate Assembly, Black - Right	1	1
15	B,C	1029435-NLA	NLA - Flip-Up Footplate Assembly, Black - Left	1	1
16	A,D	1032699-NLA	NLA - Flip-Up Footplate Assembly - Right	1	1
16	A,D	1032700-NLA	NLA - Flip-Up Footplate Assembly - Left	1	1
16	B,D	1032703-NLA	NLA - Flip-Up Footplate Assembly - Right	1	1
16	B,D	1032704-NLA	NLA - Flip-Up Footplate Assembly - Left	1	1

NOTE: A - 10-11" Wide Chairs
B - 12-14" Wide Chairs
C - Swingaway Front Frames
D - Fixed Front Frames