
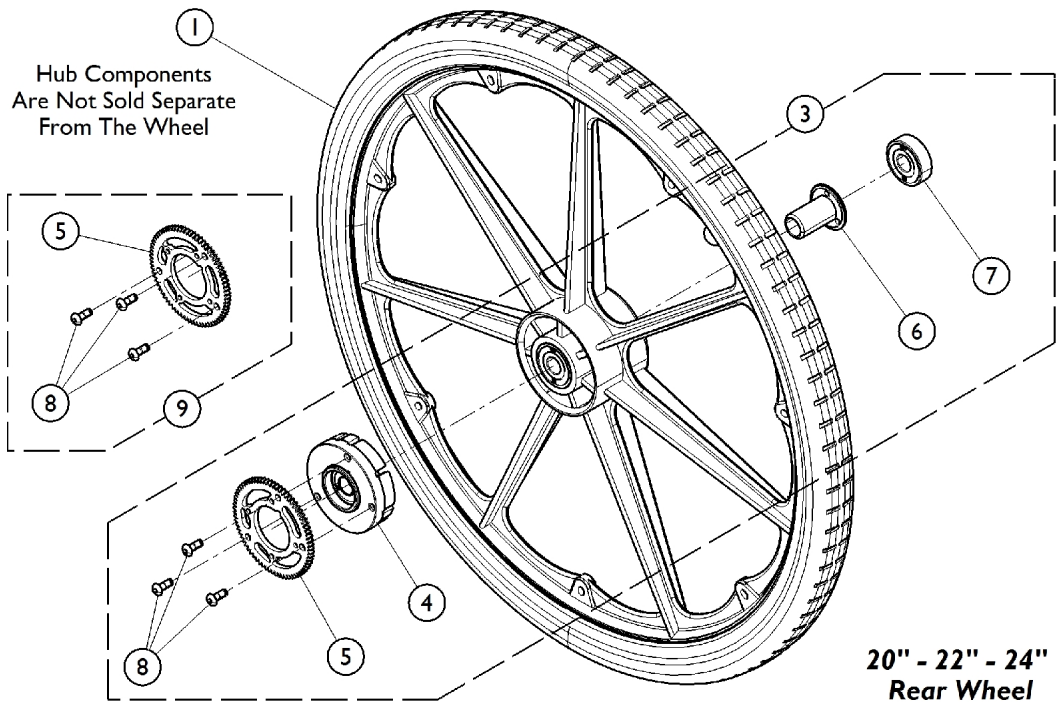


# Composite Rear Wheels - w/ Hublocks, w/o Handrims (18", 20", 22", 24" Wheels)

The original wheel hardware is not compatible with these replacement wheels, please order the corresponding wheel hublock hardware kit.

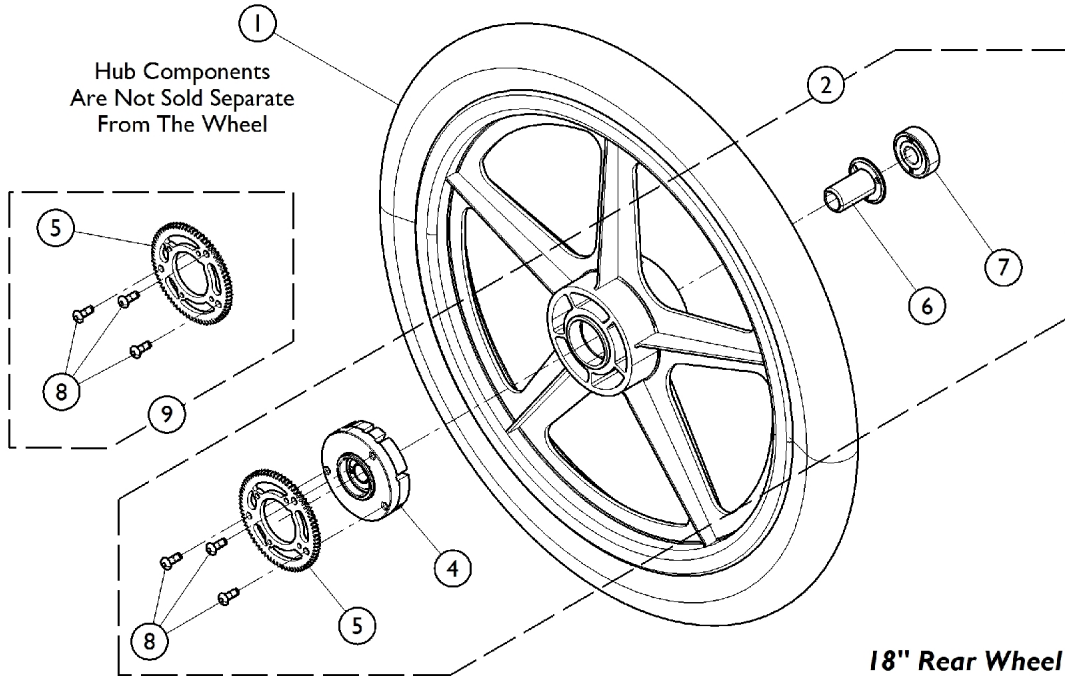

# Composite Rear Wheels - w/ Hublocks, w/o Handrims (18", 20", 22", 24" Wheels)

The original wheel hardware is not compatible with these replacement wheels, please order the corresponding wheel hublock hardware kit.

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		See Chart	#1 Rear Wheels w/ Hublocks	1	1
2	A	1164042	Kit, Hublock Hardware, Wheel (18")	1	1
4			Hub Assembly, Composite	1	1
5			Ring, Gear, Hublock	1	1
6			Nut, Hub Retaining	1	1
7			Bearing (1/2" I.D. x 1-3/8" O.D.)	1	1
8			Screw, Button Head Cap w/ Patch (#10-24 x 1/2)	1	3
3	A	1164038-NLA	NLA - Kit, Hublock Hardware, Wheel (20", 22", 24")	1	1
4			Hub Assembly, Composite	1	1
5			Ring, Gear, Hublock	1	1
6			Nut, Hub Retaining	1	1
7			Bearing (1/2" I.D. x 1-3/8" O.D.)	1	1
8			Screw, Button Head Cap w/ Patch (#10-24 x 1/2)	1	3
9	B	1164035	Kit, Hublock Hardware, Wheel (20" - 24")	1	1
5			Ring, Gear, Hublock	1	1
8			Screw, Button Head Cap w/ Patch (#10-24 x 1/2)	1	3

NOTE: A - Kit includes items 4 - 8.  
B - Kit includes items 5 and 8.

## #1 Rear Wheels with Hublocks

Style	Wheel Size	Wheel Asm. w/o Handrim	Tire Type	Tire	Tube/Insert	Outer Bearing	Inner Bearing
18COM	18 x 1-3/4"	1074839	Pneumatic	1071441	1071440	40117X022	N/A
18COM	18 x 1-3/4"	1137256	Pneumatic w/ Flat Free	1071441	N/A	N/A	N/A
20COM	20 x 1-3/8"	1164064 See Note(s) A	Pneumatic	1017519	1017520	40117X022	N/A
20COM	20 x 1-3/8"	1164065 See Note(s) A	Pneumatic w/ Flat Free	1017519	1023322	40117X022	N/A
20COM	20 x 1-1/4"	1164066 See Note(s) A	Urethane	1037774	N/A	40117X022	N/A
22COM	22 x 1-3/8"	1164067 See Note(s) A	Pneumatic	1018544	1018545	40117X022	N/A
22COM	22 x 1-3/8"	1164068 See Note(s) A	Pneumatic w/ Flat Free	1018544	1023323	40117X022	N/A
22COM	22 x 1-1/4"	1164069 See Note(s) A	Urethane	1028250	N/A	40117X022	N/A
24COM7	24 x 1-3/8"	1164070 See Note(s) A	Pneumatic	40176X060	40176X061	40117X022	N/A
24COM7	24 x 1-3/8"	1164071 See Note(s) A	Pneumatic w/ Flat Free	40176X060	1010906	40117X022	N/A
24COM7	24 x 1-1/4"	1164072 See Note(s) A	Urethane	1026904	N/A	40117X022	N/A

# **Composite Rear Wheels - w/ Hublocks, w/o Handrims (18", 20", 22", 24" Wheels)**

**The original wheel hardware is not compatible with these replacement wheels, please order the corresponding wheel hublock hardware kit.**

NOTE: A - These wheels come with assembled with Hub Assembly, Retaining Nut, and Bearing.