


Pivot/Slide Tube w/ Tubular Footplates with Attaching Hardware and Heel Loops

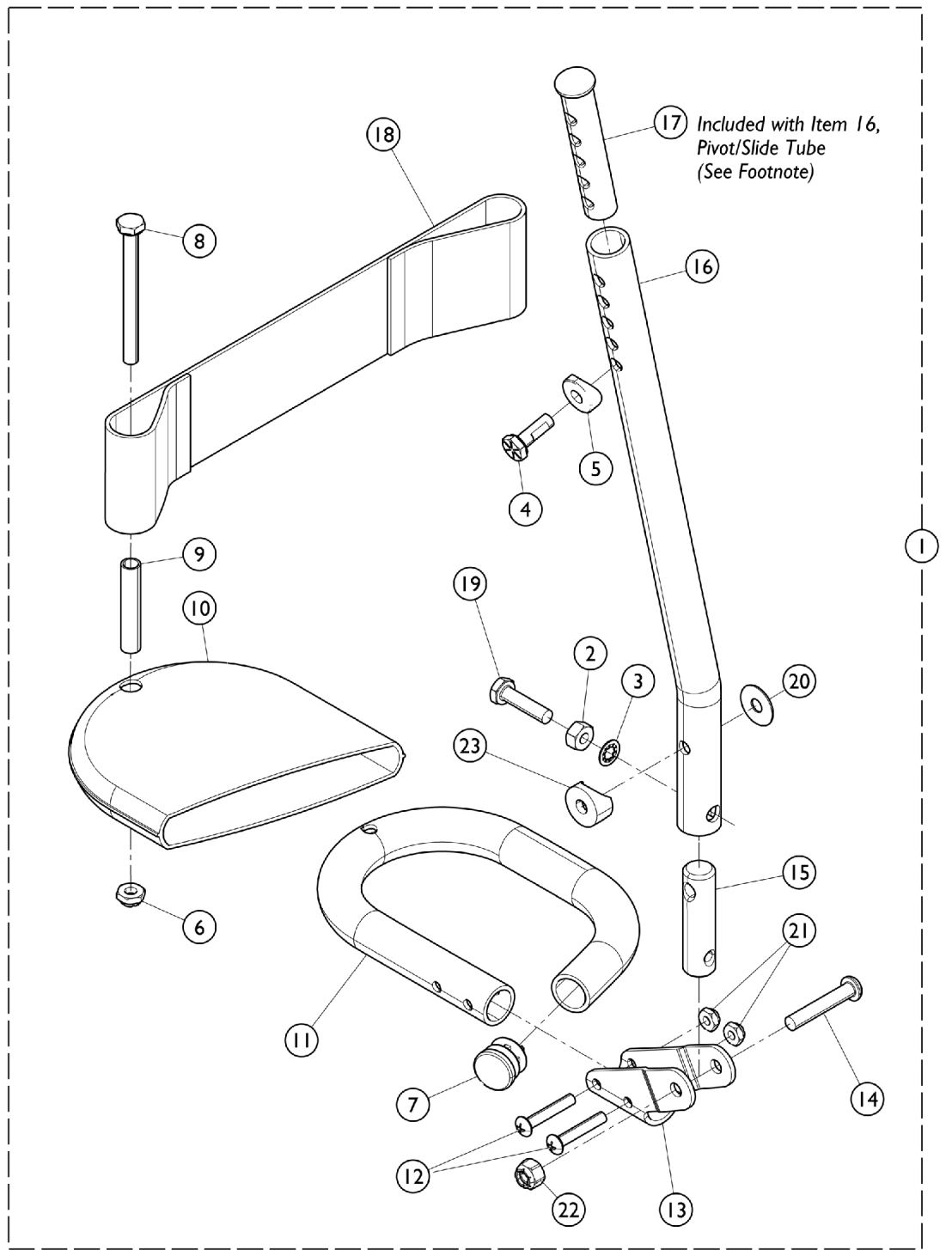

Pivot/Slide Tube w/ Tubular Footplates with Attaching Hardware and Heel Loops

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	B	See Chart	#1 Tubular Footplate Assembly	1	1
2		1025711-NLA	NLA - Nut, Hex (1/4-20)	1	1
3		1026108	Lockwasher (1/4 x 15/32 x 1/64")	1	1
4		1045267	Screw, Hex Head w/ Patch (1/4-20 x 3/4")	1	1
5		1026099	Spacer, Coved (1/4 x 11/16 x 1/2")	1	1
6		1025195	Locknut (1/4-20)	1	1
7		1027097	Plug Button, Black (3/4")	1	1
8		1025709-NLA	NLA - Screw, Hex Head (1/4-20 x 2-3/4")	1	1
9		1006477-NLA	NLA - Spacer (1/4 x 11/32 x 1-3/4")	1	1
10	C	1008552-NLA	NLA - Cover, Footplate - Medium	1	1
10		1008553-NLA	NLA - Cover, Footplate - Large	1	1
11		1026404-NLA	NLA - Tube, Footrest, Black - Medium	1	1
11		1026405-NLA	NLA - Tube, Footrest, Black - Large	1	1
12		1025714	Screw, Phillips Pan Head (#10-32 x 1-1/8")	1	2
13		1012182-NLA	NLA - Bracket, Pivot, Black	1	1
14		1023968-NLA	NLA - Screw, Socket Button Head (1/4-20 x 1-7/16")	1	1
15	C	1012158-NLA	NLA - Insert, Lower Slide Tube	1	1
16	A	1026406-NLA	NLA - Tube, Pivot/Slide w/ Threaded Insert, Black	1	1
16	A	1020533-NLA	NLA - Tube, Pivot/Slide w/ Threaded Insert, Black (2" Longer)	1	1
17	A	1020534	Insert, Threaded	1	1
18		1017621	Heel Loop - Right/Left	1	1
19		1025713-NLA	NLA - Screw, Hex Head (1/4-20 x 1")	1	1
20		1025456	Washer, Nylon (3/4 x 1/4 x 1/32")	1	1
21		1025735	Locknut (#10-32)	1	2
22		1025736	Locknut (1/4-20)	1	1
23		1025457	Spacer, Coved (1/4 x 3/4 x 11/32")	1	1

NOTE: A - Pivot/Slide Tube (Item 16) includes the Threaded Insert (Item 17) which can be ordered separately and is identified in the drawing. Item 17 was revised from 3 to 5 Holes as of 8/15/03 and is backward compatible and does not require ordering in pairs.
 B - Includes items 2 - 23.
 C - Part number is no longer available as a service part individually. See item 1 and order a complete Tubular Footplate Assembly as a replacement.

#1 Tubular Footplate Assemblies

Right/Left	Medium 15" W - Non Tapered 16-20" W - Tapered	Medium, 2" Longer 15" W - Non Tapered 16-20" W - Tapered	Large 16-20" W Non Tapered	Large, 2" Longer 16-20" W Non Tapered
Right	1026695-NLA	1055509-NLA	1026699-NLA	1055511-NLA
Left	1026696-NLA	1055510-NLA	1026700-NLA	1055512-NLA