


Pivot/Slide Tube w/ Flip-Up Footplates - A1562

60 Degree MFX and 90 Degree Support Assemblies

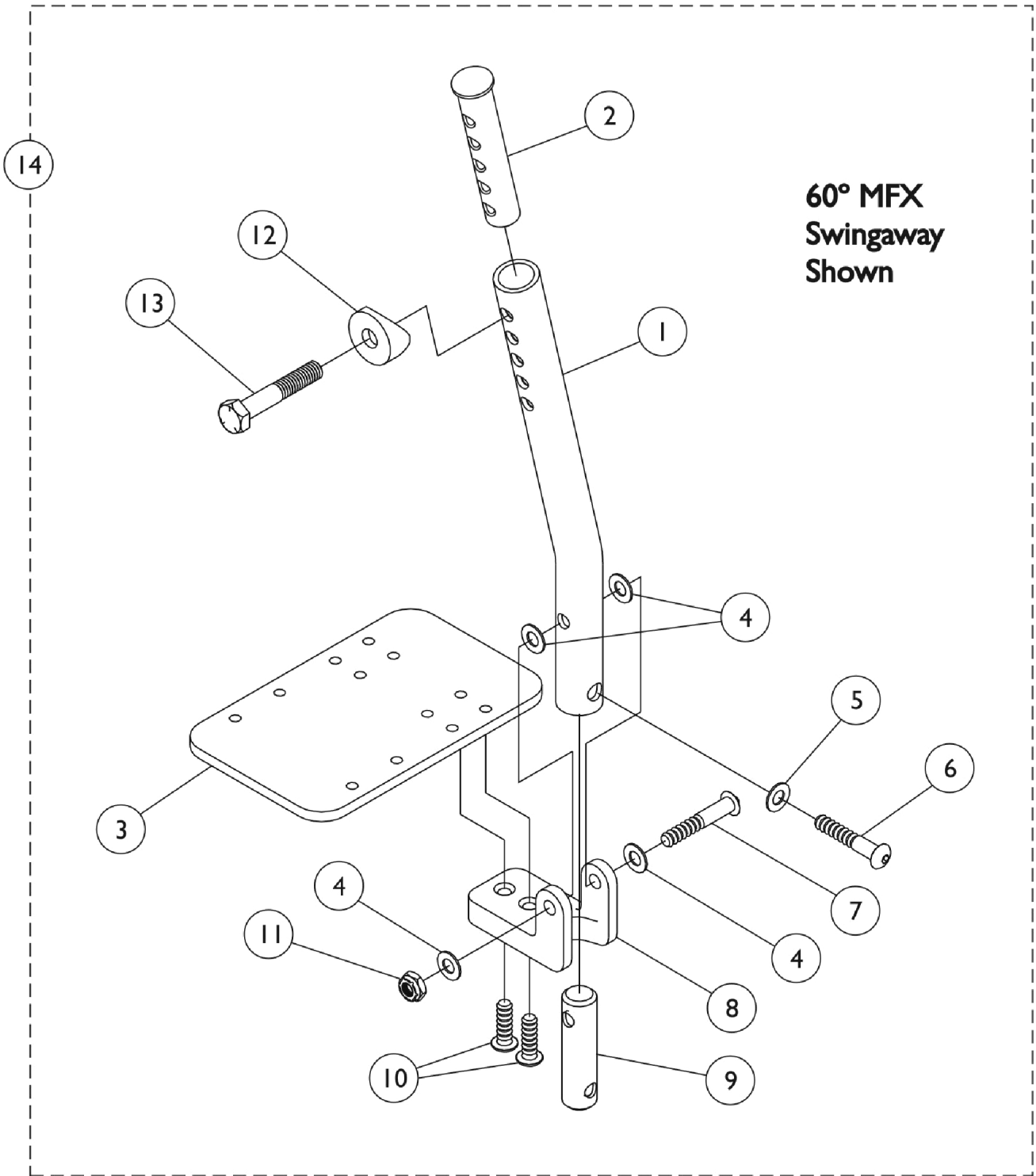

Pivot/Slide Tube w/ Flip-Up Footplates - A1562

60 Degree MFX and 90 Degree Support Assemblies

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1026407-NLA	NLA - Tube, Pivot/Slide, 60 Degree MFX	1	1
1		1026681-NLA	NLA - Tube, Pivot/Slide, 90 Degree MFX	1	1
2	C	1020534	Insert, Threaded	1	1
3	A	1044078-NLA	NLA - Footplate, Black (3-1/4" W x 5-1/4" L) (10" W)	1	1
3	B	1044082-NLA	NLA - Footplate, Black (4-1/4" W x 5-1/4" L) (12" W)	1	1
4		1026152-NLA	NLA - Washer (1/4 x 5/8 x 1/32")	1	4
5		1026108	Lockwasher (1/4 x 15/32 x 1/64")	1	1
6		1007269	Screw, Socket Button Head (1/4-20 x 5/8")	1	1
7		1023968-NLA	NLA - Screw, Socket Button Head (1/4-20 x 1-7/16")	1	1
8		1022817-NLA	NLA - Block, Hinge	1	1
9	D	1012158-NLA	NLA - Insert, Lower Slide Tube	1	1
10		1025716	Screw, Phillips Pan Head (#10-32 x 11/16")	1	2
11		1025195	Locknut (1/4-20)	1	1
12		1026099	Spacer, Coved (1/4 x 11/16 x 1/2")	1	1
13		1045267	Screw, Hex Head w/ Patch (1/4-20 x 3/4")	1	1
14	A	1026408-NLA	NLA - Pivot Slide Tube w/ Footplate, Black - Right (60 Degree MFX)	1	1
14	A	1026409-NLA	NLA - Pivot Slide Tube w/ Footplate, Black - Left (60 Degree MFX)	1	1
14	B	1026690-NLA	NLA - Pivot Slide Tube w/ Footplate, Black - Right (60 Degree MFX)	1	1
14	B	1026691-NLA	NLA - Pivot/Slide Tube w/ Footplate, Black - Left (60 Degree MFX)	1	1
14	B	1034342-NLA	NLA - Pivot Slide Tube w/ Footplate, Black - Right (90 Degree MFX)	1	1
14	B	1034343-NLA	NLA - Pivot/Slide Tube w/ Footplate, Black - Left (90 Degree MFX)	1	1

NOTE: A - 10" Wide Chairs
 B - 12-14" Wide Chairs
 C - As of 8/15/03, the Threaded Insert was revised from 3 to 5 Holes. Threaded Insert is Backward Compatible and does not require ordering in Pairs.
 D - Part number is no longer available individually. See item 14 and order the complete Pivot Slide Tube w/ Footplate Assembly.