


Footrest Support Assembly

93ST - Black, Chrome, and Color

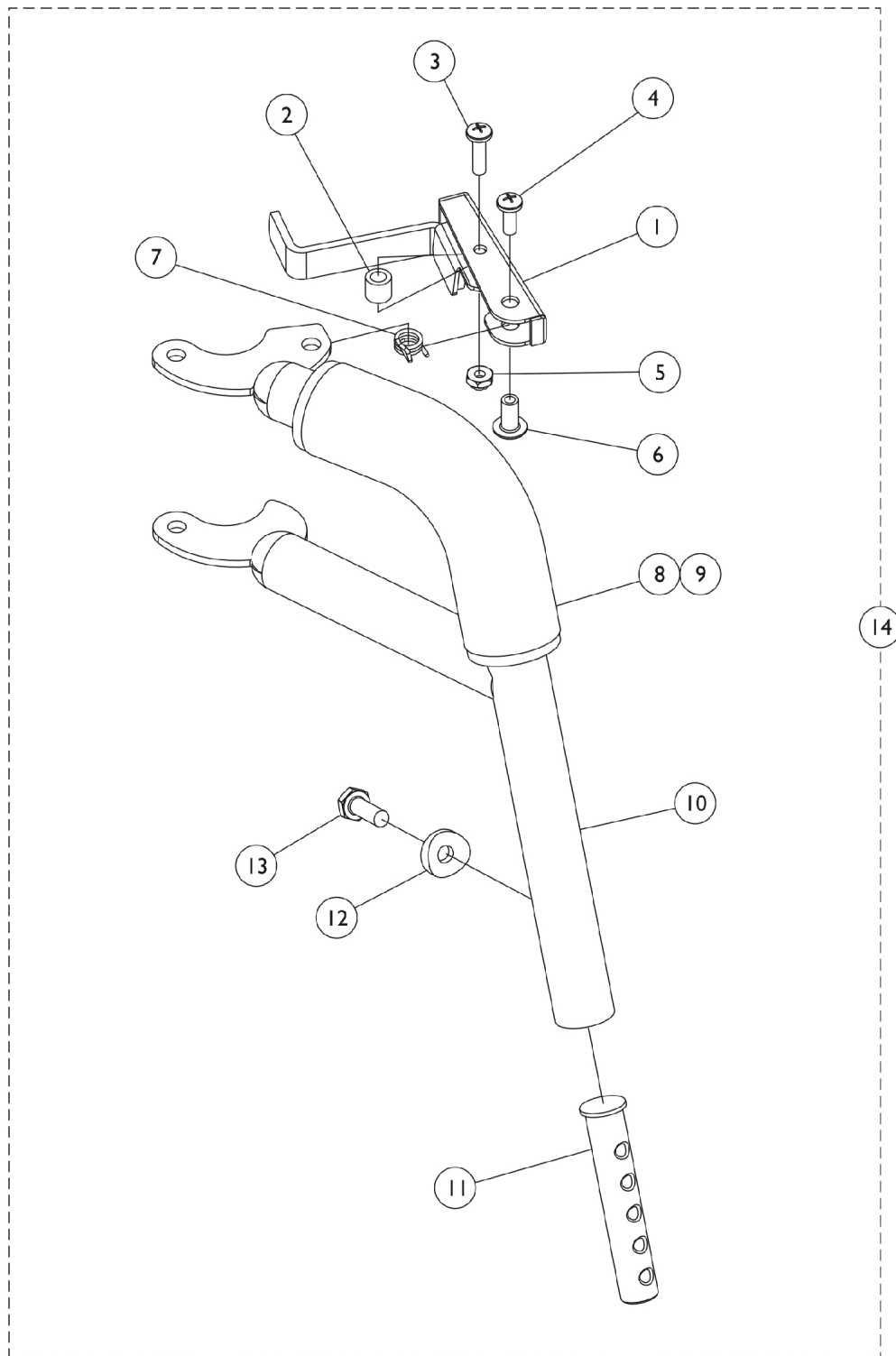

Footrest Support Assembly

93ST - Black, Chrome, and Color

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1035539	Handle, Release, Black - Right	1	1
1		1035540	Handle, Release, Black - Left	1	1
2		47565X057	Bushing (7/32 x 3/8 x 11/32")	1	1
3		1025716	Screw, Phillips Pan Head (#10-32 x 11/16")	1	1
4		1026114	Screw, Phillips Pan Head (#10-32 x 1/2")	1	1
5		1025735	Locknut (#10-32)	1	1
6		1028846-NLA	NLA - Rivnut (#10-32)	1	1
7		1028707	Spring, Release - Right	1	1
7		1028708	Spring, Release - Left	1	1
8		1030435-NLA	NLA - Cover, Foam Cushion	1	1
9		1030434-NLA	NLA - Cushion, Tubular Foam	1	1
10		1028013-NLA	NLA - Tube, Footrest Support, Adult, Black - Right	1	1
10		1028014-NLA	NLA - Tube, Footrest Support, Adult, Black - Left	1	1
10		1027980-NLA	NLA - Tube, Footrest Support, Adult, Chrome - Right	1	1
10		1027981-NLA	NLA - Tube, Footrest Support, Adult, Chrome - Left	1	1
10	A	8881037852-NLA	NLA - Tube, Footrest Support, Adult, Color - Right	1	1
10	A	8881037853-NLA	NLA - Tube, Footrest Support, Adult, Color - Left	1	1
11	E	1020534	Insert, Threaded	1	1
12		1026099	Spacer, Coved (1/4 x 11/16 x 1/2")	1	1
13		1045267	Screw, Hex Head w/ Patch (1/4-20 x 3/4")	1	1
	B,C	1028826-NLA	NLA - Tip, Release Handle, Grey	1	1
	B,C	1028823-NLA	NLA - Tip, Release Handle, Teal	1	1
	B,C	1028824-NLA	NLA - Tip, Release Handle, Lavender	1	1
	B,C	1028825-NLA	NLA - Tip, Release Handle, Red	1	1
14	D	1028015-NLA	NLA - Footrest Support Assembly, Adult, Black - Right	1	1
14	D	1028016-NLA	NLA - Footrest Support Assembly, Adult, Black - Left	1	1
14	D	1027992-NLA	NLA - Footrest Support Assembly, Adult, Chrome - Right	1	1
14	D	1027993-NLA	NLA - Footrest Support Assembly, Adult, Chrome - Left	1	1
14	A,D	8881037854-NLA	NLA - Footrest Support Assembly, Adult, Color - Right	1	1
14	A,D	8881037855-NLA	NLA - Footrest Support Assembly, Adult, Color - Left	1	1
	C	See Chart	Pivot Slide Tube with Composite Footplate	1	1

Footrest Support Assembly

93ST - Black, Chrome, and Color

NOTE: A - Specify frame color when ordering, see Frame Finishes, Touch-Up Paints, Upholstery Options and Accent Packages page.
B - Before 10/1/00 only
C - Not Shown
D - Assembly includes Release Handle, Tip, Bolt, and Locknut.
E - As of 8/15/03, the Threaded Insert was revised from 3 to 5 Holes. Threaded Insert is Backward Compatible and does not require ordering in Pairs.

Pivot Slide Tube with Composite Footplate

Description	Size	Right	Left
Narrow Adult	16"	1028391	1028392
Adult	18"	1028387	1028388
Large Narrow	20"	1028383	1028384