



Casters - Mounting Hardware (After 6/29/11)


Caster Mounting Hardware
Caster Options
AT900, 41DC, 4SPLT, 51DC
61DC, 615DC

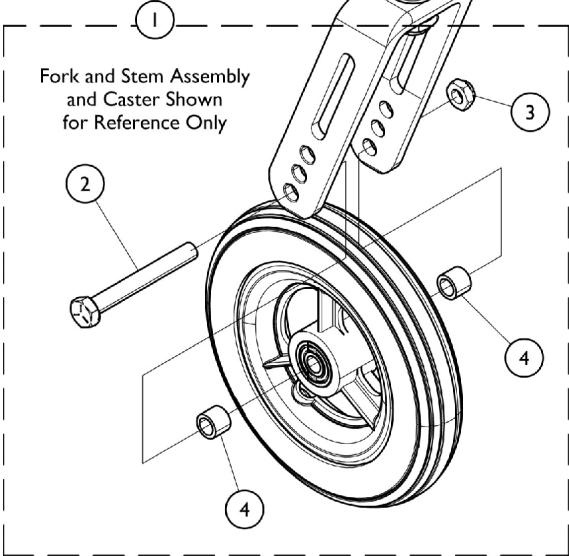

Caster Mounting Hardware
Caster Option
1259

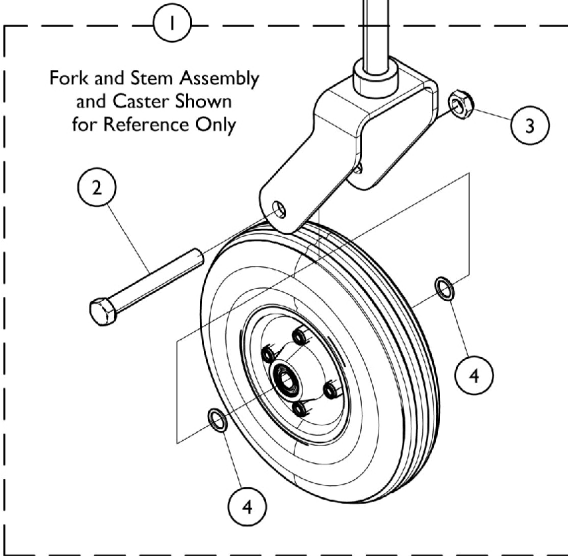

Caster Mounting Hardware
Caster Options
81DC, 815DC

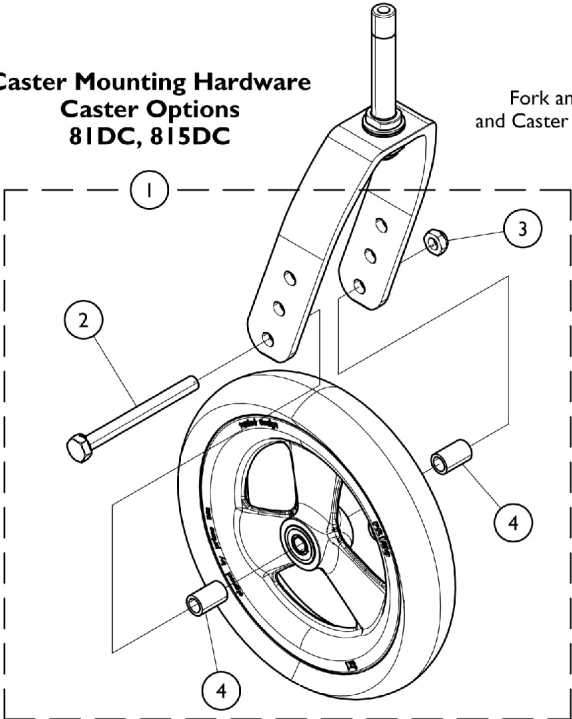

Casters - Mounting Hardware (After 6/29/11)

Caster Option	Wheel Size	Tire Type	Item 1. Kit, Caster Hardware (Pair)	Item 2. Axle	Item 3. Locknut	Item 4. Axle Spacer
AT900	3"	Roller	1026926-NLA	1026908	1025730	1026160
4SPLT 41DC 5SOFT 51DC	4 x 1" 4 x 1" 5 x 1" 5 x 1"	Urethane	1026926-NLA	1026908	1025730	1026160
61DC 615DC	6 x 1" 6 x 1-1/2"	Urethane	1026926-NLA	1026908	1025730	1026160
1259	6 x 2"	Semi Pneumatic	1042489 See Note(s) A	1025726	1025734	1025724
81DC	8 x 1"	Urethane	1161851-NLA	N/A	N/A	N/A
815DC	8 x 1-1/2"	Urethane	1161851-NLA	N/A	N/A	N/A
NOTE: A - Caster Hardware #1042489 accommodates 1 side only.						
NOTE: Kit, Caster Hardware includes Axle, Locknut, and Axle Spacer.						