


Forks & Hardware (After 6/29/11)

(Upper hole on fork determines the lowest STF dimension.)

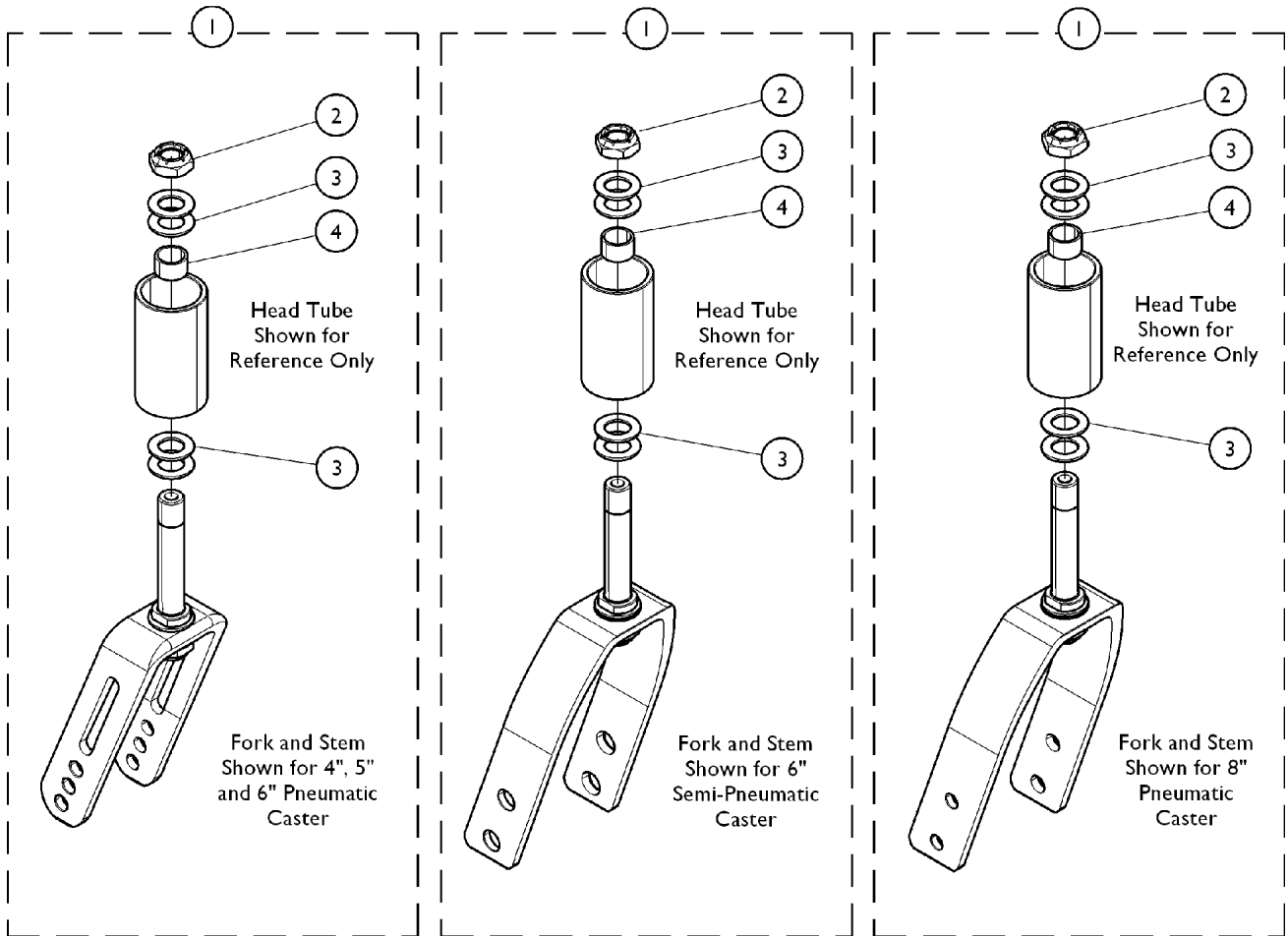

**Hardware for Caster Options
4SPLT, 5SOFT, 5IDC,
515DC, 61DC**

**Hardware for Caster Options
1259**

**Hardware for Caster Options
81DC, 815DC**

Forks & Hardware (After 6/29/11)

(Upper hole on fork determines the lowest STF dimension.)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1118545	Kit, Fork (5") w/ Short Stem	1	1
			NLA - Fork, Caster	1	1
			Stem, Fork, Short	1	1
2			Locknut (1/2-20)	1	2
3			Washer, Plain (1/2 x 7/8 x 1/16)	1	6
4			Spacer (1/2 x 5/8 x 3/8")	1	1
1	A	1118546	Kit, Fork Assembly w/ Hardware (6 x 2" - 8" Caster)	1	1
			Fork, Caster	1	1
			Stem, Fork, Short	1	1
2			Locknut (1/2-20)	1	2
3			Washer (1/2 x 7/8 x 3/64")	1	6
4			Spacer (1/2 x 5/8 x 3/8")	1	1
1	A	1111933	NLA - Kit, Fork and Stem Assembly w/ Hardware (3 - 4" Caster)	1	1
			Fork, Caster	1	1
			Stem, Fork, Short	1	1
2			Locknut (1/2-20)	1	2
3			Washer, Plain (1/2 x 7/8 x 1/16)	1	6
4			Spacer (1/2 x 5/8 x 3/8")	1	1
1	A	1174529	Kit, Fork Assembly w/ Hardware (8" Caster)	1	1
			Fork, Caster	1	1
			Stem, Fork, Short	1	1
2			Locknut (1/2-20)	1	2
3			Washer (1/2 x 7/8 x 3/64")	1	7
4			Spacer (1/2 x 5/8 x 3/8")	1	1
2		1025737	Locknut (1/2-20)	1	1
3		1026174	Washer (1/2 x 7/8 x 3/64")	1	4
4		1111931	Spacer (1/2 x 5/8 x 3/8")	1	1
	B	1088523	Plug Button, Head Tube, Black	1	1
		See Chart	Fork Assembly w/ Hardware, Rear Wheels and STF Dimensions	1	1

NOTE: A - Includes pre-assembled Fork and Stem in addition to Head Tube mounting hardware items 2, 3, and 4.

B - Not shown in illustration.

Forks & Hardware (After 6/29/11)

(Upper hole on fork determines the lowest STF dimension.)

Fork Assembly w/ Hardware, Rear Wheels and STF Dimensions

Caster Option	Caster Size	Rear Wheel Option	Seat to Floor Dimensions	Part Number
4SPLT	4 x 1"	18COM (18")	14-1/2"	1111933
4SPLT	4 x 1"	12COM (12") 18COM (18") 20COM (20")	16" 14-1/2 - 16" 15 - 15-1/2"	1118545
5SOFT 51DC, 515DC	5 x 1" 5 x 1-1/2"	12COM (12") 18COM (18") 20COM (20") 22COM (22")	16 - 16-1/2" 14-1/2 - 16" 15 - 16-1/2" 16 - 16-1/2"	1118545
61DC	6 x 1"	12COM (12") 18COM (18") 20COM (20") 22COM (22") 24COM (24")	16 - 17" 15-1/2 - 16" 15-1/2 - 17" 16 - 17" 17"	1118545
1259	6 x 2"	12COM (12") 18COM (18") 20COM (20") 22COM (22") 24COM (24")	16 - 18" 16" 16 - 16-1/2" 16 - 18" 17-1/2 - 18"	1118546
81DC, 815DC	8 x 1" 8 x 1-1/2"	12COM (12") 18COM (18") 20COM (20") 22COM (22") 24COM (24")	17 - 17-1/2" 18-1/2 - 19" 17" 17 - 17-1/2" 17 - 19"	1174529

Seat to Floor Heights are in 1/2 inch increments.