



Caster and Mounting Hardware (Before 7/1/11)

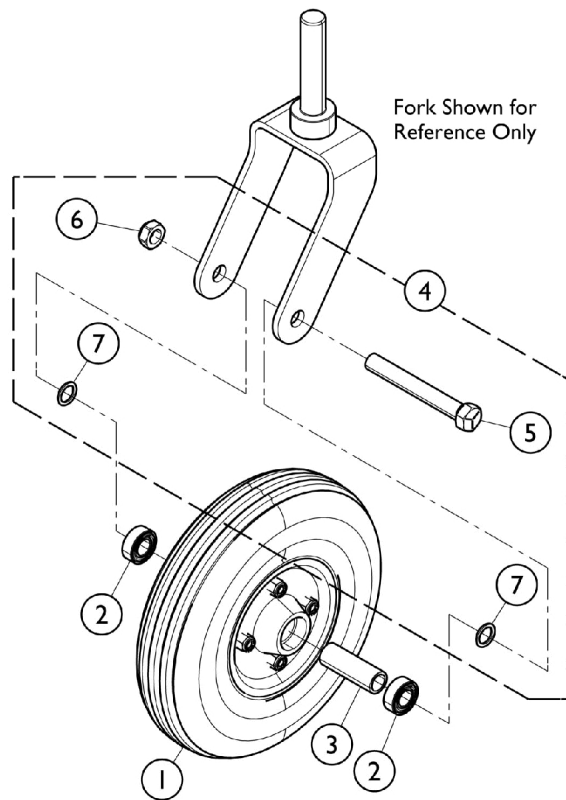

**Caster and Hardware for
1259, AT903, 1228 Options**

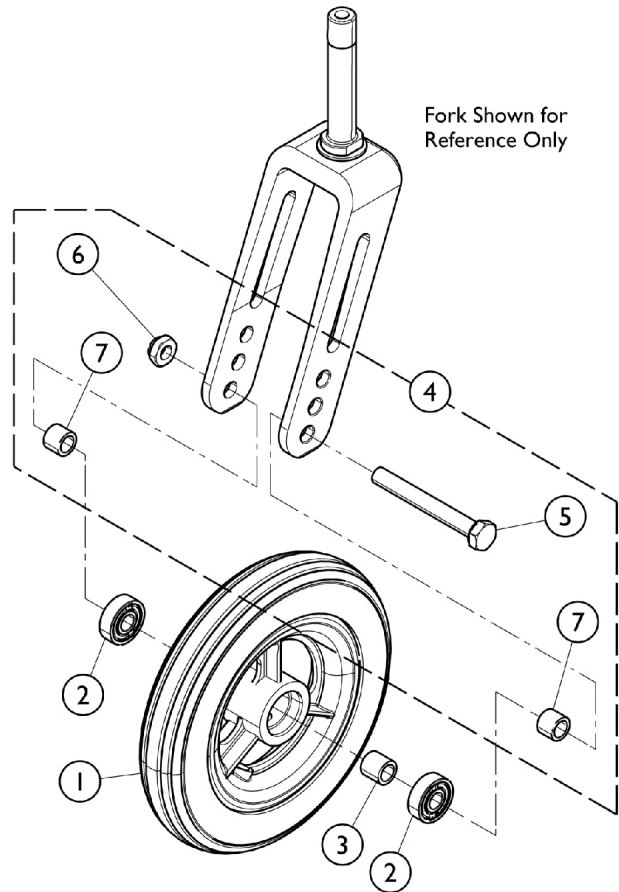

**Caster and Hardware for
4SPLT, 5ALUM2, 5SOFT
5SPLT, 6ALUM2, AT902 Options**

Caster and Mounting Hardware (Before 7/1/11)

Casters (Before 7/1/11)

Wheel Option	Wheel Size	Tire Type	Item 1 Caster Wheel Assembly	Item 2 Bearing	Item 3 Bearing Spacer	Wheel	Tire
4SPLT	4 x 1"	Urethane	1060910	40117X026	1026172	N/A	N/A
4ALUM2	4 x 1-1/2"	Urethane	1128070	40117X026	1026172	N/A	N/A
5SPLT	5 x 1"	Urethane	1159541	40117X026	N/A	N/A	N/A
5ALUM2	5 x 1-1/2"	Urethane	1128071	40117X026	1026172	N/A	N/A
5SOFT See Note(s) A	5 x 1-1/2"	Urethane	1128071	N/A	N/A	N/A	N/A
AT902	6 x 1"	Urethane	1034352 See Note(s) B	N/A	N/A	N/A	N/A
6ALUM2	6 x 1-1/2"	Urethane	1128072	40117X026	1026172	N/A	N/A
1259	6 x 2"	Semi Pneumatic Urethane	1115179	1006049	1020592	N/A	N/A
AT903	8 x 1-1/4"	Urethane	1038161	1006049	1020592	N/A	N/A
1228	8 x 1-3/4"	Urethane	1070811	1006049	1020592	N/A	N/A

NOTE: A - After 10/4/09, the 5SOFT (5") Semi-Pneumatic is no longer available and is replaced as a 5SOFT (5") Urethane. Order caster in pairs for wheelchairs manufactured before 10/5/09.

B - Consists of (2) Casters.

NOTE: Caster Wheel Assembly includes Tire, Wheel, Bearings (2), and Spacer

Caster and Mounting Hardware (Before 7/1/11)

Caster Hardware (Before 7/1/11)

Wheel Option	Wheel Size	Tire Type	Item 4. Caster Hardware (Pair)	Item 5. Axle	Item 6. Locknut	Item 7. Axle Spacer/ Washer
4SPLT, 5ALUM 5SOFT, 5SPLT 5ALUM2	4", 5"	Urethane	1154001	40317X000	60264X001	1026160
6ALUM2	6 x 1-1/2"	Urethane	1154001	40317X000	60264X001	1026160
AT902	6 x 1"	Urethane	1154001	40317X000	60264X001	1026160
1259	6 x 2"	Semi Pneumatic	1154000	N/A	N/A	N/A
AT903	8 x 1-1/4"	Urethane	1154000	N/A	N/A	N/A
1228	8 x 1-3/4"	Urethane	1154000	N/A	N/A	N/A