
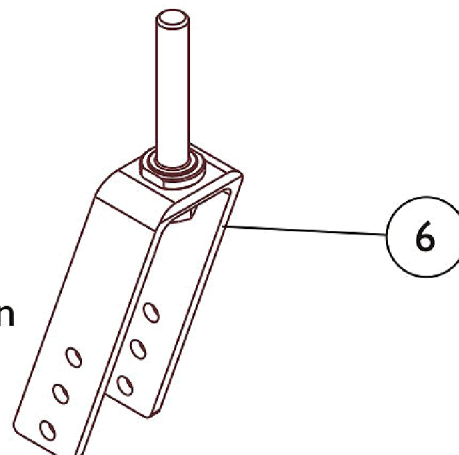


## Caster Forks & Hardware (Before 7/1/11)

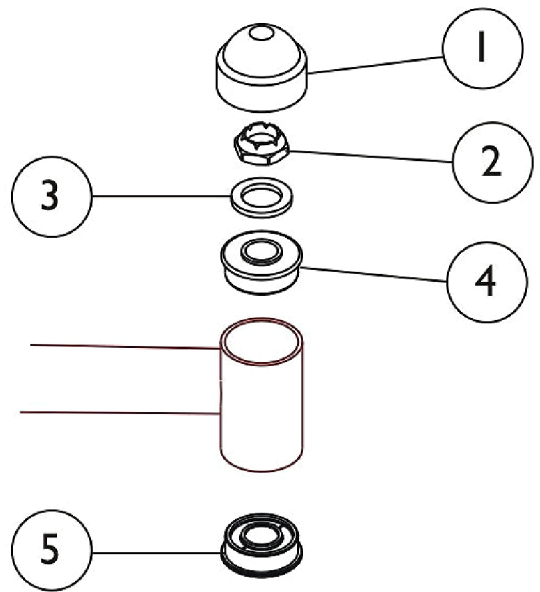

6" Caster Fork shown

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1001843	Cap, Head Tube, Black	1	1
1		1131189	Package, Head Tube Cap	10	1
2		1001979	Locknut (1/2-20)	1	1
3		1021508	Washer, Nylon (5/8 x 1 x 3/32")	1	1
4		1001801	Bearing, Flange (1/2")	1	1
5		1085298	Bearing (1-1/4" O.D. x 1/2" I.D.)	1	1
6		1037420	Fork and Stem Assembly, Black (6 x 1" Caster)	1	1
6		1056640	Fork and Stem Assembly, Black (8" & 6 x 2" Caster)	1	1