


Forks & Hardware (Before 7/1/11)

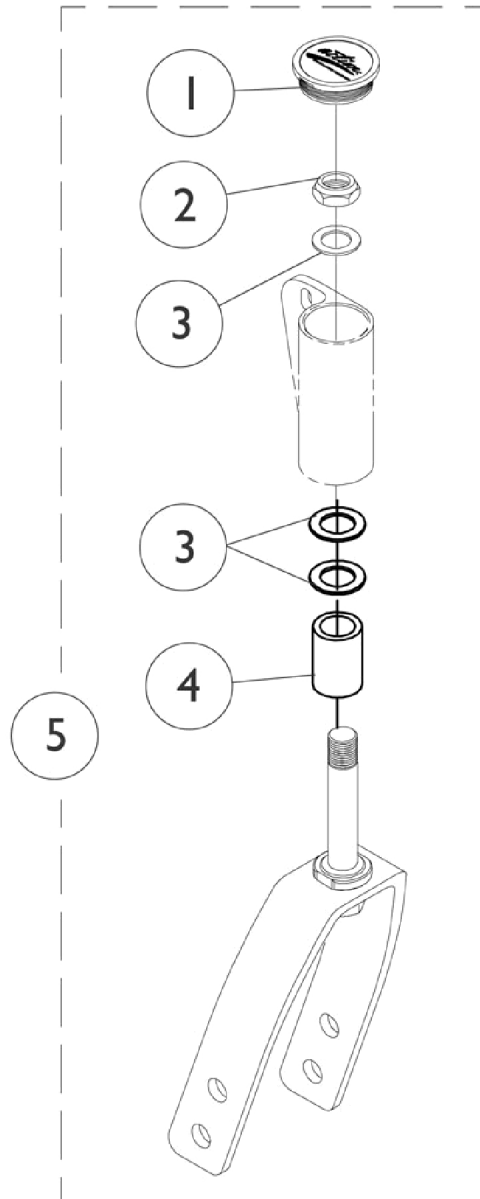

Forks & Hardware (Before 7/1/11)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1038353	Plug Button, Headtube, Black	1	1
2		1025737	Locknut (1/2-20)	1	4
3		1026174	Washer (1/2 x 7/8 x 3/64")	1	1
4		1046495	Spacer (1/2 x 3/4 x 1")	1	2
5		1065623	Kit, Fork w/ Hardware (3" Caster)	1	2
5		1065624	Kit, Fork w/ Hardware, 1" Extension (3 and 5" Caster)	1	2
5		1065626	Kit, Fork w/ Hardware (5" Caster)	1	2
5		1065627	Kit, Fork w/ Hardware, 1" Extension (5 and 6" Caster)	1	2
5		1065629	Kit, Fork w/ Hardware (6 x 2" and 8" Casters)	1	2