


Arms - T Style Adjustable - Myon Active HD (TAHD) (20 - 24" W Heavy Duty Chairs)

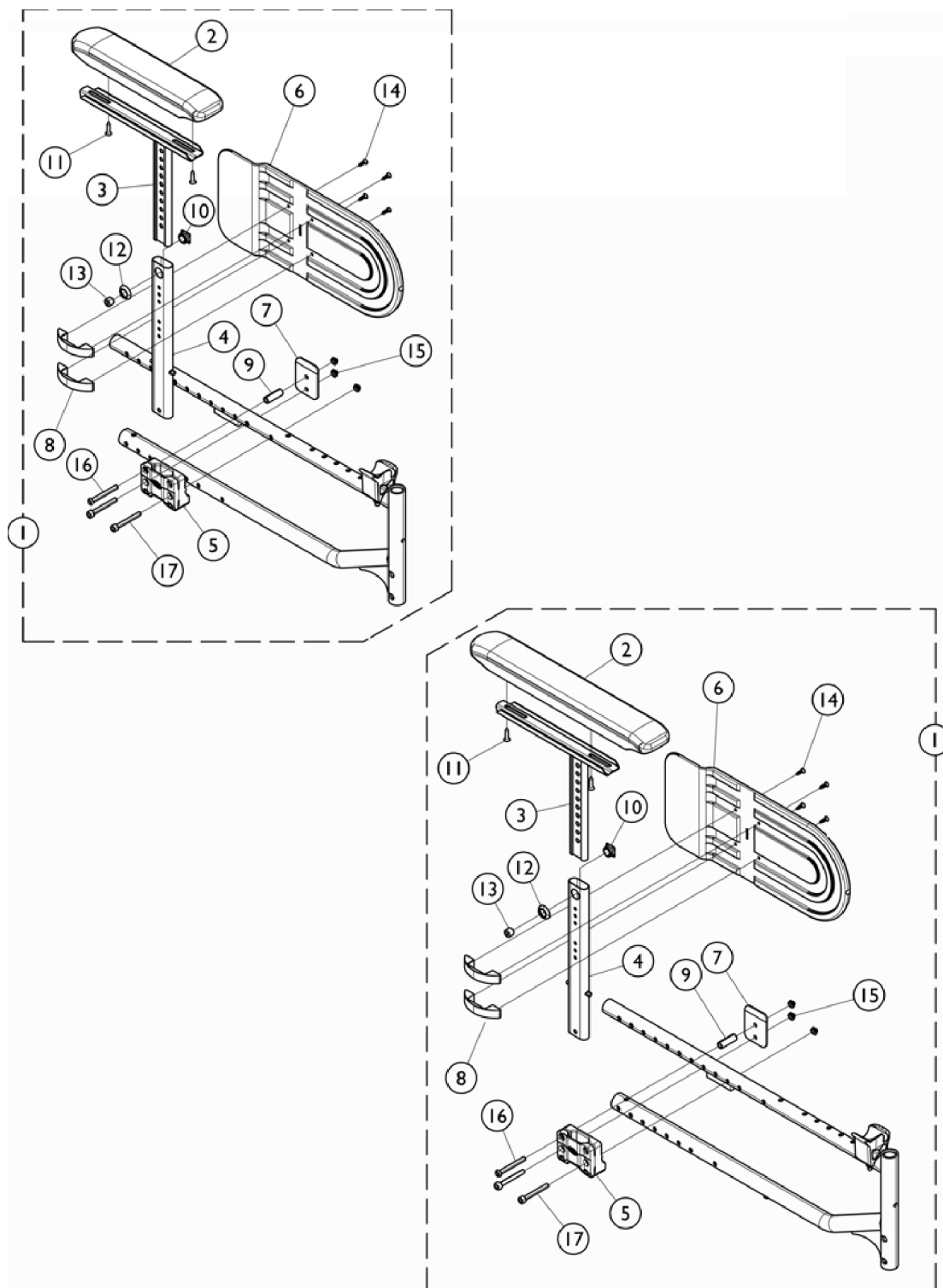

Arms - T Style Adjustable - Myon Active HD (TAHD)

(20 - 24" W Heavy Duty Chairs)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1478770	T-Arm Assembly w/ Hardware, Desk Length, Heavy Duty - Right	1	1
2			Arm Pad, Desk Length	1	1
3			Tube, Armrest - Right	1	1
4			NLA - Support, Armrest	1	1
5			Guard, Side, Clothing	1	1
6			Support, T Arm, Heavy Duty	1	1
7			Spacer, Nylon	1	1
8			Plug Button, Black (3/4")	1	2
9			Screw, Socket, Button Head (M5 x 35mm)	1	2
10			Screw, Phillips Flat Head (#4-40 x 3/8")	1	1
11			Screw (M40 x 14mm)	1	2
12			Nut, Double, Plastic	1	1
13			Screw, Socket Button Head (M6 x 53mm)	1	1
14			Screw, Socket Head (M6 x 50)	1	2
15			NLA - Locknut (M6 x 1)	1	4
1	A	1478771	T-Arm Assembly w/ Hardware, Desk Length, Heavy Duty - Left	1	1
2			Arm Pad, Desk Length	1	1
3			Tube, Armrest, Black - Left	1	1
4			NLA - Support, Armrest	1	1
5			Guard, Side, Clothing	1	1
6			Support, T Arm, Heavy Duty	1	1
7			Spacer, Nylon	1	1
8			Plug Button, Black (3/4")	1	2
9			Screw, Socket, Button Head (M5 x 35mm)	1	2
10			Screw, Phillips Flat Head (#4-40 x 3/8")	1	1
11			Screw (M40 x 14mm)	1	2
12			Nut, Double, Plastic	1	1
13			Screw, Socket Button Head (M6 x 53mm)	1	1
14			Screw, Socket Head (M6 x 50)	1	2
15			NLA - Locknut (M6 x 1)	1	4
NOTE: A - Includes items 2 - 15.					