



# Headwings

THE PRODUCTION OF THE HTR IS CURRENTLY SUSPENDED AND SOME SERVICE PARTS MAY NOT BE AVAILABLE TO ORDER.

Headwing Assembly

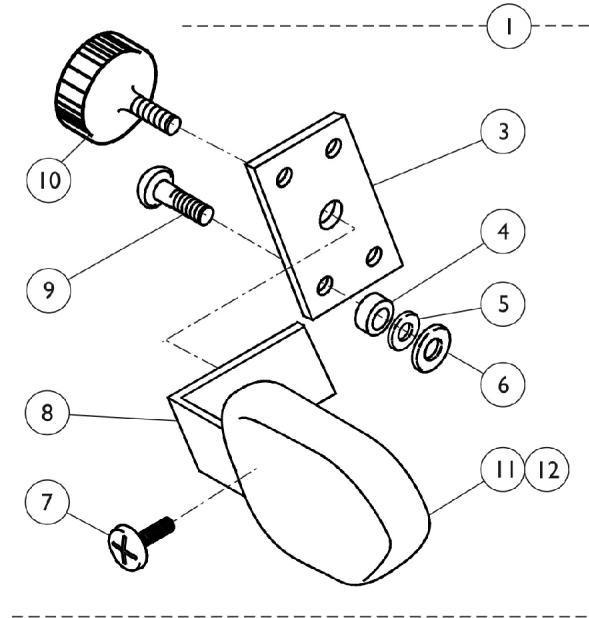

Headwing Hardware

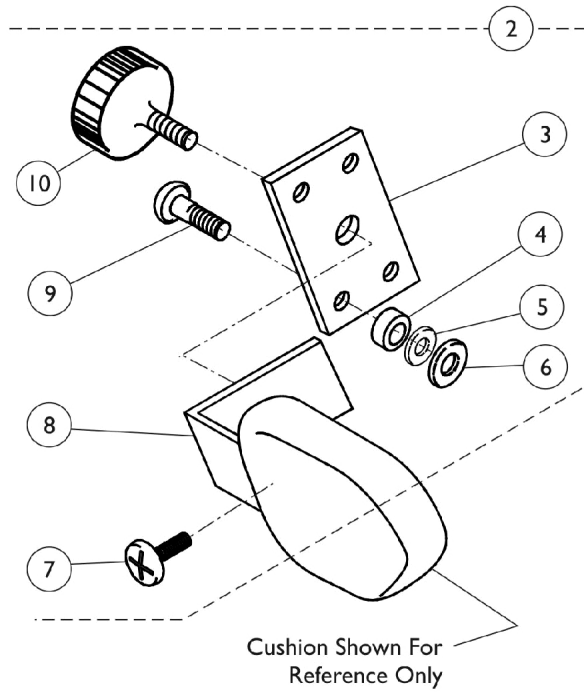

# Headwings

THE PRODUCTION OF THE HTR IS CURRENTLY SUSPENDED AND SOME SERVICE PARTS MAY NOT BE AVAILABLE TO ORDER.

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	D,A	1072294	Headwing Assembly, Mauve, Chrome	2	1
1	D,A	1072301	Headwing Assembly, Grotto, Chrome	2	1
1	D,A	1072302	Headwing Assembly, Blueberry, Chrome	2	1
1	E,A	1129611	Headwing Assembly, Grotto, Grey	2	1
1	E,A	1129612	Headwing Assembly, Blueberry, Grey	2	1
1	E,A	1129628	Headwing Assembly, Mauve, Grey	2	1
2	D,B	HTR5HWK	Kit, Headwing Hardware - Chrome	2	1
2	E,B	1129627	Kit, Headwing Hardware - Grey	2	1
3		QC47A	Plate, Headwing - Chrome	1	1
3		1129545-NLA	Plate, Headwing - Grey	1	1
4		1108143	Spacer, Nylon (3/16 x 5/16 x 5/16")	1	4
5		1126487	Washer, Flat Nylon (3/16 x 7/16 x 1/16")	1	4
6		1013218	Washer (#10 Flat)	1	4
7		5000136	Screw, Phillips Truss Head (1/4-20 x 3/4")	1	2
8	D	TR5300	Bracket, "L", Chrome	1	1
8	E	1129515-NLA	Bracket, "L", Grey	1	1
9	C	1108131-NLA	NLA - Screw, Phillips Pan Head (#10-24 x 1")	1	4
9		1108193-NLA	NLA - Screw, Phillips Pan Head (#10-24 x 3/4")	1	4
10		FB883	Knob, Adjustment	1	1
11		450104-25-NLA	Cushion, Headwing, Mauve	2	1
11		450104-26-NLA	Cushion, Headwing, Grotto	2	1
11		450104-27	Cushion, Headwing, Blueberry	2	1
12		450105-25-NLA	NLA - Cover, Headwing, Mauve	1	1
12		450105-26-NLA	NLA - Cover, Headwing, Grotto	1	1
12		450105-27	Cover, Headwing, Blueberry	1	1

NOTE: A - Includes items 2 - 12.  
B - Includes items 2 - 10.  
C - Items no longer available, refer to Part No. 1108193 when ordering replacements.  
D - Before 01/24/2005, for use on chairs with Chrome frame finish.  
E - After 01/23/2005, for use on chairs with a Gray frame finish.