


Rear Anti-Tipper

THE PRODUCTION OF THE HTR IS CURRENTLY SUSPENDED AND SOME SERVICE PARTS MAY NOT BE AVAILABLE TO ORDER.

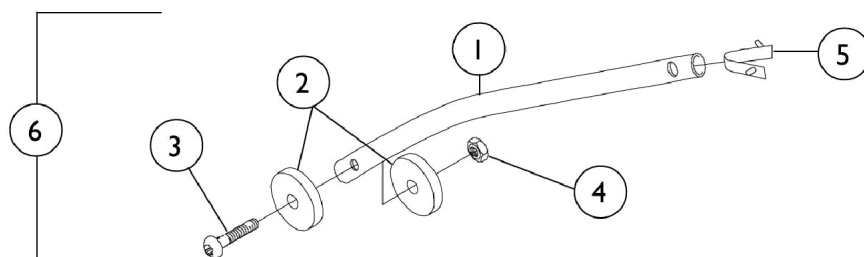

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	E	1088040	Tube, Anti-Tip, Chrome (20" Wheel)	1	1
1	F	1129508-NLA	Tube, Anti-Tip, Grey (20" Wheel)	1	1
1	E	4600581-NLA	NLA - Tube, Anti-Tip, Chrome (12" Wheel)	1	1
1	F	1129513-NLA	Tube, Anti-Tip, Grey (12 - 24" Wheel)	1	1
2		361552-NLA	NLA - Wheel, Anti-Tipper Roller	1	2
3		1108128	Screw, Socket Button Head (1/4-20 x 1-1/2")	1	1
4		60275X000	Locknut (1/4-20)	1	1
5		1108118-NLA	NLA - Button, Snap, Double Ended	1	1
6	E,A,B	440565	Anti-Tipper, Chrome (24" Wheel) (Pair)	2	1
6	F,A,B	1129598-NLA	Anti-Tipper, Grey (24" Wheel) (Pair)	2	1
6	C,E,A	1088041	Anit-Tipper, Chrome (20" Wheel) (Pair)	2	1
6	C,F,A	1129509-NLA	Anti-Tipper, Grey (20" Wheel) (Pair)	2	1
6	D,E,A	440573-NLA	NLA - Anti-Tipper, Chrome (12" Wheel) (Pair)	2	1
6	D,F,A	1131377-NLA	NLA - Anti-Tipper, Grey (12" Wheel) (Pair)	2	1

NOTE: A - Includes Items 1-5.
 B - 24" Rear Wheel.
 C - 20" Rear Wheel.
 D - 12" Rear Wheel.
 E - Before 1/24/05. For use on chairs with "Chrome Frame Finish."
 F - After 1/23/05. For use on chairs with "Grey Frame Finish."