


# Recliner - Low Shear - Retro-Fit Kit w/ Upholstery

The Recliner Retro-Fit Kit can be used on the Solara® 2G as long as the seat depth is not set at its deepest settings (Small Frame: 16" D, Medium Frame: 19" D, Large Frame: 22" D)

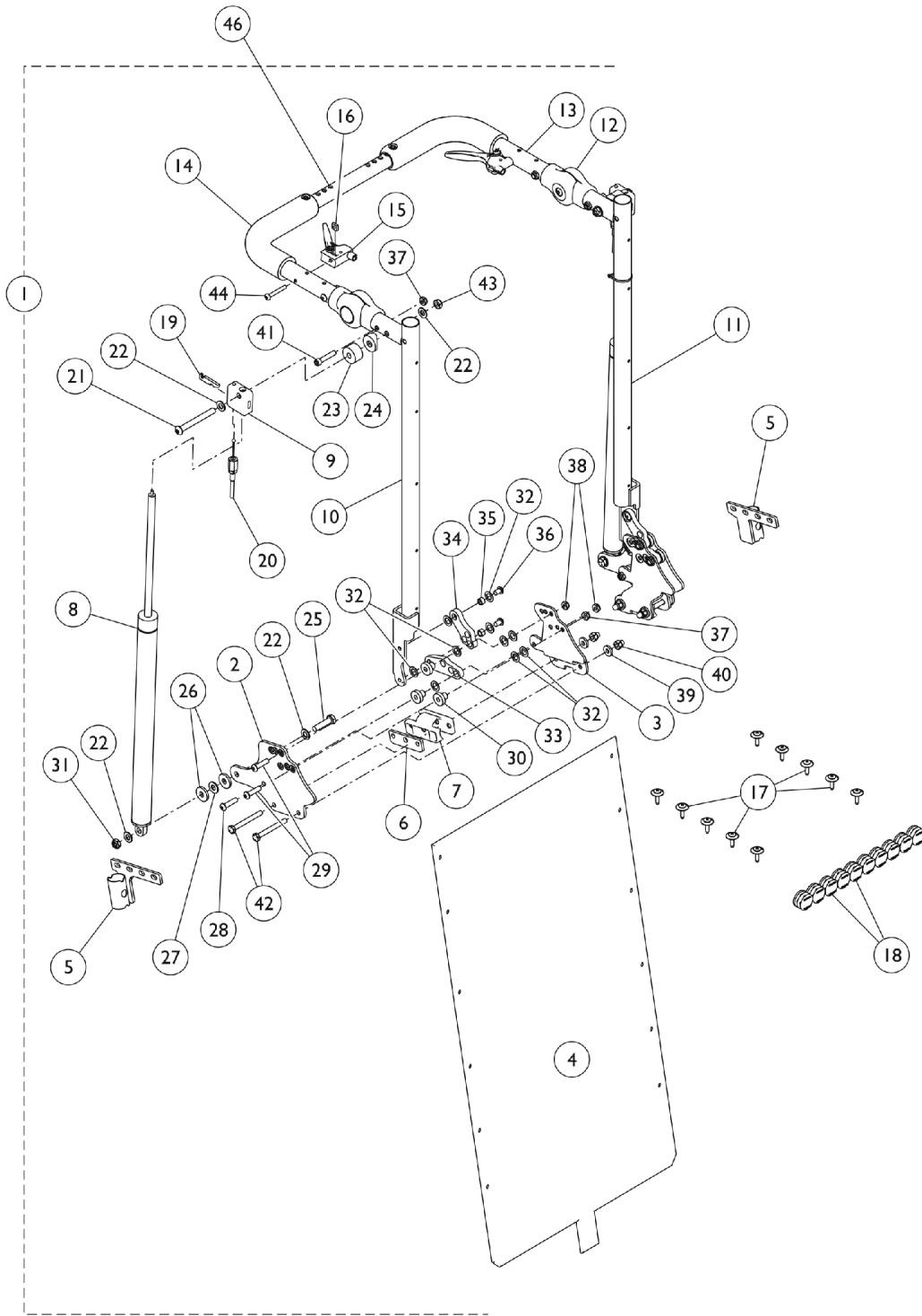

# Recliner - Low Shear - Retro-Fit Kit w/ Upholstery

The Recliner Retro-Fit Kit can be used on the Solara® 2G as long as the seat depth is not set at its deepest settings (Small Frame: 16" D, Medium Frame: 19" D, Large Frame: 22" D)

| Item Number | Note | Part Number | Description   | Quantity Per |          |
|-------------|------|-------------|---|--------------|----------|
|             |      |             |   | Package      | Assembly |
| 1           | A    | See Chart   | #1 Retro-Fit Kit, Low Shear w/ Upholstery (14-20" Wide) | 1            | 1        |
| 12          | B    | 1134190     | Kit, Stroller Handle, Adjustable Angle - Outside Mount  | 1            | 1        |
|             |      |             | Screw, Socket Button Head (1/4-28 x 1-1/4")             | 1            | 2        |
|             |      |             | Gear, Stroller Handle                                   | 1            | 2        |
|             |      |             | Screw, Socket Head (1/4-28 x 1-1/4")                    | 1            | 2        |
|             |      |             | Screw, Socket Head (1/4-28 x 1-1/4")                    | 1            | 2        |
|             |      |             | Locknut (1/4-28)  | 1            | 2        |
|             |      |             | Locknut (1/4-28)  | 1            | 4        |
| 46          |      | See Chart   | #46 Crossbrace Tube                                     | 1            | 1        |

NOTE: A - Kit includes items 2 - 11, and 13 - 44.  
B - Includes Gear, Screws, and Nuts.

## #1 Retro-Fit Kit, Low Shear w/ Upholstery (14-20" Wide)

| Chair Width | Kit Part Number          |
|-------------|--------------------------|
| 14" W       | 1171137<br>See Note(s) A |
| 15" W       | 1171138<br>See Note(s) A |
| 16" W       | 1171139<br>See Note(s) A |
| 17" W       | 1171140<br>See Note(s) A |
| 18" W       | 1171141<br>See Note(s) A |
| 19" W       | 1171142<br>See Note(s) A |
| 20" W       | 1171143<br>See Note(s) A |

NOTE: A - Includes items 2 - 11, and 13 - 44.

## #46 Crossbrace Tube

| 12-16" Wide Tube | 14-18" Wide Tube | 16-20" Wide Tube | 18-22" Wide Tube | 20-24" Wide Tube |
|------------------|------------------|------------------|------------------|------------------|
| 1129473          | 1129474          | 1129475          | 1129476          | 1159455          |