


Faceplate Components

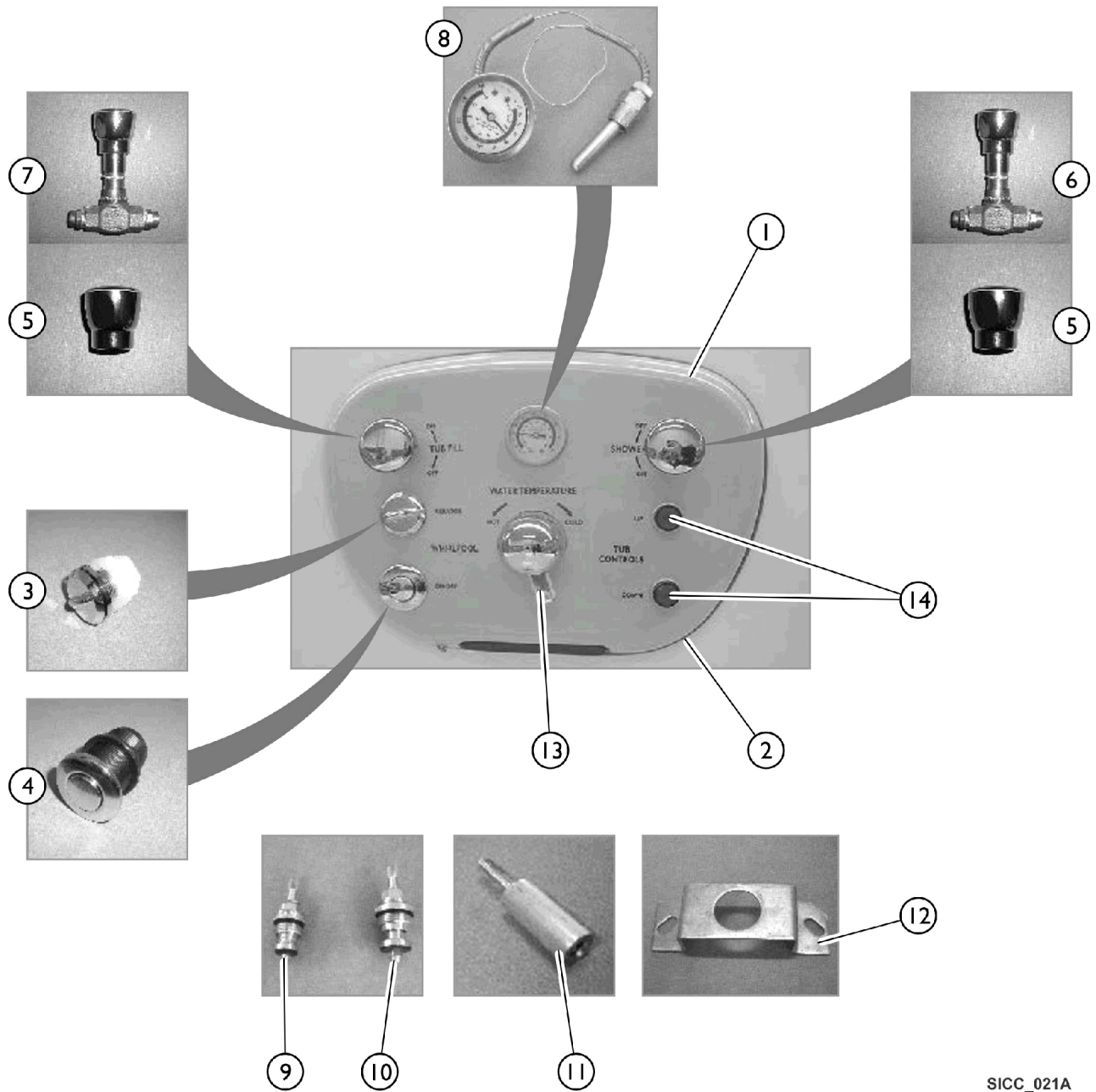

Faceplate Components

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1090096-NLA	NLA - Faceplate, Tub Controls	1	1
1		1093346-NLA	NLA - Faceplate, Fixed Height, Tub Controls	1	1
2		1095070-NLA	NLA - Faceplate Trim, Grey	1	1
3	A	PAR5590A-NLA	NLA - Air Vent Assembly, White	1	1
4		PAR5510C-NLA	NLA - On/Off Button	1	1
5		PAR6652A-NLA	NLA - Handle, Lynwood Plastic (Metal no longer available)	1	1
6		IHPAR6652-NLA	NLA - Valve, Shower (1/2")	1	1
7		PAR6653	Valve, Fill, Tub (3/4")	1	1
8		PAR5710E	Gauge, Temperature	1	1
9		PAR6652B-NLA	NLA - Stem and Cartridge, Shower (1/2")	1	1
10		PAR6653B	Stem and Cartridge, Tub Fill (3/4")	1	1
11		1093365-NLA	NLA - Stud, Mounting, Faceplate	1	1
12		1093352-NLA	NLA - Bracket, Mounting, Mixing Valve	1	1
13		PAR6670CG	NLA - Valve, Complete RADA Mixing	1	1
14		1151444-NLA	NLA - Switch Assembly	1	2

NOTE: A - White Air Vent Assembly replaced Chrome. Functionality is the same.