
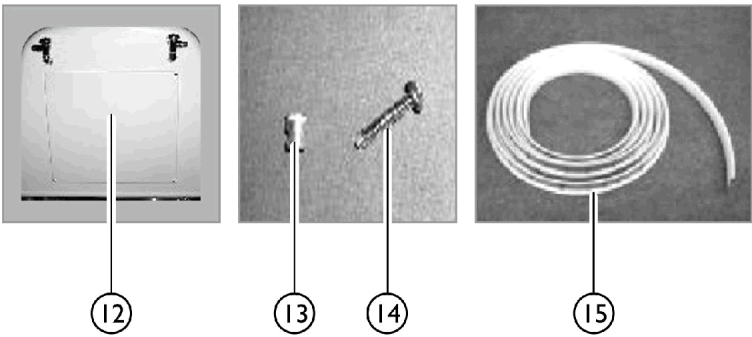
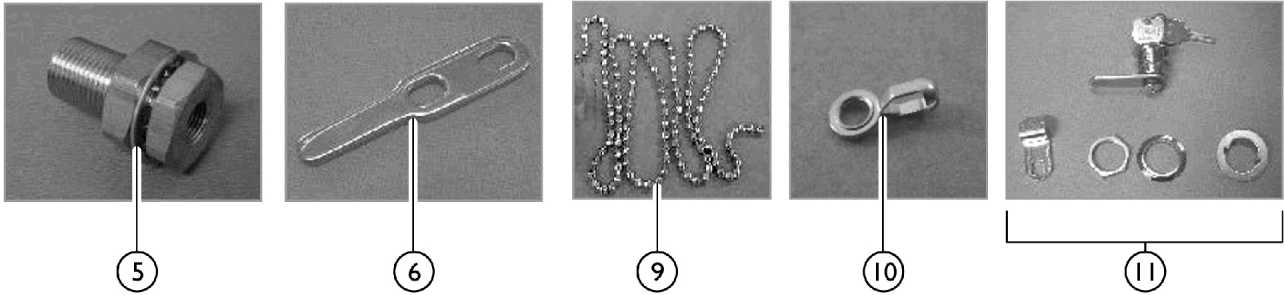
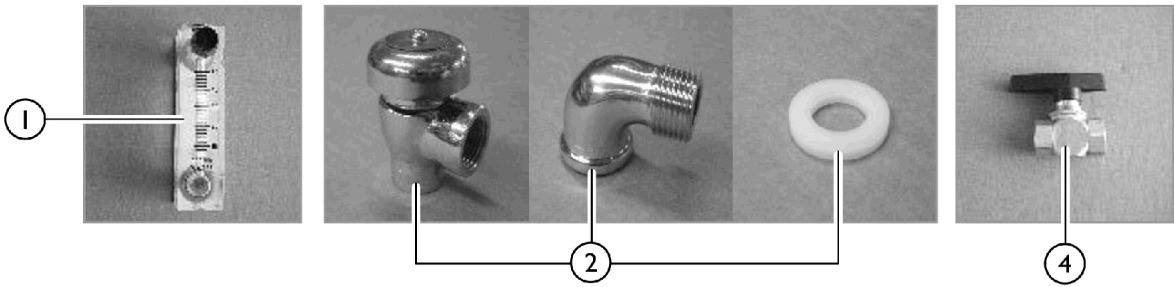


Disinfectant Compartment, Access Panel, and Trim

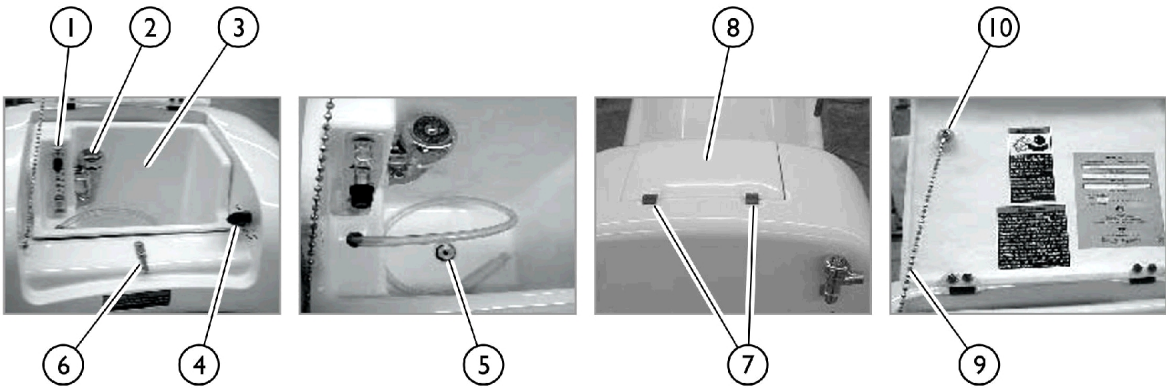

Disinfectant Compartment, Access Panel, and Trim

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		PAR1108	Metering Valve	1	1
2		PAR7380AG	Back Flow Preventer - Conbraco (1/2") (One Screw in Cap)	1	1
2		IHPAR7383C	Bonnet	1	1
2		PAR1136	Elbow, 90 Degrees - Chrome	1	1
2		PAR6160D	Washer, Nylon, Backflow Preventer and Chrome Elbow	1	1
		IHPAR7384C	Kit, Repair	1	1
3		1090097-NLA	NLA - Storage Compartment	1	1
4		PAR1110C	Valve, Two Way Ball (Disinfectant on/off)	1	1
5		1094447-NLA	NLA - Fitting, Bulkhead	1	1
6		1096930-NLA	NLA - Plate, Lock	1	1
7		PAR5410	Hinge, Access Door	1	1
8		1090098-NLA	NLA - Lid, Storage	1	1
9		PAR9818-NLA	NLA - Chain, Beaded (1")	1	1
10		PAR9821	Chain Clip, Drain Plug	1	1
11		1096981-NLA	NLA - Lock w/ Keys	1	1
12		1096986-NLA	NLA - Panel, Back Access	1	1
13		1097286	Threaded Insert (#10-24)	1	1
14		IHPAR9931	Screw (#10-24 x 1)	1	1
15	A	1143642	Trim Kit	1	1

NOTE: A - Twenty One feet required.