
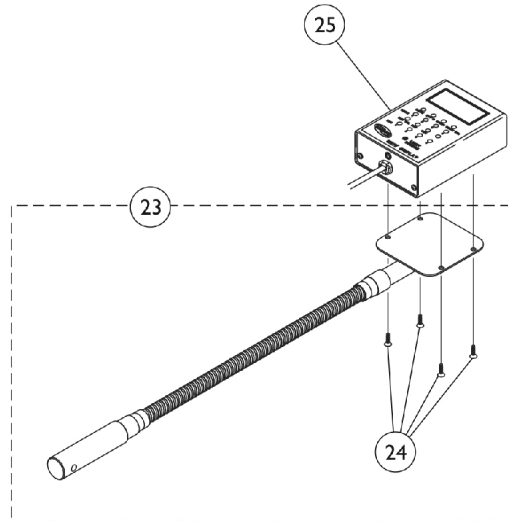
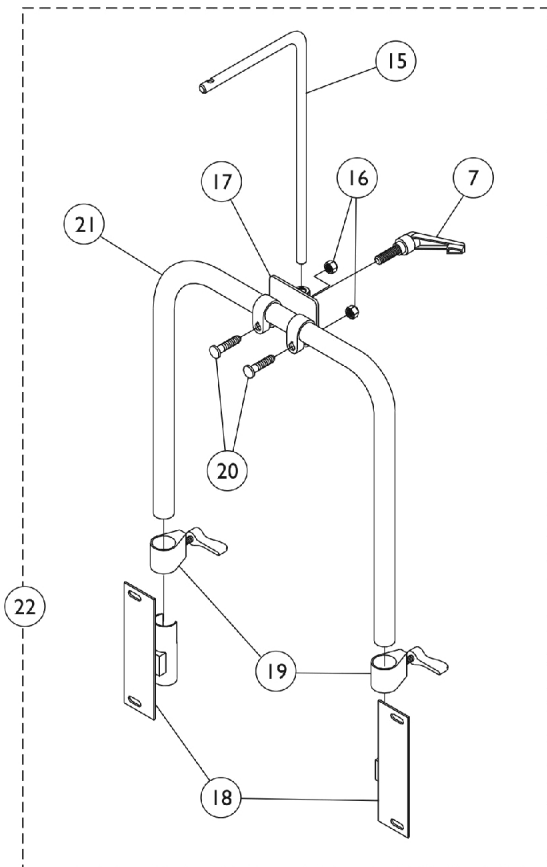


MKIV R.I.M. Head Control Assembly (1500) (Before 8/16/95)

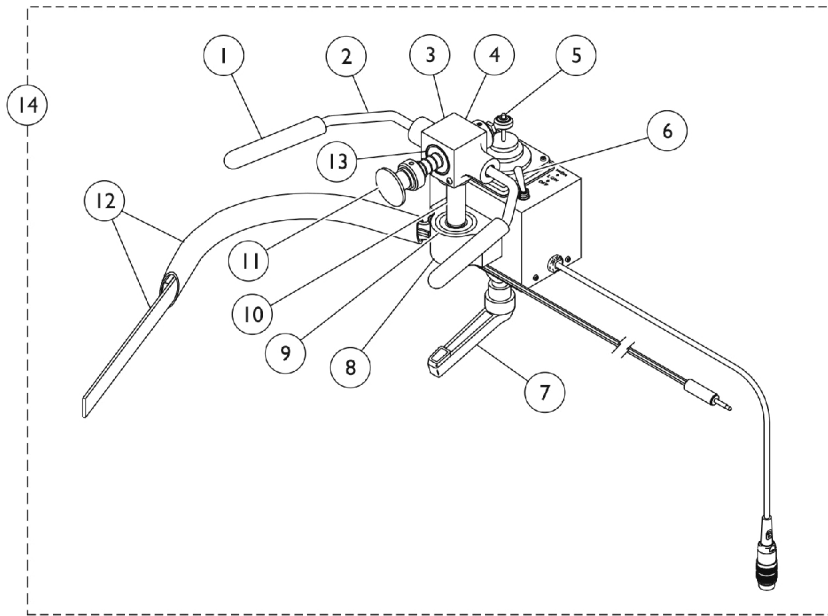

MKIV R.I.M. Head Control Assembly (1500) (Before 8/16/95)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1025258-NLA	NLA - Rod, Cushion	1	1
2		1025263-NLA	NLA - Bar, Control	1	1
3		1025241-NLA	NLA - Block, Bearing	1	1
4	E	1010042-NLA	NLA - Switch, Toggle SPDT	1	1
5		1001625	Tie Rod End	1	1
6		1028914-NLA	NLA - Switch, Hi / Lo / Off (Black)	1	1
7		1022420-NLA	NLA - Handle, Rim Control	1	1
8		40314X003-NLA	NLA - Clip, Spring, Rotor	1	1
9		1025269-NLA	NLA - Bearing, Pillow Block	1	1
10		1025238-NLA	NLA - Rod, Pivot	1	1
11	A	1025270-NLA	NLA - Push Rod Assembly	1	1
11		1016428-NLA	NLA - Guard, Axle	1	1
12		1025271-NLA	NLA - Ribbon Reset Switch and Bracket Assembly	1	1
13		1018113	Bearing, Linear	1	1
14		1038633-NLA	NLA - Joystick, Remote	1	1
15		1025248-NLA	NLA - Rod, Joystick Mounting	1	1
16		60264X001	Locknut (5/16-24)	1	2
17		1025250-NLA	NLA - Bracket, Rod Mounting, Black	1	1
18		1025245-NLA	NLA - Bracket, Upright, Black -Right	1	1
18		1025246	Bracket, Upright, Black - Left	1	1
19		1027274	Adjustment Lock Assemnbly, Black	1	2
20		11232X009	Bolt, Lug (5/16-24 x 1-3/8")	1	2
21	B	1025277-NLA	NLA - Tube, Support Upright	1	1
21	C	1025276-NLA	NLA - Tube, Support, Upright, Black	1	1
21	D	1025278-NLA	NLA - Tube, Support, Upright, Black	1	1
22	B	1025281-NLA	NLA - Hardware Assembly, Rim Control	1	1
22	C	1025265-NLA	NLA - Hardware Assembly, Rim Control	1	1
22	D	1025266-NLA	NLA - Hardware Assembly, Rim Control	1	1
23		1049594-NLA	NLA - Tube, Flex with Screws	1	1
24		1052041	Screw, Phillips Pan Head (#4-40 x 3/8")	1	4
25		1108490-NLA	NLA - Display without Switches	1	1

NOTE: A - Includes: push rod cap, collar with set screw, spring push rod, nut, and tie rod end.
B - 12" Wide Chairs
C - 14" Wide Chairs
D - 16" Wide Chairs
E - Black Switch is no longer available. The Chrome Switch part number 1010042 serves as a direct replacement