


# Wheel Lock Assembly

(TDX SI & TDX SI-2 Before 9/15/11) & TDX SI-HD

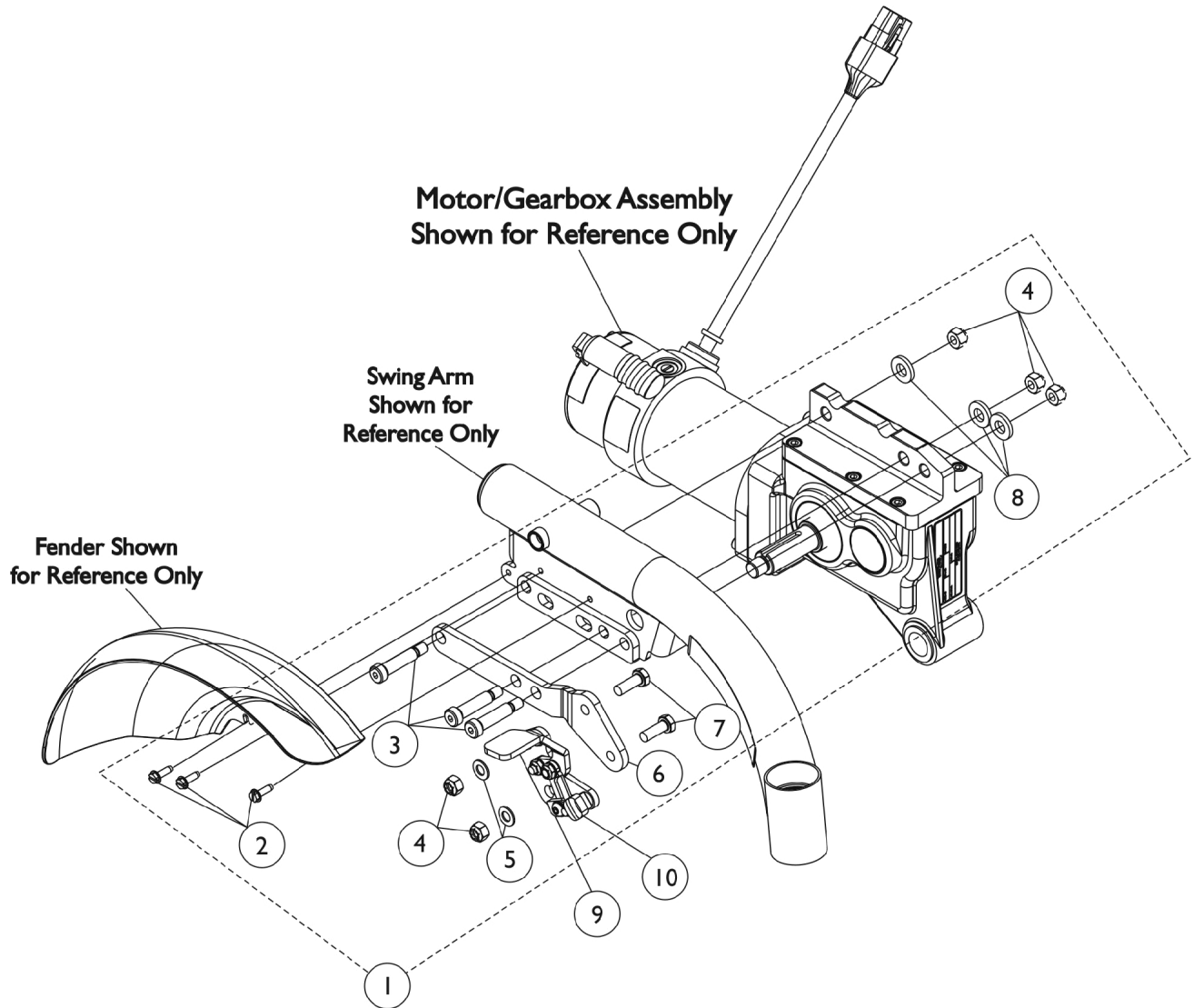

## Wheel Lock Assembly (TDX SI & TDX SI-2 Before 9/15/11) & TDX SI-HD

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1145072-NLA	NLA - Kit, Wheel Lock Mounting Hardware - Right	1	1
2			Screw, Slotted Hex Washer Head (#10-32 x 5/8")	1	3
3			Screw, Shoulder Hex Socket (3/8 x 1-1/4", 5/16-18)	1	3
4			Locknut (5/16-18)	1	5
5			Washer, Flat (5/16 x 5/8 x 1/16")	1	2
6			Bracket, Wheel Lock, Black - Right	1	1
7			NLA - Screw, Hex Head w/ Patch (5/16-18 x 7/8")	1	2
8			NLA - Washer, Steel (7/16 x 13/16 x 1/8")	1	3
1	A	1145073-NLA	NLA - Kit, Wheel Lock Mounting Hardware - Left	1	1
2			Screw, Slotted Hex Washer Head (#10-32 x 5/8")	1	3
3			Screw, Shoulder Hex Socket (3/8 x 1-1/4", 5/16-18)	1	3
4			Locknut (5/16-18)	1	5
5			Washer, Flat (5/16 x 5/8 x 1/16")	1	2
6			Bracket, Wheel Lock, Black - Left	1	1
7			NLA - Screw, Hex Head w/ Patch (5/16-18 x 7/8")	1	2
8			NLA - Washer, Steel (7/16 x 13/16 x 1/8")	1	3
9		1139885	Wheel Lock Assembly - Right	1	1
9		1139886	Wheel Lock Assembly - Left	1	1

NOTE: A - Includes items 2-8.