


Matrix PCS Back Wire Tray

(Matrix Brand name replaced InTouch as of 6/30/2011)

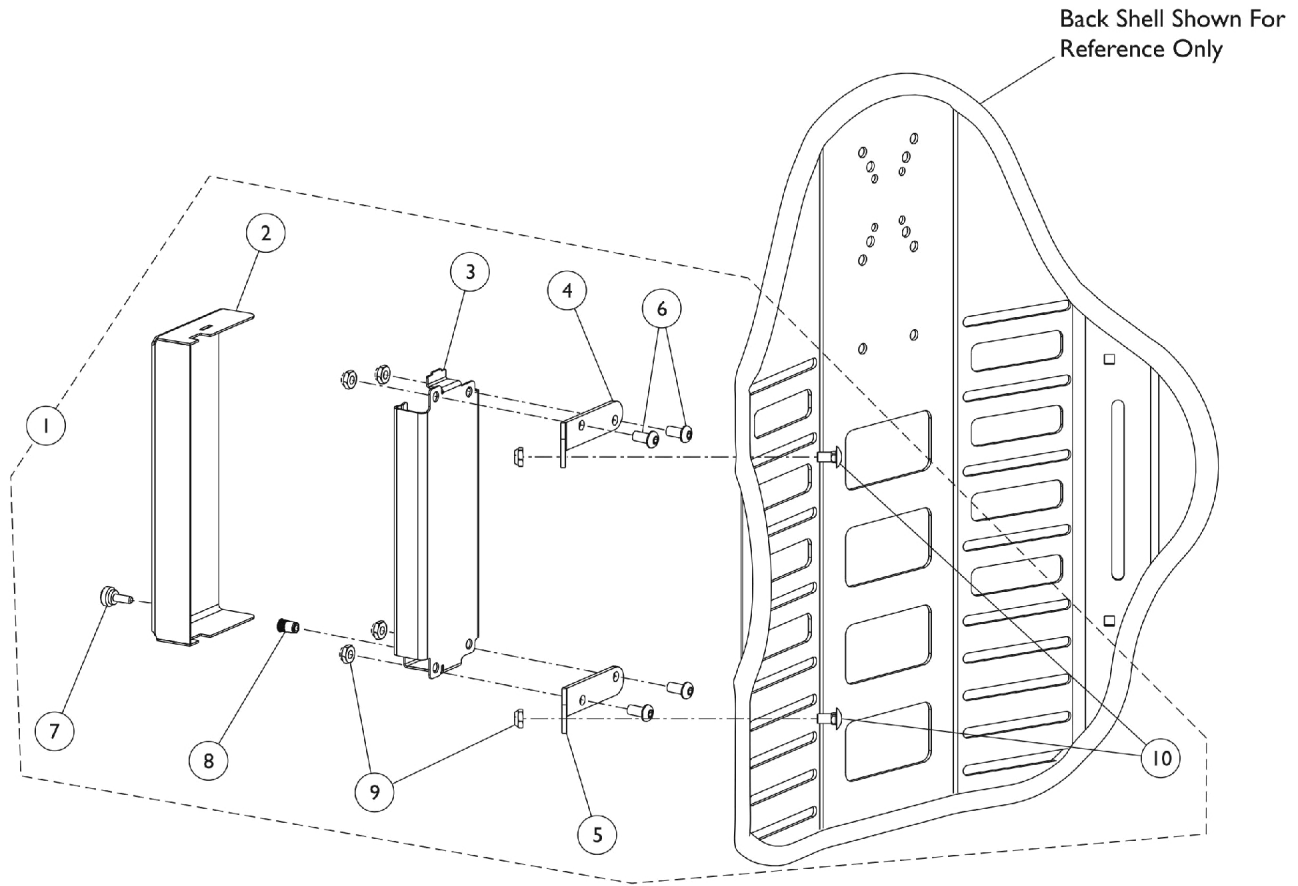

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1164441	Kit, Wire Tray Mounting Hardware	1	1
2			Cover, MK6i Connector Bracket	1	1
3			Bracket, MK6i Connector, Black	1	1
4			Bracket, Wire Tray, Upper	1	1
5			Bracket, Wire Tray, Lower	1	1
6			NLA - Screw, Socket Button Head (1/4-20 x 1/2")	1	4
7			Screw, Thumb (#10-32 x 1/2")	1	1
8			Insert, Threaded (#10-32)	1	1
9			Locknut (1/4-20)	1	6
10			Bolt, Round Head, Square Neck (1/4-20 x 1/2")	1	2
			Carton, Shipper	1	1

NOTE: A - For use with MK6i Electronics only. Carton not shown.