

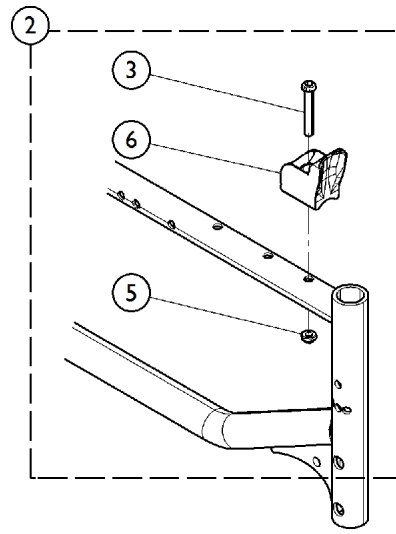


# Frame with Hardware

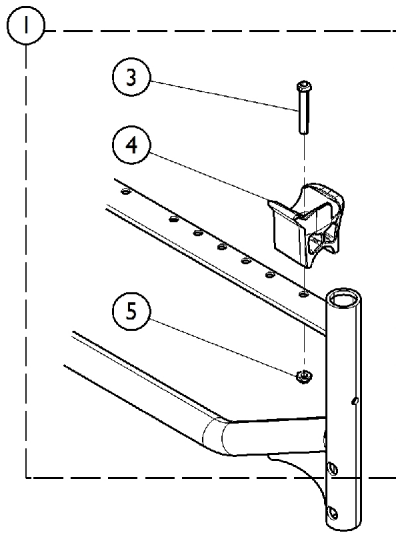

Frames, Back Cans  
and Forks Shown for  
Reference Only

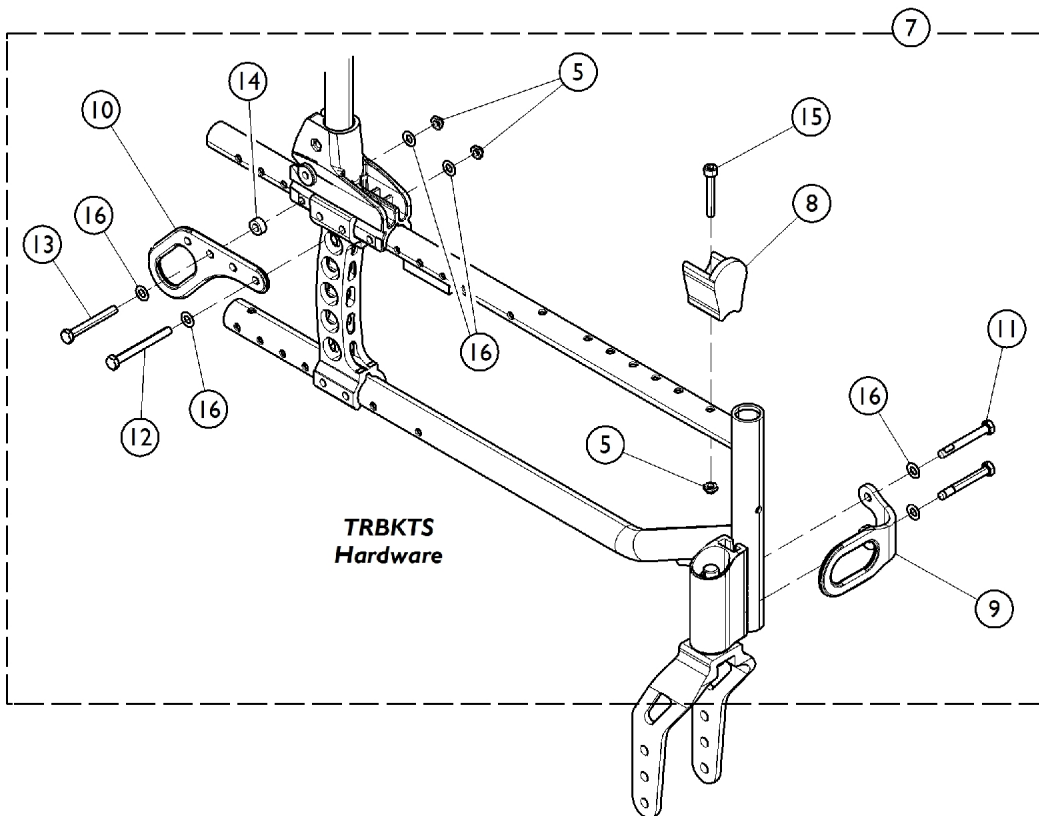

## Frame with Hardware

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1172219	Kit, H-Block Hardware	1	1
3			NLA - Screw, Socket Button Head (M6 x 40mm)	1	2
4			H-Block, Front - Left	1	1
4			H-Block, Front - Right	1	1
5			NLA - Locknut (M6 x 1)	1	2
2	B	1172246	Kit, H-Block Hardware, Junior	1	1
			Locator, Seat	1	2
			NLA - Screw, Socket Button Head (M6 x 40mm)	1	2
			NLA - Locknut (M6 x 1)	1	2
2	B	1172248	Kit, H-Block Hardware, Junior (12 - 15" W)	1	1
3			Screw, Socket Head, Cap (M6 x 35mm)	1	2
5			NLA - Locknut (M6 x 1)	1	2
6			H-Block, Front - Right	1	1
6			H-Block, Front - Left	1	1
7	C	1172217	Kit, TRBKTS Mounting Hardware	1	1
8			H-Block, Front - Right	1	1
8			H-Block, Front - Left	1	1
9			Bracket, Tie Down, Front	1	2
10			Plate, Tie Down, Rear	1	2
11			nla - Screw, Hex Head w/ Patch (M6 x 1 x 50mm)	1	4
12			Screw, Hex Head Cap (M6 x 1 x 60)	1	2
13			Screw, Hex Head Cap (M6 x 1 x 50)	1	2
14			NLA - Spacer	1	2
15			NLA - Screw, Socket Head Cap (M6 x 1 x 40mm)	1	2
16			Washer (1/4 x 1/2 x 1/32")	1	12
		See Chart	Frame Assembly w/ Hardware, Standard	1	1
		See Chart	Frame Assembly w/ Hardware, Heavy Duty	1	1
		See Chart	Frame Assembly w/ Hardware - TRRO/TRBKTS	1	1

NOTE: A - Kit includes items 3, 4, and 5.  
 B - Kit includes items 3, 5, and 6.  
 C - Kit includes items 8 - 16.

## Frame Assembly w/ Hardware, Standard

Frame Style	Seat Depth	Part Number	Hardware Part No.
MYONADLT MYONTO MYONTS	15"	8881172238	1172219
MYONADLT MYONTO MYONTS	16 - 18"	8881172244	1172219
MYONADLT MYONTO MYONTS	19 - 20"	8881172239	1172219

Specify Frame Color when ordering, see "Frame Finishes" Category for Color choices.

# Frame with Hardware

## Frame Assembly w/ Hardware, Heavy Duty

Frame Style	Seat Depth	Part Number	Hardware Part No.
MYONAC - HDPG	15 - 16"	8881172241	1172219
MYONAC - HDPG	17 - 18"	8881172242	1172219
MYONAC - HDPG	19 - 20"	8881172243	1172219

Specify Frame Color when ordering, see "Frame Finishes" Category for Color choices.

# Frame with Hardware

## Frame Assembly w/ Hardware - TRRO/TRBKTS

Frame Style	Seat Depth	Part Number	Hardware Part No.
MYONADLT MYONTO	15"	8881172238	1172216
MYONADLT MYONTO	16 - 18"	8881172244	1172216
MYONADLT MYONTO	19 - 20"	8881172239	1172216
MYONADLT MYONTS	15"	8881172238	1172217
MYONADLT MYONTS	16 - 18"	8881172244	1172217
MYONADLT MYONTS	19 - 20"	8881172239	1172217

Specify Frame Color when ordering, see "Frame Finishes" Category for Color choices.