
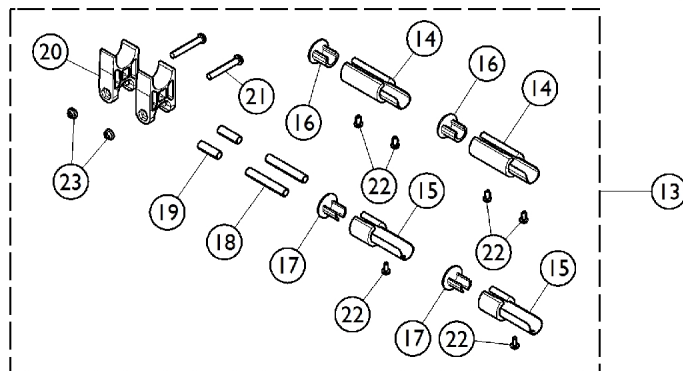
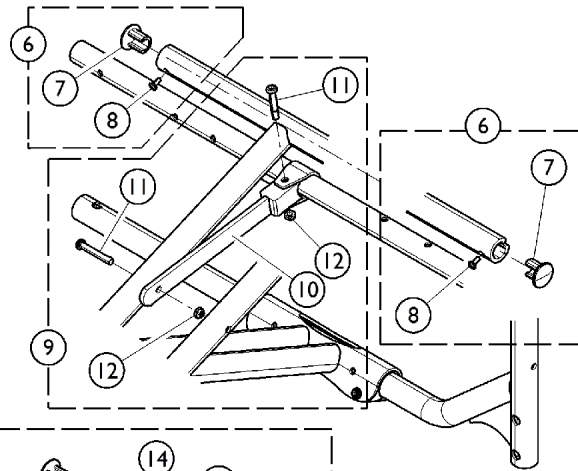
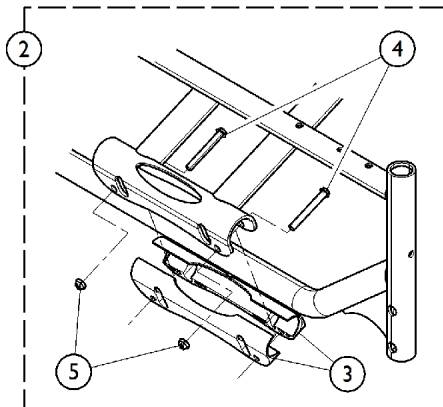


Crossbrace - Conventional Adjustable Depth, Heavy Duty (HDPG ADJDPTH)

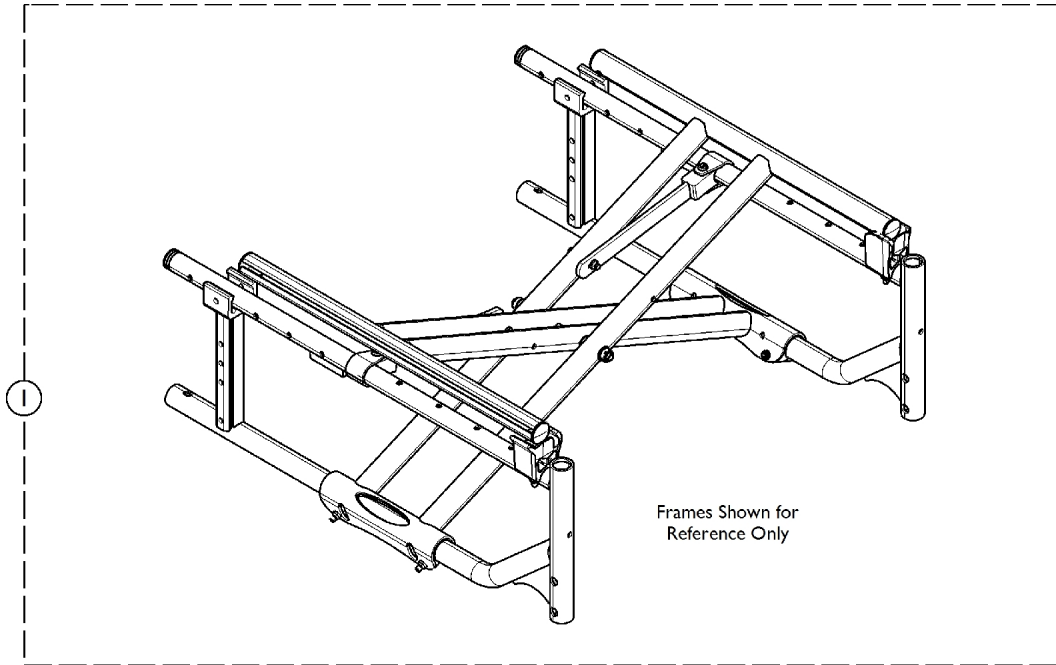

Crossbrace - Conventional Adjustable Depth, Heavy Duty (HDPG ADJDPTH)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	See Chart	#1 Crossbrace Assembly - Myon Junior	1	1
2	B	1173766	Kit, Saddle Clamp w/ Hardware (Pair)	1	1
3			Clamp, Saddle, Crossbar	1	2
4			Screw (M6 x 45mm)	1	4
5			NLA - Locknut (M6 x 1)	1	4
6	C	1173774-NLA	NLA - Kit, Plug Buttons w/ Hardware	1	1
7			NLA - Plug Button	1	4
8			NLA - Screw, Phillips Pan Head (M3.5 x .6 x 9.5mm)	1	4
9	D	1173768	Kit, Pivot Link w/ Hardware (18 - 20" W) (Pair)	1	1
10			Link, Pivot, Crossbrace (18-20" W)	1	2
11			Screw, Socket Head, Cap (M6 x 35mm)	1	2
12			NLA - Locknut (M6 x 1)	1	2
			Carton	1	1
			NLA - Bag, Poly Zip-Lip (8-3/4" L x 6" W)	1	1
9	D	1173769	Kit, Pivot Link w/ Hardware (22" W)	1	1
10			Link, Pivot, Crossbrace (22" W)	1	2
11			NLA - Screw, Socket Button Head (M6 x 40mm)	1	2
12			NLA - Locknut (M6 x 1)	1	2
9	D	1173770	Kit, Pivot Link w/ Hardware (24" W)	1	1
10			Link, Pivot, Crossbrace (24" W)	1	2
11			NLA - Screw, Socket Button Head (M6 x 40mm)	1	2
12			NLA - Locknut (M6 x 1)	1	2
13	E	1172274	Kit, Seat Rail Extensions (16 - 18" D)	1	1
14			Extension, Seat Rail (2")	1	2
15			Extension, Seat Rail (1")	1	2
16			NLA - Plug Button	1	2
17			Cap, Extension, Seat Rail (1")	1	2
18			Rod, Upholstery (2")	1	2
19			Rod, Upholstery (1")	1	2
20			Guide, Seat	1	2
21			NLA - Screw, Socket Button Head (M6 x 40mm)	1	2
22			NLA - Screw, Phillips Pan Head (M3.5 x .6 x 9.5mm)	1	6
23			NLA - Locknut (M6 x 1)	1	2
			Instruction Sheet, Seat Rail Extensions	1	1

NOTE: A - Includes items 2 - 12.
 B - Includes items 3 - 5.
 C - Includes items 7 and 8.
 D - Includes items 10 - 12.
 E - Includes items 14 - 23.

#1 Crossbrace Assembly - Adjustable Depth HD

Seat Depth	20" Wide (D20)	22" Wide (D22)	24" Wide (D24)
------------	-------------------	-------------------	-------------------

Crossbrace - Conventional Adjustable Depth, Heavy Duty (HDPG ADJDPTH)

#1 Crossbrace Assembly - Adjustable Depth HD

Seat Depth	20" Wide (D20)	22" Wide (D22)	24" Wide (D24)
16 - 18" Deep (SDR1618)	1488619	1488621	1488622
18 - 20" Deep (SDR1820)	N/A	1169113	1169114