



Junior Elevating Legrest Hangers and Calf Pad (ELVOFF, ELVOFFWO)

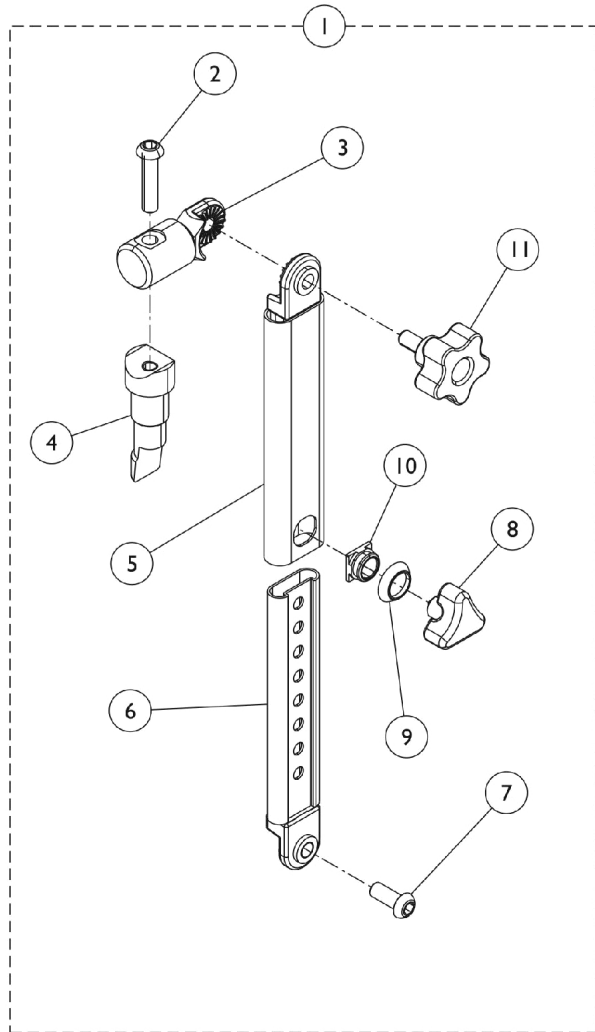

Elevating Legrest Hanger

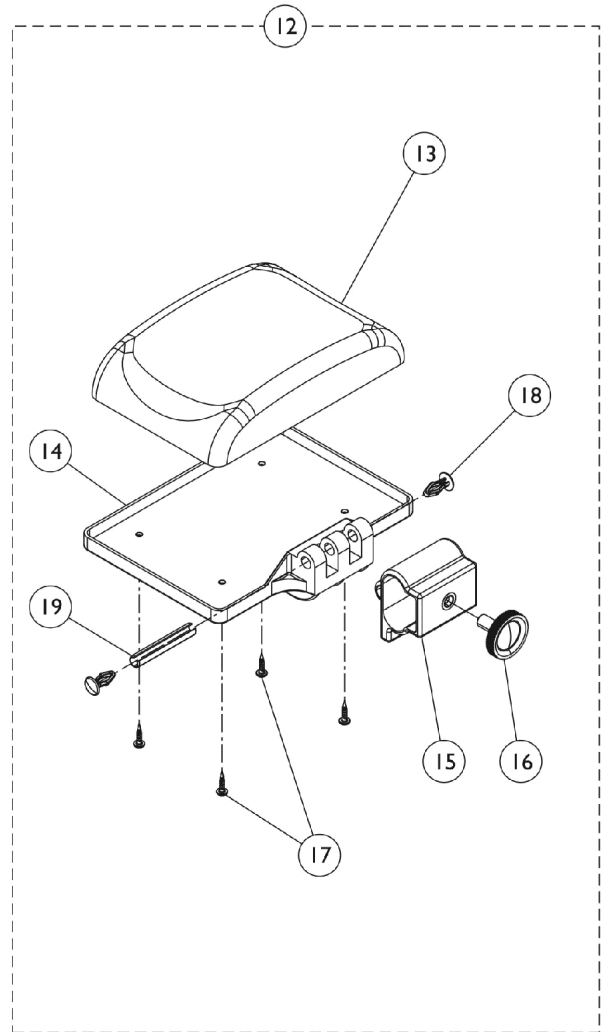

Elevating Legrest Calfpad

Junior Elevating Legrest Hangers and Calf Pad (ELVOFF, ELVOFFWO)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1450950	Hanger Assembly, Elevating Legrest (12" W)	1	2
2			Screw (M8 x 35mm)	1	1
3			Support, Hanger, Elevated Legrest	1	1
4			Indexer, Hanger, 90 Degree - Myon Junior	1	1
5			Tube, Upper, Elevating Legrest (12" W)	1	1
6			Tube, Lower, Elevating Legrest (12" W)	1	1
7			Screw, CBHC (M8 x 20mm)	1	1
8			Screw, Ball (7.5 mm)	1	1
9			Bushing, Armrest	1	1
10			Nut, Square, Armrest	1	1
1		1428808	Hanger Assembly, Elevating Leg Rest (13 - 15" W)	1	2
2			Screw (M8 x 35mm)	1	1
3			Support, Hanger, Elevated Legrest	1	1
4			Indexer, Hanger, 90 Degree - Myon Junior	1	1
5			Tube, Upper, Elevating Legrest (13 - 14" W)	1	1
6			Tube, Lower, Elevating Legrest (13 - 14" W)	1	1
7			Screw, CBHC (M8 x 20mm)	1	1
8			Screw, Ball (7.5 mm)	1	1
9			Bushing, Armrest	1	1
10			Nut, Square, Armrest	1	1
12		1403981	Calf Pad Assembly w/ Hardware (12 - 15" W)	1	1
13			Calf Pad (15 - 17" W)	1	1
14			Plate, Attaching, Calf Pad (15 - 17" W)	1	1
15			Mounting, Calf Pad Back	1	1
16			Button, Tightening	1	1
17			Screw, VBA (M3.5 x 15)	1	4
18			Clamp, Calf Pad Plate	1	2
19			Pin, Spring (M8 x 50mm)	1	1