


# Wheel Locks - Push to Lock

Push to Lock (WLPUSH), Push to Lock Extended (WLPUSHX)

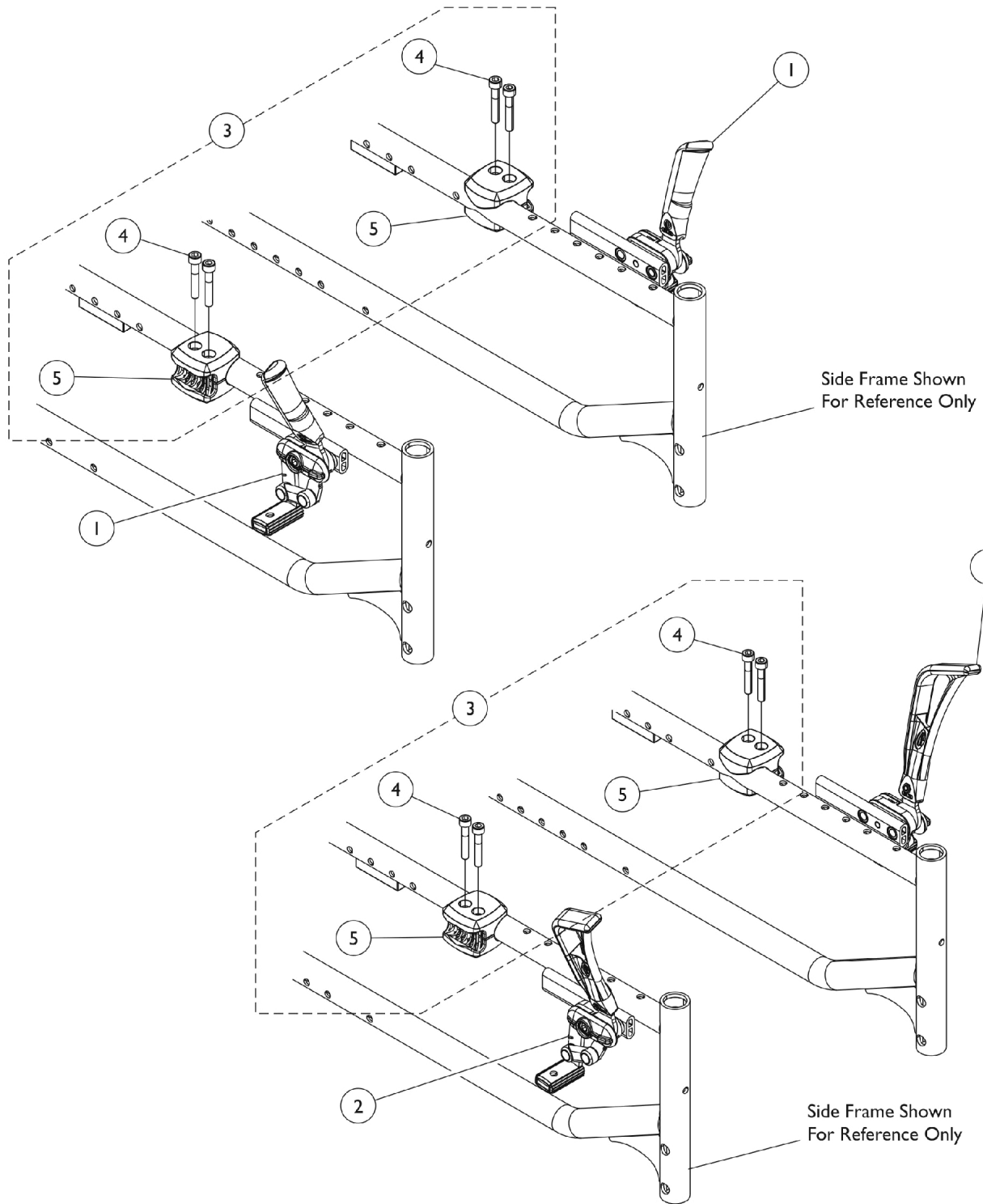

## Wheel Locks - Push to Lock

### Push to Lock (WLPUSH), Push to Lock Extended (WLPUSHX)

| Item Number   | Note | Part Number | Description   | Quantity Per |          |
|---|------|-------------|---|--------------|----------|
|   |      |             |   | Package      | Assembly |
| 1   |      | 1425634     | Wheel Lock Assembly, Push to Lock - Left              | 1            | 1        |
| 1   |      | 1425635     | Wheel Lock Assembly, Push to Lock - Right             | 1            | 1        |
| 2   |      | 1425602     | Wheel Lock Assembly, Push to Lock, Extended - Left    | 1            | 1        |
| 2   |      | 1425603     | Wheel Lock Assembly, Push to Lock, Extended - Right   | 1            | 1        |
|   | A    | 1425642     | Wheel Lock Assembly w/ Hardware, Dual Amputee - Left  | 1            | 1        |
|   | A    | 1425643     | Wheel Lock Assembly w/ Hardware, Dual Amputee - Right | 1            | 1        |
| 3   | B    | 1423107     | Kit, Hardware, Clamp Brake (Pair)                     | 1            | 1        |
| 4   |      |             | Screw (M6 x 1 x 35mm)                                 | 1            | 4        |
| 5   |      |             | Mounting Clamp, Wheel Lock                            | 1            | 2        |
| NOTE: A - Not shown in illustration.<br>B - Includes items 4 and 5. |      |             |   |              |          |