


Seat Frame Mounting Hardware & Interface Mtg. Plates (CGT/CGTR) For TDX SP/SP-GT w/ 22NF Batteries After 7/13/11

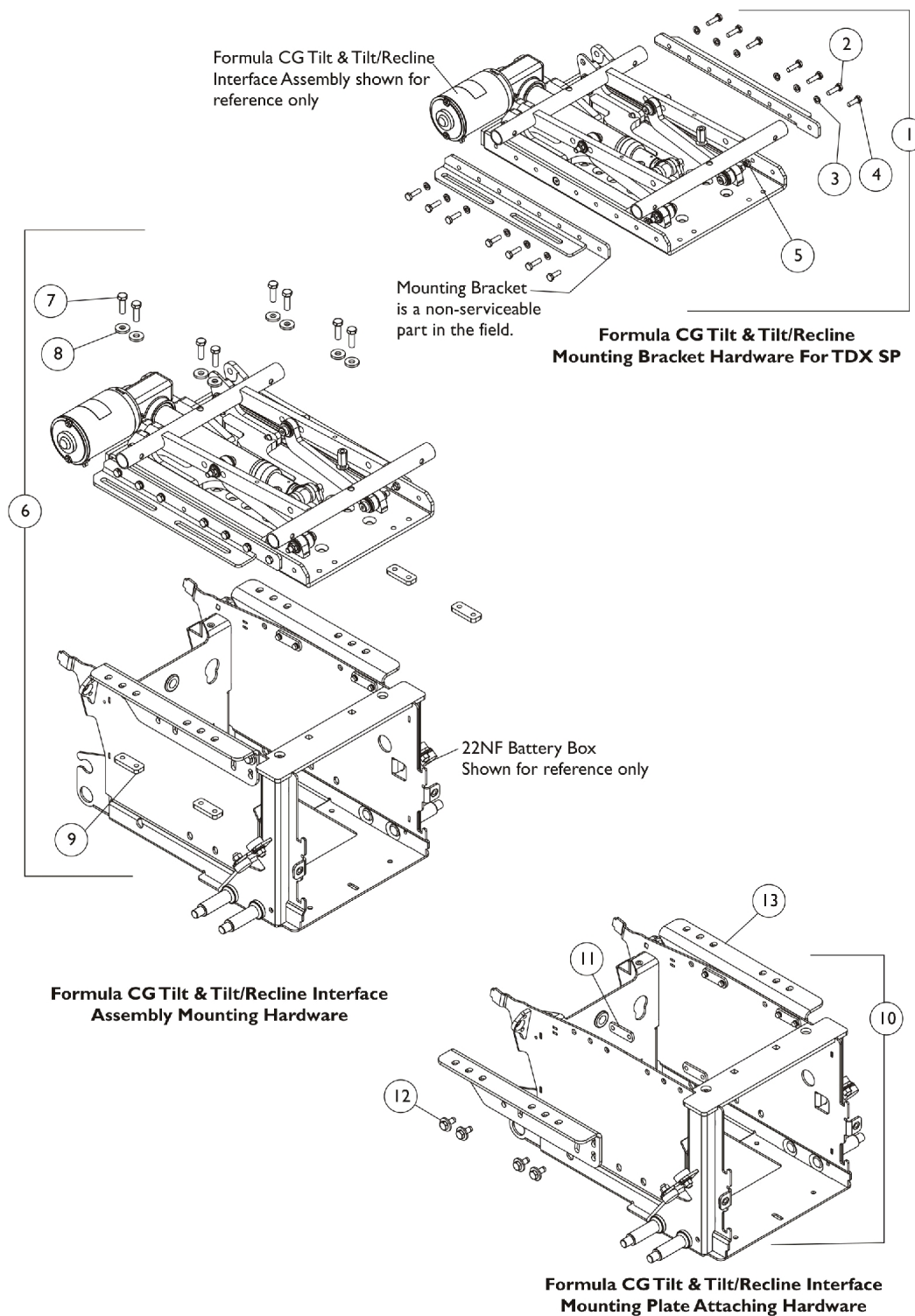

Seat Frame Mounting Hardware & Interface Mtg. Plates (CGT/CGTR) For TDX SP/SP-GT w/ 22NF Batteries After 7/13/11

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1121713-NLA	NLA - Kit, Formula CG Tilt & Tilt/Recline Mounting Bracket Hardware	1	1
2			Screw, Hex Head (1/4-20 x 7/8")	1	12
3			Lockwasher, Split (1/4")	1	12
4			NLA - Bolt, Hex Head Cap (1/4-20 x 3/4")	1	2
5			Locknut (1/4-20)	1	2
6	B	1173256	Kit, Formula CGT/CGR/CGTR Interface Asbly. Mtg. Hdw. (For TDX SP w/ 22NF Batteries)	1	1
7			Screw, Hex Head (5/16-18 x 1")	1	8
8			Washer, Flat (21/64 x 53/64 x 7/64")	1	8
9			Plate, Nut (5/16-18)	1	4
10	C	1121700	Kit, Interface Mounting Plate Attaching Hardware	1	1
11			Nut, Weld (5/16-18 x 1-3/8" Spacing)	1	4
12			Screw, Hex Head SEMS w/ Patch (5/16-18 x 3/4")	1	8
13	D	1173230	Plate, Low Interface Mounting, Black - Right	1	1
13	D	1173231	Plate, Low Interface Mounting, Black - Left	1	1
13	E	1173226	Plate, Medium Interface Mounting, Black - Right	1	1
13	E	1173227	Plate, Medium Interface Mounting, Black - Left	1	1
13	F	1173228	Plate, High Interface Mounting, Black - Right	1	1
13	F	1173229	Plate, High Interface Mounting, Black - Left	1	1
NOTE: A - Includes items 2-5 B - Includes items 7-9 C - Includes items 11 and 12 D - Low Seat-To-Floor = 17.25-18.25" adjustment range. E - Medium Seat-To-Floor = 18-19" adjustment range. F - High Seat-To-Floor = 19-20" adjustment range.					