


Front Riggings - Swingaway Supports Type 75

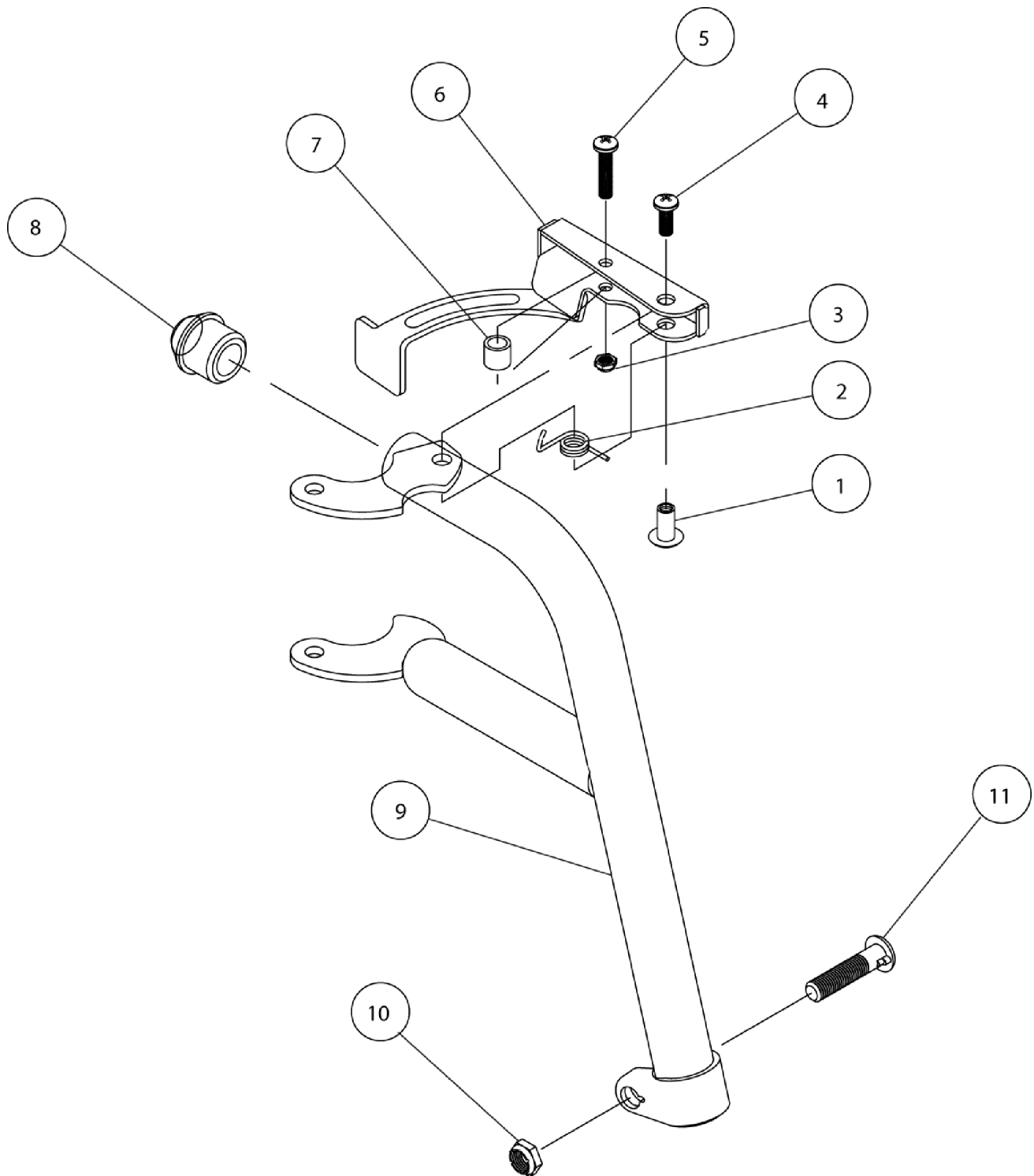

Front Riggings - Swingaway Supports Type 75

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
	A	1037848-NLA	NLA - Footrest Support Assembly, Adult, Black - Right	--	1
	A	1037849-NLA	NLA - Footrest Support Assembly, Adult, Black - Left	--	1
1		1028846-NLA	NLA - Rivnut (#10-32)	1	--
2		1028707	Spring, Release - Right	1	--
2		1028708	Spring, Release - Left	1	--
3		1025735	Locknut (#10-32)	1	--
4		1026114	Screw, Phillips Pan Head (#10-32 x 1/2")	1	--
5		1025716	Screw, Phillips Pan Head (#10-32 x 11/16")	1	--
6		1032721	Handle, Release, Spring - Right	1	--
6		1032722	Handle, Release, Spring - Left	1	--
7		1031884-NLA	NLA - Bushing, Black or Color Chairs (9/32 x 3/8 x 11/32")	1	--
8		1020719	Anti-Rattle, Black	1	--
9	B	1030288-NLA	NLA - Tube, Footrest Support, Adult, Black - Right	1	--
9	B	1030289-NLA	NLA - Tube, Footrest Support, Adult, Black - Left	1	--
10		1025730	Locknut (5/16-24)	1	--
11		1026030	Bolt, Lug (5/16-24 x 1-3/8")	1	--
	C	02303X107-NLA	NLA - Footrest Assembly, Adult, Black, 93BK - Right	1	--
	C	02303X108-NLA	NLA - Footrest Assembly, Adult, Black, 93BK - Left	1	--
NOTE: A - Includes Items 1 thru 11. B - No longer available as a service part individually. Must order a complete right and left side Footrest Assembly as a replacement. Order part numbers 02303X107 and 02303X108. C - Not Shown. Includes a Footrest Support Assembly, Pivot/Slide Tube w/ Deluxe Aluminum Footplate (1450) and Heel Loop attached. Sold complete.					