


Footrest Support Assembly

93-Black, Chrome and Color

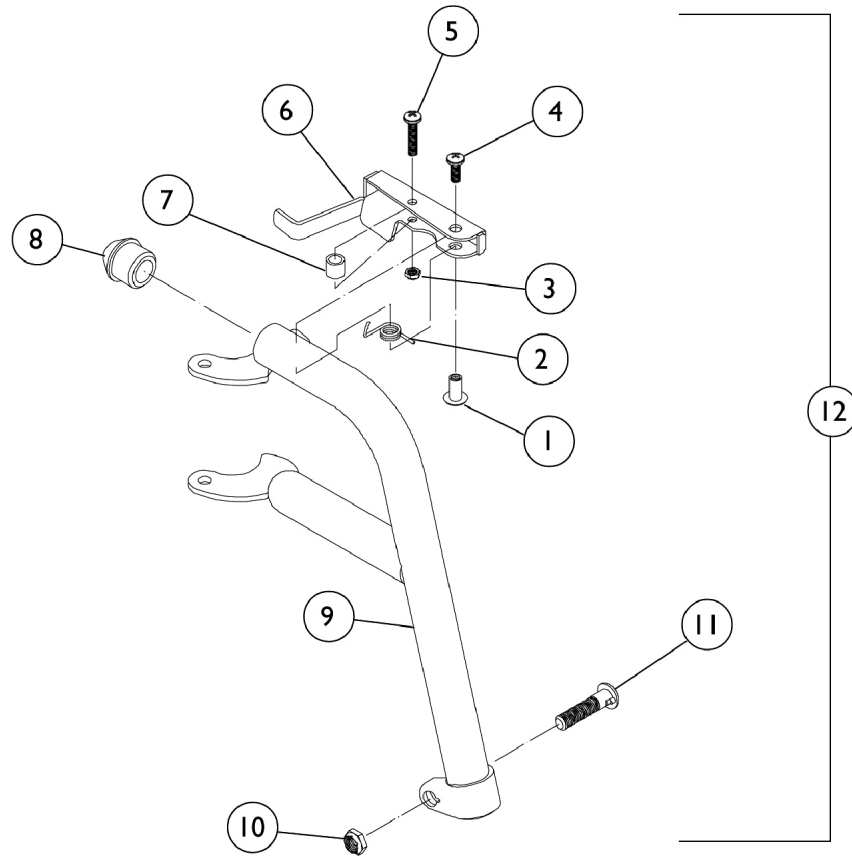

Footrest Support Assembly

93-Black, Chrome and Color

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1028846-NLA	NLA - Rivnut (#10-32)	1	1
2		1028707	Spring, Release - Right	1	1
2		1028708	Spring, Release - Left	1	1
3		7930332-NLA	NLA - Locknut, High Profile (#10-32)	25	1
4		7930335-NLA	NLA - Package, Screw, Phillips Pan Head (#10-32 x 1/2")	25	1
5		7930328-NLA	NLA - Package, Screw, Phillips Pan Head (#10-32 x 11/16")	25	1
6		1035539	Handle, Release, Black - Right	1	1
6		1035540	Handle, Release, Black - Left	1	1
7		47565X057	Bushing (7/32 x 3/8 x 11/32")	1	1
8		1020719	Anti-Rattle, Black	1	1
9	E	1030288-NLA	NLA - Tube, Footrest Support, Adult, Black - Right	1	1
9	E	1030289-NLA	NLA - Tube, Footrest Support, Adult, Black - Left	1	1
9		1020418-NLA	NLA - Tube, Footrest Support, Adult, Chrome - Right	1	1
9		1020419-NLA	NLA - Tube, Footrest Support, Adult, Chrome - Left	1	1
9	A	8881037846-NLA	NLA - Tube, Footrest Support, Adult, Color - Right	1	1
9	A	8881037847-NLA	NLA - Tube, Footrest Support, Adult, Color - Left	1	1
10		1107321	Nut, Low Cap (5/16"-24)	1	1
11		1026030	Bolt, Lug (5/16-24 x 1-3/8")	1	1
	B,C	1028826-NLA	NLA - Tip, Release Handle, Grey	1	1
	B,C	1028823-NLA	NLA - Tip, Release Handle, Teal	1	1
	B,C	1028824-NLA	NLA - Tip, Release Handle, Lavender	1	1
	B,C	1028825-NLA	NLA - Tip, Release Handle, Red	1	1
12	D	1037848-NLA	NLA - Footrest Support Assembly, Adult, Black - Right	1	1
12	D	1037849-NLA	NLA - Footrest Support Assembly, Adult, Black - Left	1	1
12	D	1030708	Footrest Support Assembly, Adult, Silver Vein - Right	1	1
12	D	1030709	Footrest Support Assembly, Adult, Silver Vein - Left	1	1
12	A,D	8881035779	Footrest Support Assembly, Adult, Color - Right	1	1
12	A,D	8881035780	Footrest Support Assembly, Adult, Color - Left	1	1
	C	See Chart	Pivot Slide Tubes with Aluminum Footplates	1	1

Footrest Support Assembly

93-Black, Chrome and Color

NOTE: A - Specify frame color when ordering, see Frame Finishes, Touch-Up Paints, Upholstery Options and Accent Packages page.
 B - Before 10/1/00 only
 C - Not Shown
 D - Assembly includes Release Handle, Tip, Bolt, and Locknut.
 E - No longer available as a service part individually.

Pivot Slide Tubes with Aluminum Footplates

Description	Size	Right	Left
Adult, and Narrow Adult and Large Adult (Rubber Edged Plate) See Note(s) C	(16" to 24")	1028892-NLA See Note(s) A, C	1028893-NLA See Note(s) C, A
Slim Narrow (Plain Aluminum Plate)	(14" and 15")	1058573-NLA See Note(s) B	1058574-NLA See Note(s) B
NOTE: A - Front Riggings built before 12/93 had footplates with a flat black finish. After 11/93 the finish is "Wrinkle Black." For Front Riggings before 12/93, order in pairs to match. B - Aluminum footplates on chairs built before 2/96 had a "Bright Aluminum" finish. Footplates after 1/96 now have a black finish. For Front Riggings before 2/96, order in pairs to match. C - No longer available as a service part individually.			