
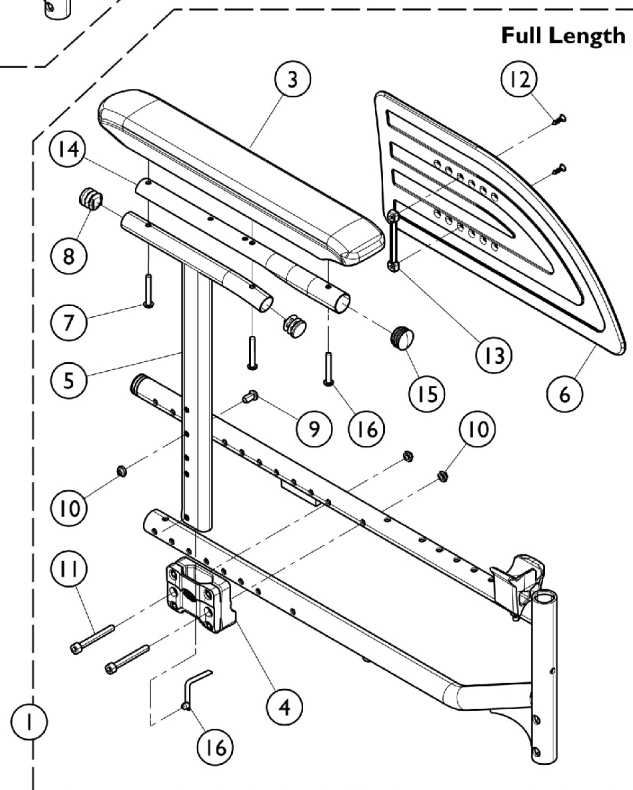


Arms - T Style Adjustable - Myon Active (TAHD, TAHF)

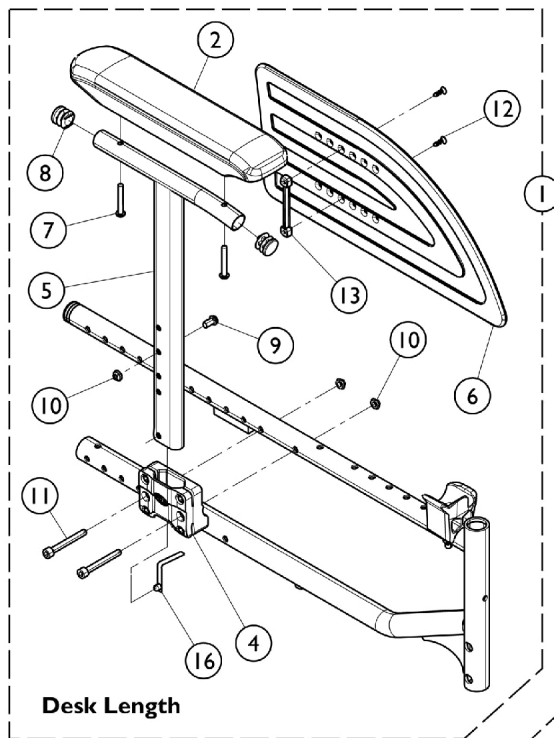

Arms - T Style Adjustable - Myon Active (TAHD, TAHF)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1449977	T-Arm Assembly w/ Hardware, Desk Length - Right	1	1
2			Arm Pad, Desk Length	1	1
4			NLA - Support, Armrest	1	1
5			Tube, Armrest - Right	1	1
6			Guard, Side, Clothing	1	1
7			Screw, Socket, Button Head (M5 x 35mm)	1	2
8			Plug Button, Black (3/4")	1	2
9			Screw, Socket Button Head (M6 x 1 x 12mm)	1	1
10			NLA - Locknut (M6 x 1)	1	3
11			Screw, Socket Head (M6 x 50)	1	2
12			Screw (M40 x 14mm)	1	2
13			Nut, Double, Plastic	1	1
16			Button, Spring	1	1
16	A	0027-10003 1449978	Button, Spring	1	1
1			T-Arm Assembly w/ Hardware, Desk Length - Left	1	1
3			Arm Pad, Desk Length	1	1
4			NLA - Support, Armrest	1	1
5			Tube, Armrest, Black - Left	1	1
6			Guard, Side, Clothing	1	1
7			Screw, Socket, Button Head (M5 x 35mm)	1	2
8			Plug Button, Black (3/4")	1	2
9			Screw, Socket Button Head (M6 x 1 x 12mm)	1	1
10			NLA - Locknut (M6 x 1)	1	3
11			Screw, Socket Head (M6 x 50)	1	2
12			Screw (M40 x 14mm)	1	2
13			Nut, Double, Plastic	1	1
16	B	0027-10003 1449979	Button, Spring	1	1
16			Button, Spring	1	1
1			T-Arm Assembly w/ Hardware, Full Length - Right	1	1
3			Arm Pad, Full Length	1	1
5			Tube, Armrest - Right	1	1
4			NLA - Support, Armrest	1	1
6			Guard, Side, Clothing	1	1
8			Plug Button, Black (3/4")	1	2
7			Screw, Socket, Button Head (M5 x 35mm)	1	3
9			Screw, Socket Button Head (M6 x 1 x 12mm)	1	1
10			NLA - Locknut (M6 x 1)	1	3
11			Screw, Socket Head (M6 x 50)	1	2
12			Screw (M40 x 14mm)	1	2
13	B	0027-10003 1449980	Nut, Double, Plastic	1	1
14			Tube, Support, Armpad, Full Length	1	1
15			Button, Plug (22mm)	1	1
16			Button, Spring	1	1
16			Button, Spring	1	1
1			T-Arm Assembly w/ Hardware, Full Length - Left	1	1
3			Arm Pad, Full Length	1	1
4			NLA - Support, Armrest	1	1

Arms - T Style Adjustable - Myon Active (TAHD, TAHF)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
5			Tube, Armrest, Black - Left	1	1
6			Guard, Side, Clothing	1	1
7			Screw, Socket, Button Head (M5 x 35mm)	1	3
8			Plug Button, Black (3/4")	1	2
9			Screw, Socket Button Head (M6 x 1 x 12mm)	1	1
10			NLA - Locknut (M6 x 1)	1	3
11			Screw, Socket Head (M6 x 50)	1	2
12			Screw (M40 x 14mm)	1	2
13			Nut, Double, Plastic	1	1
14			Tube, Support, Armpad, Full Length	1	1
15			Button, Plug (22mm)	1	1
16			Button, Spring	1	1
16		0027-10003	Button, Spring	1	1
NOTE: A - Includes items 2, 4 - 13, and 16. B - Includes items 3 - 16.					