
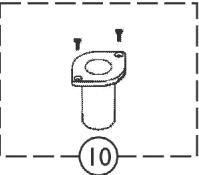
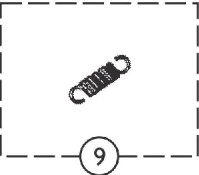
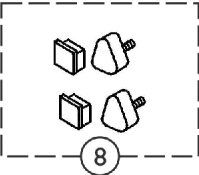
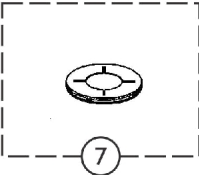
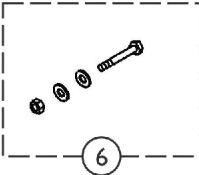
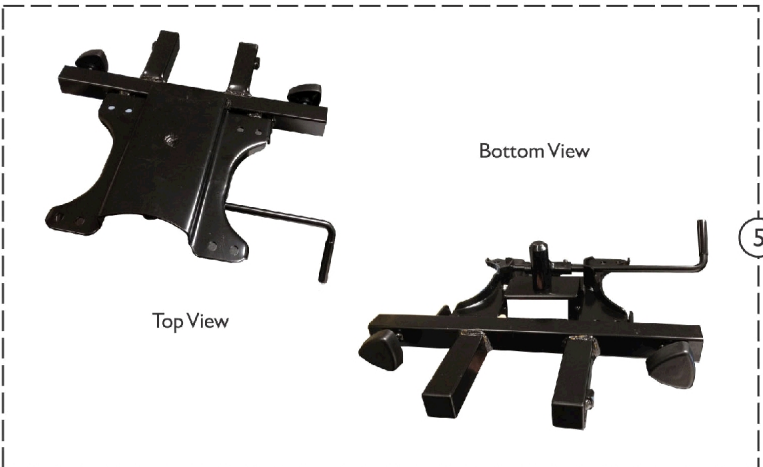
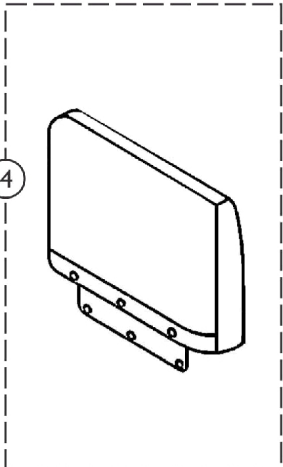
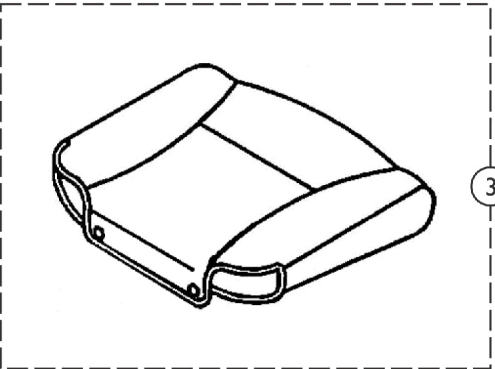


# Seat, Seat Bracket & Arms

(After 10/14/09)

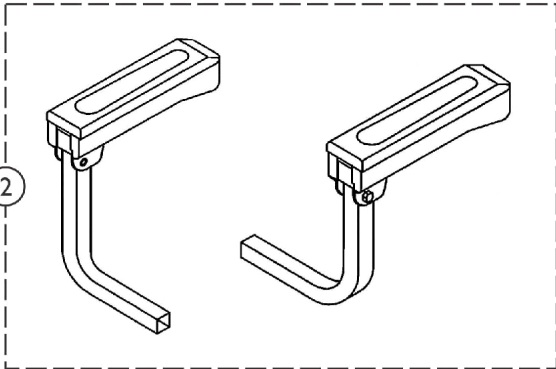

## Seat, Seat Bracket & Arms (After 10/14/09)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1146610-NLA	NLA - Seat and Arm Assembly	1	1
1	B	1164322	Seat and Arm Assembly	1	1
2		1144809	Arm Assembly w/ Arm Pad and Hardware, Black - Right	1	1
2		1144810	Arm Assembly w/ Arm Pad and Hardware, Black - Left	1	1
3		1168466	Cover, Seat	1	1
4		1168465	Cover, Backrest	1	1
5		1148846-NLA	NLA - Seat Bracket Assembly	1	1
5		1169832	Seat Bracket Assembly	1	1
6		1148850-NLA	NLA - Hardware, Seat Post Includes: (1) ea. Nylon Nut, (1) ea. Bolt, & (2) ea. Washers.	1	1
7		1148852	Liner, Shroud/Guide Seat	1	1
8		1148849-NLA	NLA - Kit, Plugs and Knobs Includes: (2) ea. Screws & (2) ea. Plugs.	1	1
9		1144812	Spring, Seat Lever	1	1
10		1144811	Kit, Seat Guide Bracket Includes: (1) ea. Seat Guide Bracket, and (2) ea. Bolts	1	1
NOTE: A - No longer available. See Part Number 1164322 for replacement. B - Includes Items 2, 3, 4, and 5.					